Assembly Instruction


Document No. 270338	Revision A				permobil
Production state FOR SERIES					
Preform Date 2015-08-17	Preform By ROHO	Approved Date 9/18/2015	Approved By GHT	Page US1	

Table of Contents

Contents.....US1
 ToolsUS1
 Installation..... US2-3
 Adjustment.....US4


Contents

TABLE ADJUSTABLE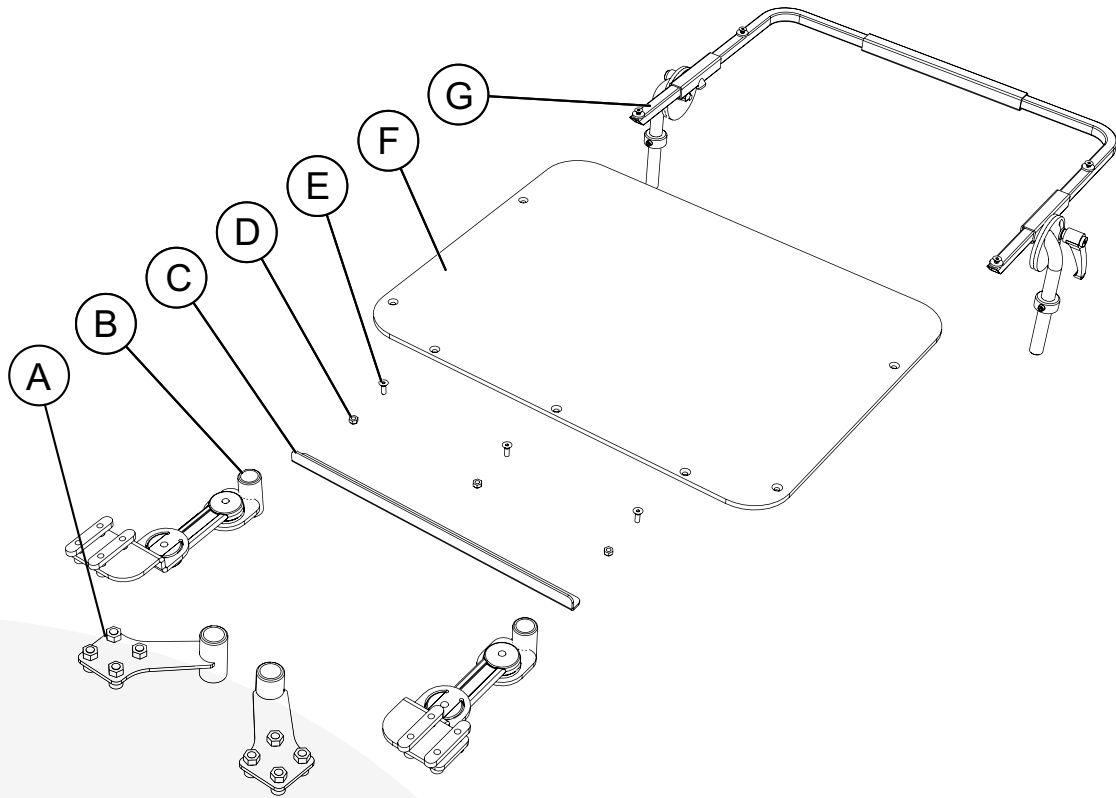


A	Armrest bracket 350/400 mm (alt. 1)	2 pcs		
B	Armrest bracket 350/400 mm (alt. 2)	2 pcs	E	Bolts 3 pcs
C	Shelf	1 pc	F	Table 1 pc
D	Nut	3 pcs	G	Support 1 pc

Tools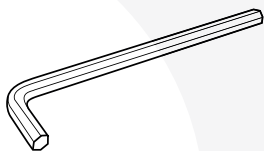

Allen key 5 mm, 4 mm.

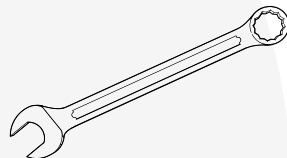
Spanner, 6 mm


Assembly Instruction

Document No. 270338	Revision A				permobil
Production state FOR SERIES					
Preform Date 2015-08-17	Preform By ROHO	Approved Date 9/18/2015	Approved By GHT	Page US2	

1 Begin by bolting the shelf to the table. Use the free nuts and bolts; counterhold the nuts with the aid of a spanner.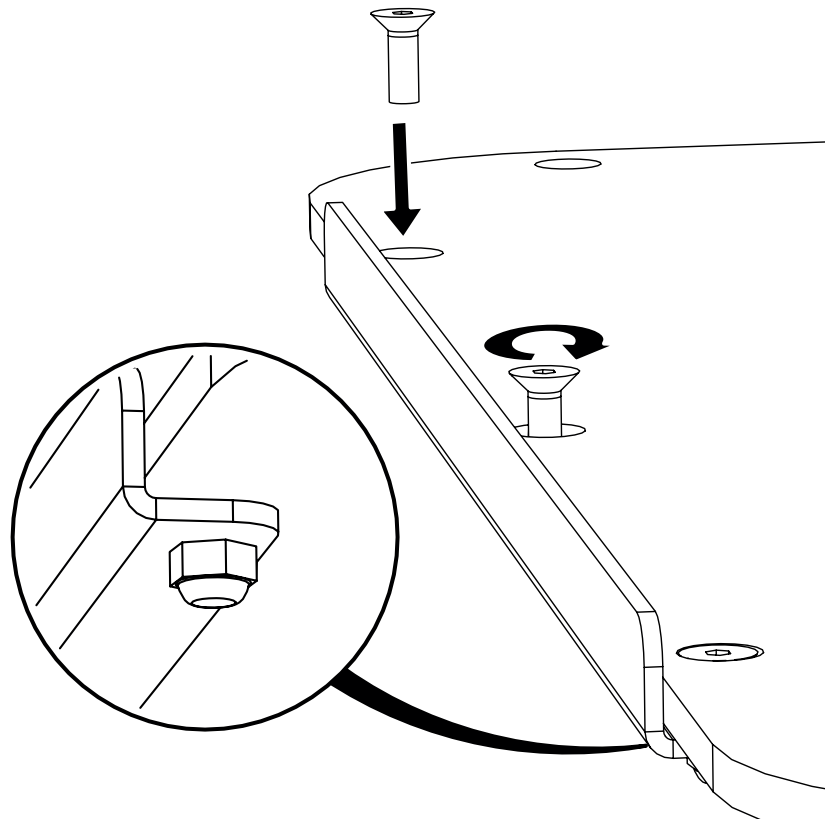



2 Once the shelf is secured, unscrew the bolts in the support and place the table on the rubber spacers. Next, bolt the table to the support.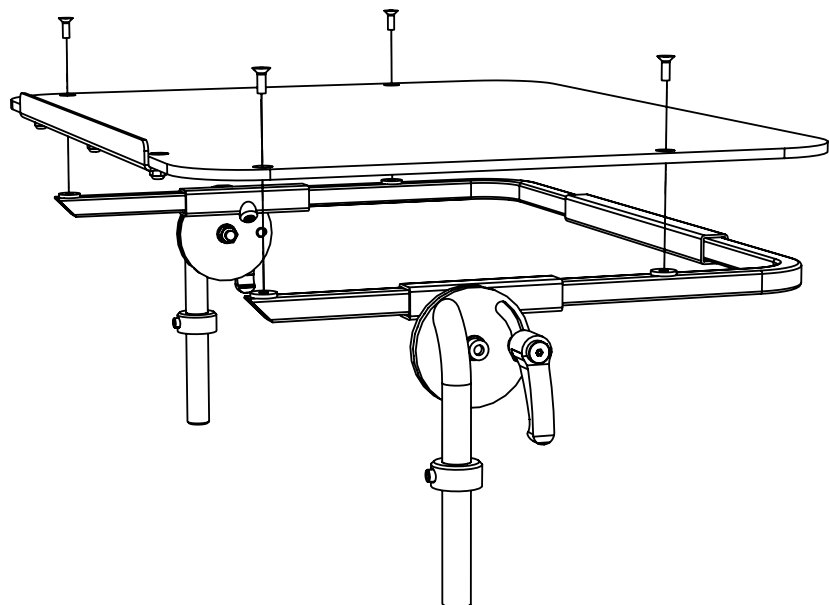


Assembly Instruction

Document No. 270338	Revision A				permobil
Production state FOR SERIES					
Preform Date 2015-08-17	Preform By ROHO	Approved Date 9/18/2015	Approved By GHT	Page US3	

- 3** Slide the armrest bracket into the track under the armrest. Do not tighten the armrest bracket bolts until the table is in place.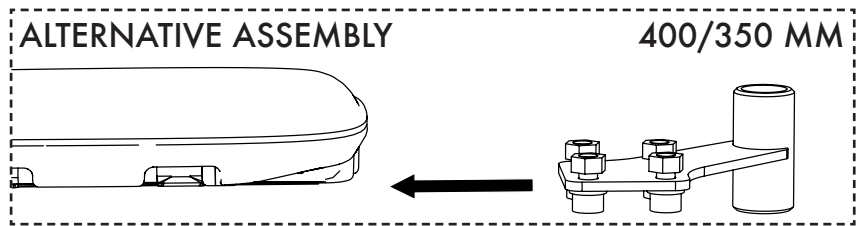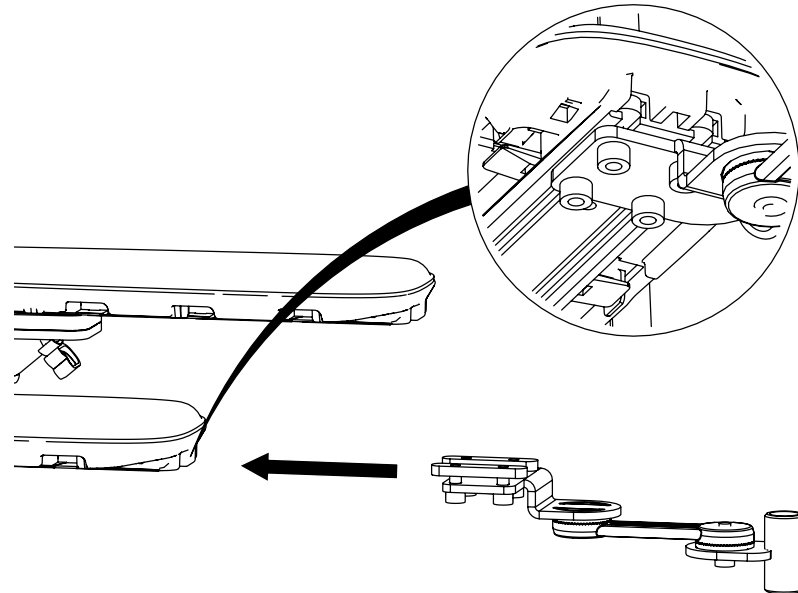


- 4** Slide the support rods into the armrest bracket holders. Adjust the distance between the table and the armrest, then bolt the armrest brackets in the track. Depending on seat width, it may be necessary to adjust the angle of the armrest brackets to get the rods to fit. This is done by loosening the bolts beneath the brackets (this applies only for the 450 armrest bracket).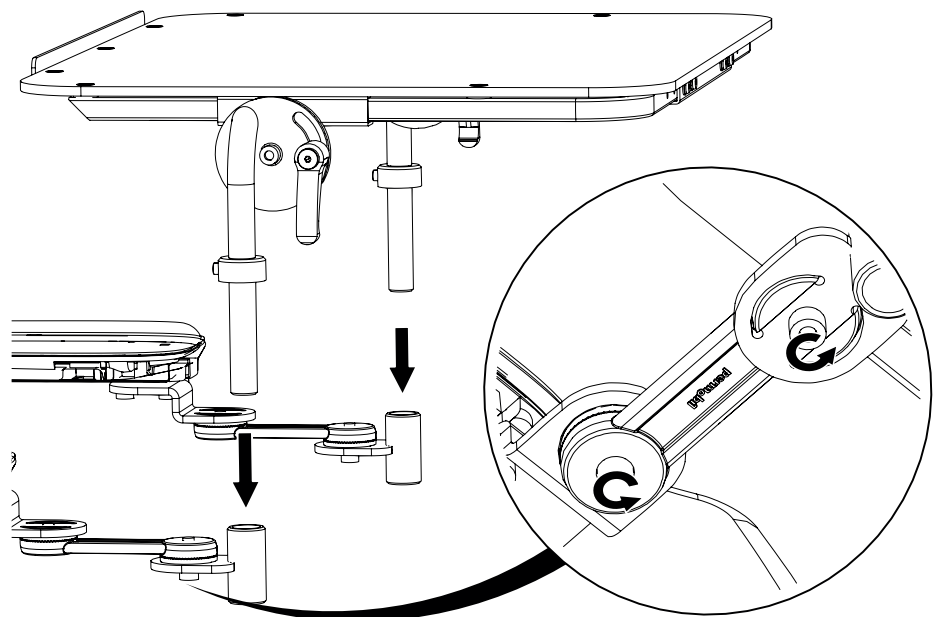


Assembly Instruction

Document No. 270338	Revision A				permobil
Production state FOR SERIES					
Preform Date 2015-08-17	Preform By ROHO	Approved Date 9/18/2015	Approved By GHT	Page US4	

- 1** To adjust table incline, release the two handles on the support rods.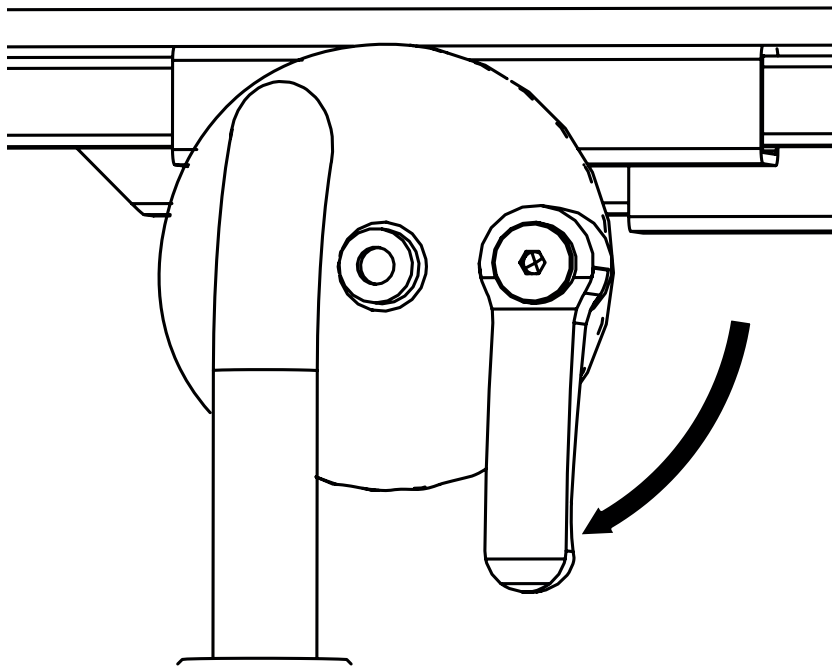

- 2** To adjust the height, undo the stop screws on the rings located on the support rods. Make sure you undo both sides.

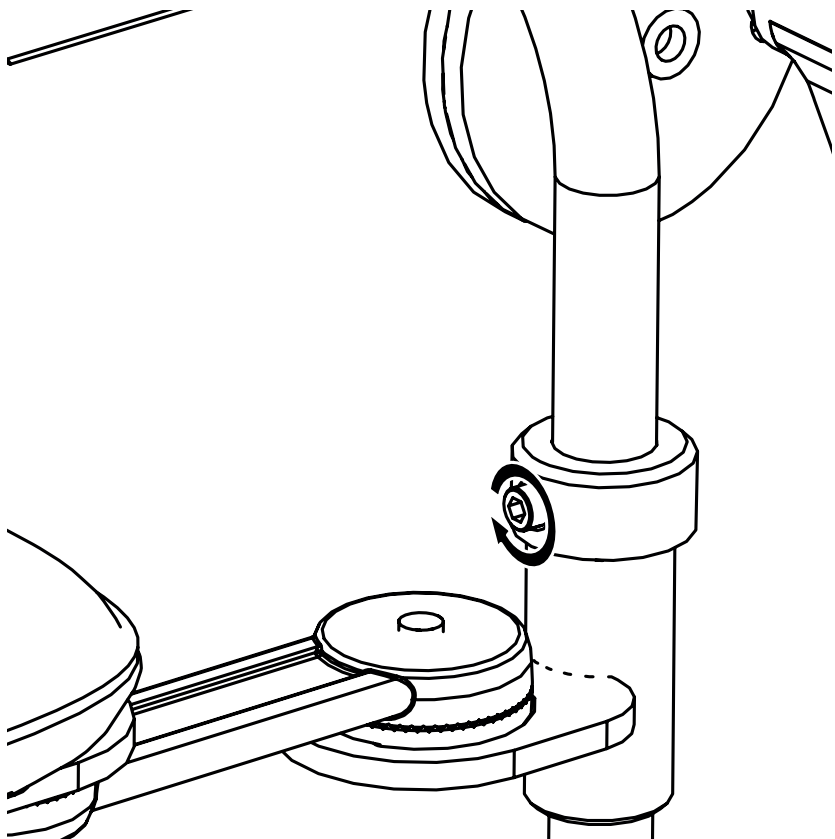Assembly Instruction

Document No. 270338	Revision A				permobil
Production state FOR SERIES					
Preform Date 2015-08-17	Preform By ROHO	Approved Date 9/18/2015	Approved By GHT	Page SE1	

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