

Assembly Instruction


Document No. 270337	Revision A				permobil
Production state SERIES					
Preform Date 2015-08-17	Preform By ROHO	Approved Date 8/31/2015	Approved By GHT	Page US1	

Table of contents

TRUNK SUPPORT


Contents.....US1
 ToolsUS1
 Installation.....US2

Contents

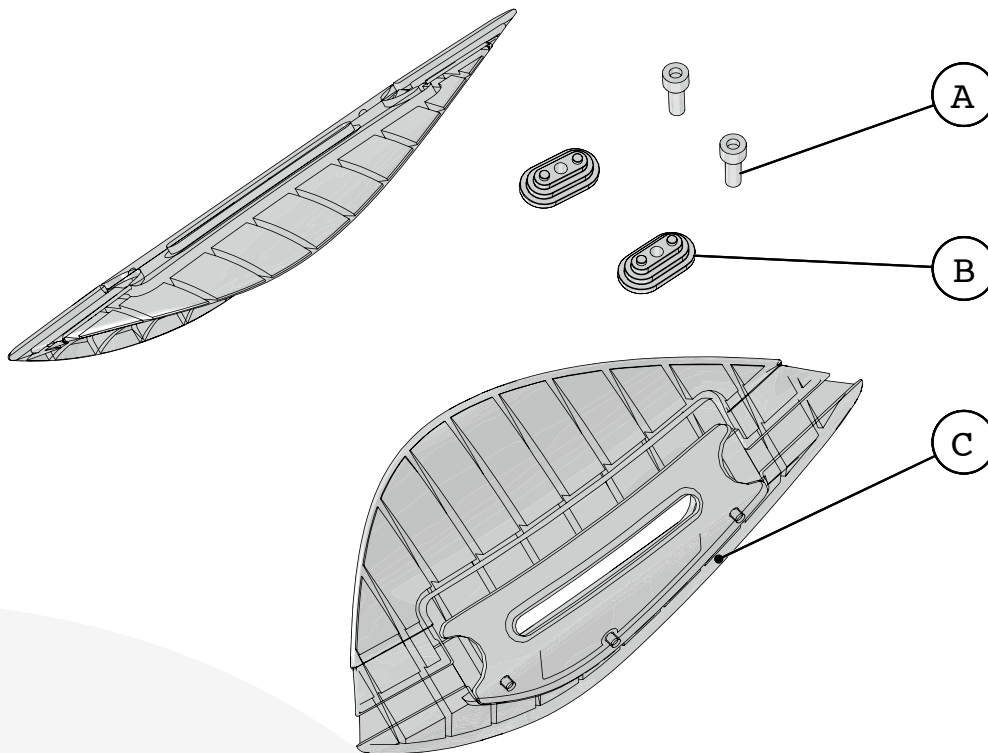

- A Bolt 2 pcs
- B Nut 2 pcs
- C Trunk supports (two sizes) 2 pcs

Tools

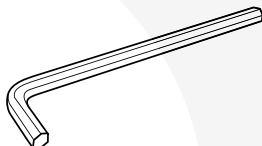


Allen key, 5 mm.


Assembly Instruction

Document No. 270337	Revision A				permobil
Production state SERIES					
Preform Date 2015-08-17	Preform By ROHO	Approved Date 8/31/2015	Approved By GHT	Page US2	

1 Begin by identifying the upper and lower parts of the trunk support. Then twist the trunk support onto the back support's outer part; slot the back support hook into the trunk support track.

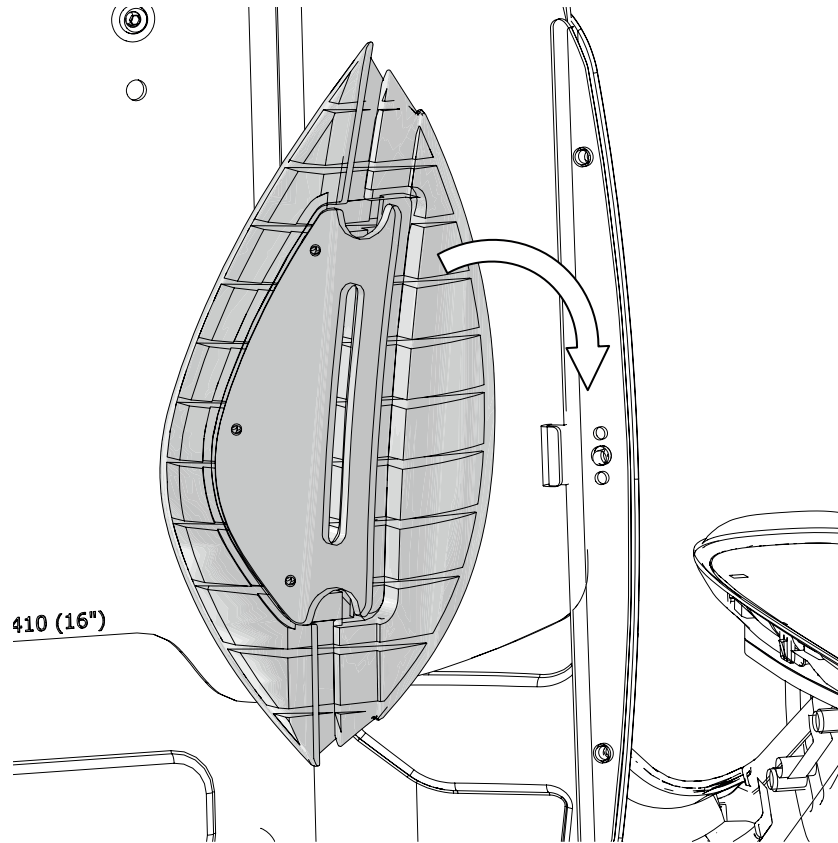

2 Position the nut in the back support hole and tighten it. To adjust trunk support height, undo the bolt until the trunk support can be moved up and down.

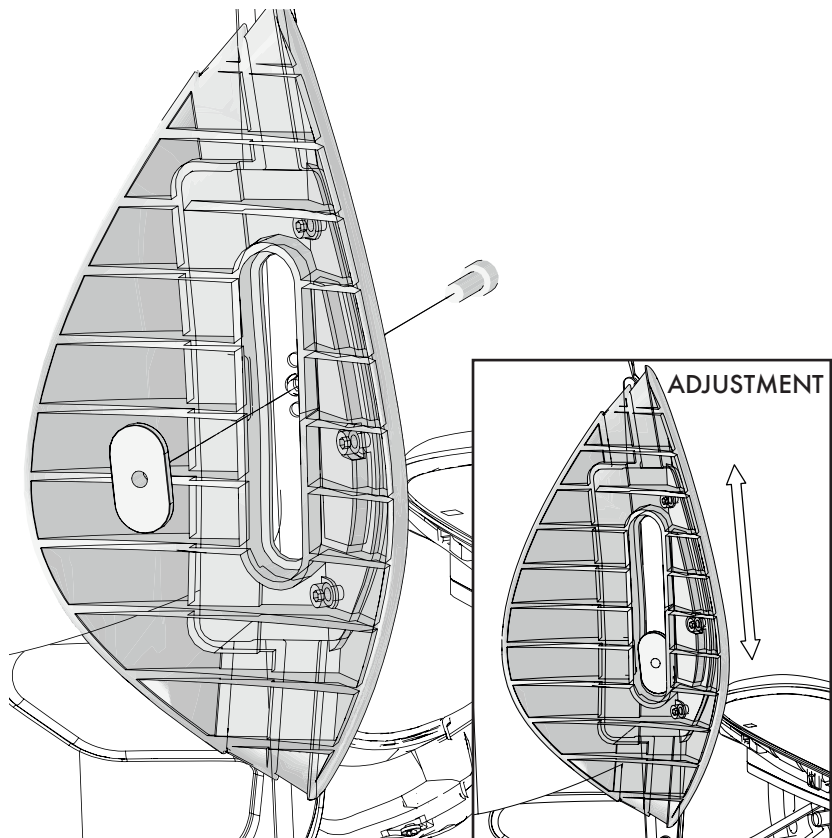

Assembly Instruction


Document No. 270337	Revision A				permobil
Production state SERIES					
Preform Date 2015-08-17	Preform By ROHO	Approved Date 8/31/2015	Approved By GHT	Page SE1	

Innehållsförteckning

BÅLSTÖD


Innehåll	SE1
Verktyg.....	SE1
Montering.....	SE2

Innehåll

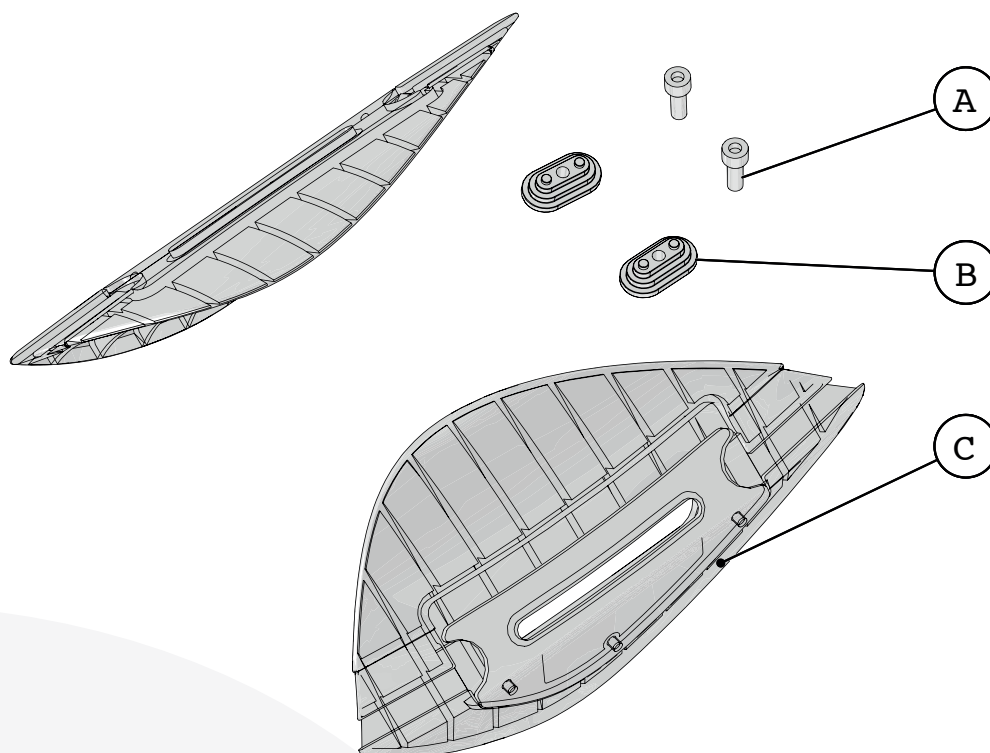

A	Skruv	2 st
B	Mutter	2 st
C	Bålstöd (finns i två storlekar)	2 st

Verktyg

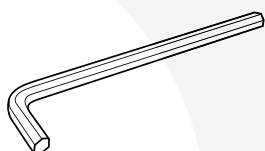


Insexnyckel 5 mm.

Assembly Instruction

Document No. 270337	Revision A				permobil
Production state SERIES					
Preform Date 2015-08-17	Preform By ROHO	Approved Date 8/31/2015	Approved By GHT	Page SE2	

1 Börja med att uppmärksamma vad som är den övre och nedre delen av bålstödet. Vinkla sedan in bålstödet på ryggstödet yttre del, kroken på ryggstödet ska hakas in i bålstödet skena.

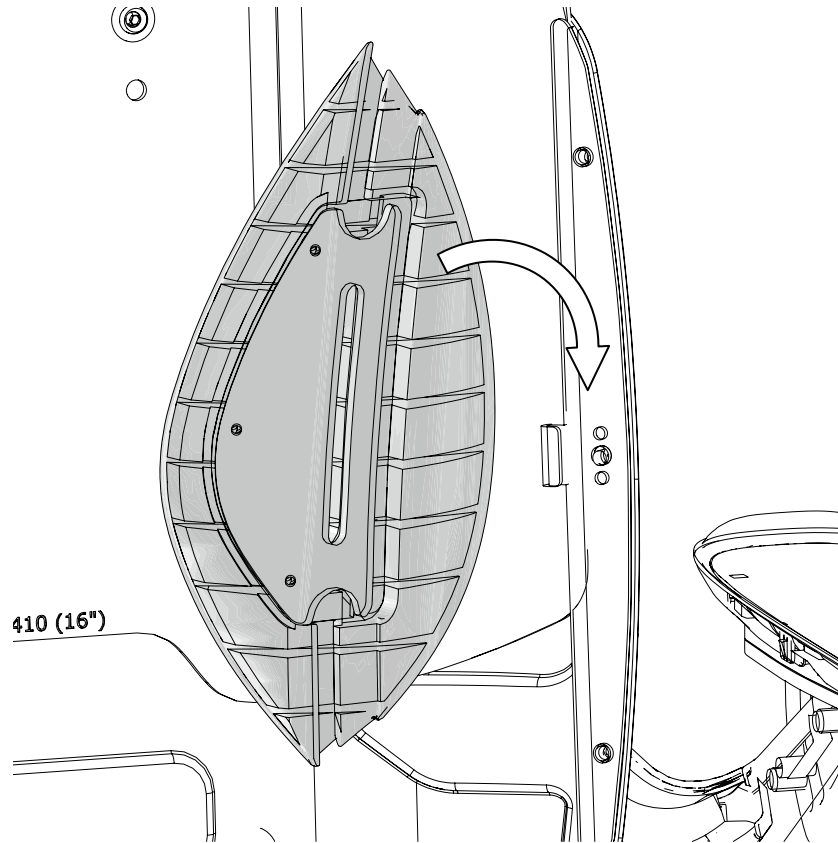

2 Placera in muttern i ryggstödet hål och skruva fast. För att justera höjden på bålstödet lossar du på skruven tills du kan röra bålstödet i höjddled.

