Assembly Instruction

Document No. 260742	Revision A				permobil
Production state SERIES					
Preform Date 2014-03-18	Preform By TSN	Approved Date 3/9/2015	Approved By GHT	Page 1/2	

Installation of Drive wheel

List of parts that should be in the set.

- | | | | | | |
|----|--------|---|----|--------|--|
| 1. | Qty. 1 | Hubcap | 4. | Qty. 1 | Washer, 8 Fe/Zn 5 C1 (TBR SB 8,4x26x5) |
| 2. | Qty. 1 | Screw, ISO 4762 M8x35 10.9 Fe/Zn 5 C2 / Locking coat DIN 267-28 | 5. | Qty. 1 | Washer, PA6 (8,4x15x1,5 type F). |
| 3. | Qty. 1 | Shimwasher DIN 988 8 A2 (DB 8x14x0.5 Stainless) | 6. | Qty. 1 | Wheel |
| | | | 7. | Qty. 1 | Parallel key DIN 6885A 6x6x36 |

⚠ WARNING!

The use of studded tires increases the risk of damage to the surface you drive on. Permobil studded tires are only meant for outdoor use and driving on eg. grass, gravel, asphalt, ice and snow.


Removal of existing wheel

1. Switch off the power supply with the On/Off key on the control panel.
2. Lift up and chock up the wheelchair chassis so that the wheel in question is free of the ground.

⚠ WARNING!

The wheelchair must not be lifted by the seat or seat lift as this can damage the wheelchair and its components.

3. Loosen and remove the hub cap 1, the screw 2 and the washers 3, 4 and 5. See the illustration.
4. Pull the wheel off the axle. Use the removing tool 304103-99-0 if the wheel is hard to remove. See the illustration.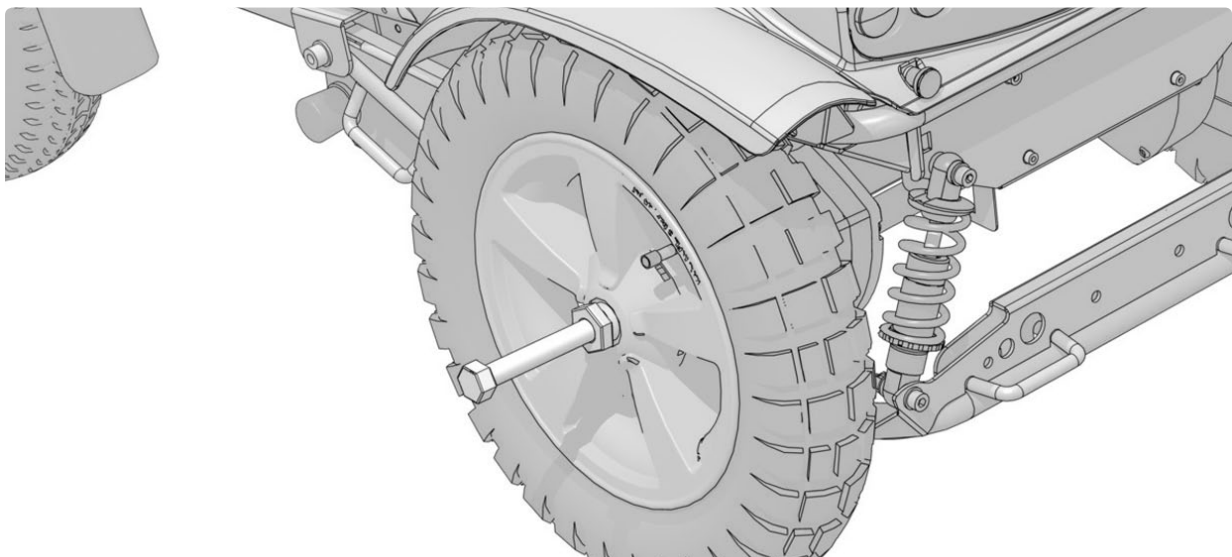

Removing tool 304103-99-0

Assembly Instruction

Document No. 260742	Revision A				permobil
Production state SERIES					
Preform Date 2014-03-18	Preform By TSN	Approved Date 3/9/2015	Approved By GHT	Page 2/2	

Installation of wheel

1. Check that the axle and rim are not damaged. If necessary, clean to remove dirt and rust. Replace damaged parts.
2. Check that the parallel key is properly located and is not damaged. Fit a new parallel key if necessary.
3. Lubricate the axle with a thin layer of copper paste (Würth 0893800x, Art. no.: 1820540).

⚠ WARNING!

The threads on the screw and in the screw hole must not be lubricated. Clean the threads if necessary.

4. Fit the wheel on the axle using just your hands. Check that the wheel is fully located on the axle. If necessary, use a rubber mallet carefully.
5. Fit the four washers 3, 4 and 5 on the screw 2 and tighten the wheel. Fit the hub cap 1. See the illustration.

Tightening torque: 33 Nm.


⚠ WARNING!

Screws must only be used once. Once removed, screws must therefore never be refitted.

No type of screw and washer other than those stated here may be used.

No other type of lock coating or lock liquid may be used.

Do not use an impact wrench for the tightening torque.

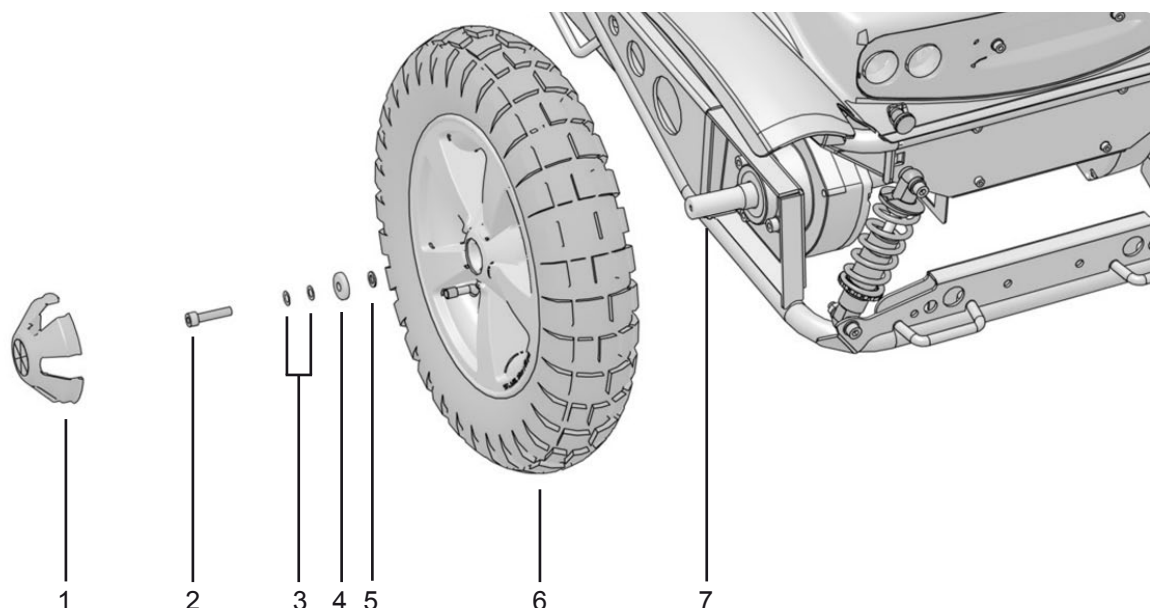Installation of wheel.

Assembly Instruction

Document No. 260742	Revision A				permobil
Production state SERIES					
Preform Date 2014-03-18	Preform By TSN	Approved Date 3/9/2015	Approved By GHT	Page 1/2	

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