Assembly Instruction

Document No. 239005	Revision A				
Production state SERIES					
Preform Date 2009-08-25	Preform By JG	Approved Date 2009-09-11	Approved By GHT	Page 1/2	


Assembly vehicle lock kit for X850*Order no.: 1825145***List of the items included in the set**

- 2 qt. Bosses
- 1 qt. Sticker
- 2 qt. Reinforcement plates (right/left)
- 4 qt. Screws M6x10
- 4 qt. Screws M10x30
- 4 qt. Washers 6.4x14x1.5
- 1 ml. Loctite


**Read the instructions carefully
before starting work**

Assembly

1. Remove the chassis covers and the batteries.
2. Lower the reinforcement plates into the relevant battery compartment. Note that there is a right and left plate. Secure with a washer and screw from the outside of the chair. *Picture 1.*
3. Drip Loctite onto the screws to be threaded into the bosses under the chair. The screws are fitted from the battery compartment. Use a socket head bit for the M10 screws and hold in place using universal or similar around the bosses from the underside of the chair. The screws should be tightened using a torque wrench to 65 Nm. *Picture 2.*
4. The accompanying warning sticker should be applied on the control panel.
5. Replace the batteries and chassis covers.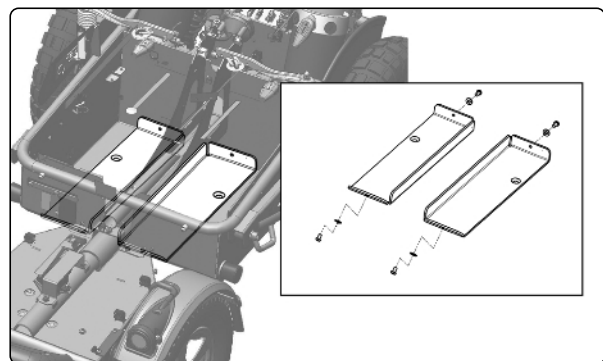

Picture 1. Reinforcement plates with associated screws and washers

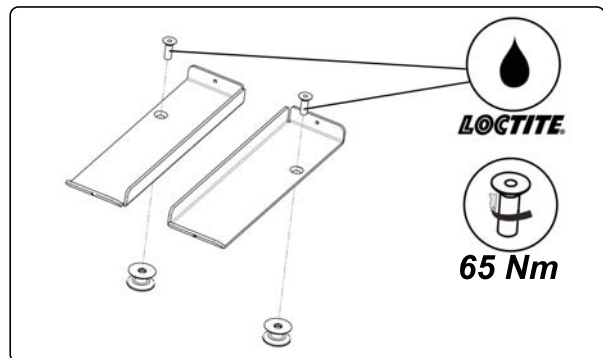Picture 2. Drip Loctite onto the screws and tighten with a torque wrench to 65 Nm

**WARNING**

This product is only intended to be used as a transport lock. Must **NOT** be used for personal transport. Could lead to risk of injury to the user or passengers, or damage to the wheelchair, the Permoblock C3 or the vehicle.

Picture 3. Sticker to be applied on the control panel

Assembly Instruction

Document No. 239005	Revision A				
Production state SERIES					
Preform Date 2009-08-25	Preform By JG	Approved Date 2009-09-11	Approved By GHT	Page 2/2	

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