


**Assembly Instruction**

Document No. 246360	Revision B				
Production state					
Preform Date 2010-12-22	Preform By UDN	Approved Date	Approved By		Page 1

## Mounting head rest bracket

### List of parts used in this instruction.

2 pcs. Groove nuts 2xM6 CC25  
 2 pcs. screws ISO 7380 M6x12  
 2 pcs. screws ISO 10642 M6x12

1 pcs. adjustable bracket  
 1 pcs. adjustable locking lever  
 1 pcs. holder for head rest

### Mounting

Read the instructions carefully *before* starting installation.

1. Undo the screws and slide the bracket's square nuts into the rails and place the bracket in the required position. (Fig. 1)
2. Tighten the screws. (Fig. 2)
3. Push the holder down into the bracket and tighten the locking lever. Check that the holder is secure. (Fig. 3)



Fig. 1. Mounting the bracket.



Fig. 2. Screw the bracket securely in the required position.

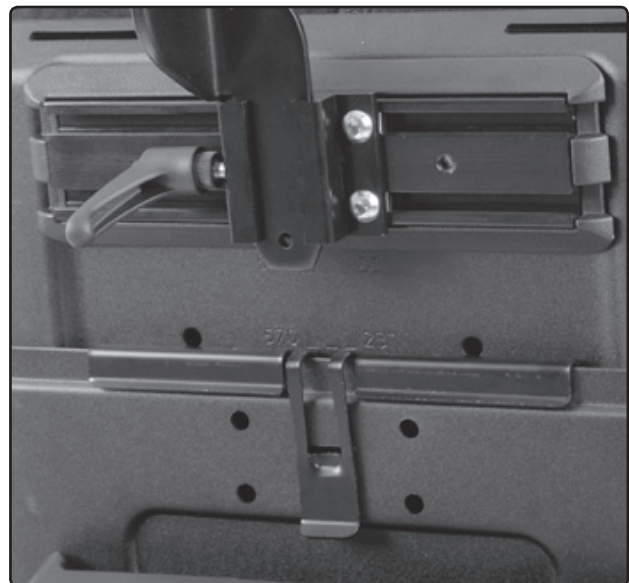



Fig. 3. The holder mounted.

**Assembly Instruction**

Document No. 246360	Revision B				
Production state					
Preform Date 2010-12-22	Preform By UDN	Approved Date	Approved By		Page 2

## Montering av Nackstödsfäste

### Förteckning över detaljer som används i den här instruktionen.

2st	Spårmutter 2xM6 CC25	1st	Justerbart fäste
2st	Skruv ISO 7380 M6x12	1st	Justerbar låsspak
2st	Skruv ISO 10642 M6x12	1st	Hållare nackstöd

## Montering

Läs igenom anvisningen *innan* montering påbörjas.


1. Lossa skruvarna och skjut i fästets fyrkantsmuttrar i skenorna och placera fästet i önskad position. *(Bild 1)*
2. Dra åt skruvarna. *(Bild 2)*
3. Skjut ner hållaren i fästet och dra åt låsspaken, kontrollera att hållaren sitter fast. *(Bild 3)*



*Bild 1. Montera fästet.*



*Bild 2. Skruva fast fästet i önskat läge.*

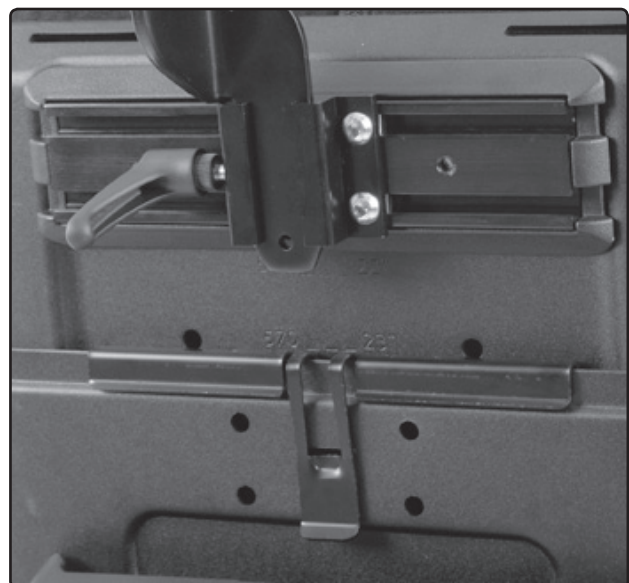

*Bild 3. Hållaren monterad.*