Assembly Instruction

Document No. 247403	Revision A.00					
Production state						
Preform Date 2011-05-12	Preform By TSN	Approved Date	Approved By		Page 1	

Mounting of Drive motor cover with indicator/reflector

List of parts in the set.

Qty. 1	Drive motor cover with indicator/reflector	Qty. 2	Screw ISO 4762 M4x10 8.8 Fe/Zn
Qty. 1	Brake Release Cover	Qty. 1	Screw ISO 4762 M4x5 8.8 Fe/Zn

Assembly

1. Remove the front and upper chassis covers. See the chassis service manual for detailed information.
2. Fit the brake release cover with the attached screw (M4x5). See fig. 1.

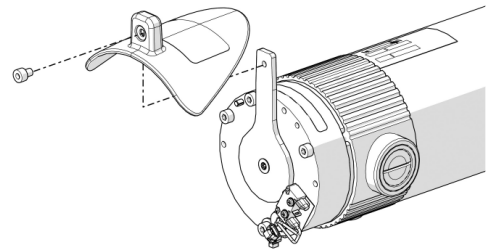
Fig. 1. The Brake Release Cover is fitted with one screw.

3. Fit the drive motor cover with the two attached screws. See fig. 2. On cover with indicator, position it's cabling together with the drive motor cabling in to the wheelchairs chassis.

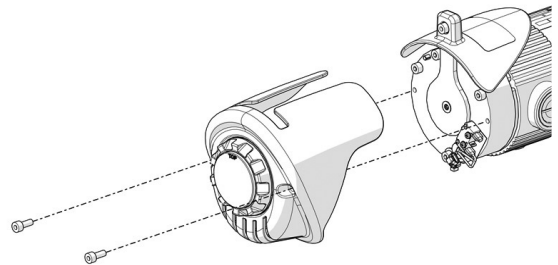
Fig. 2. The drive Motor Cover is fitted with two screws.

4. When mounting cover with indicator, connect the indicators cabling to the contact on the cabling. This is positioned on the inside of the chassis, next to the seat lift. See fig.
5. Refit the front and upper chassis covers. See the chassis service manual for detailed information.

Fig. 3. The connectors on the front light and indicators cabling.

Assembly Instruction

Document No. 247403	Revision A.00					
Production state						
Preform Date 2011-05-12	Preform By TSN	Approved Date	Approved By		Page 2	

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