Assembly Instruction

Document No. 223317	Revision B				
Production state SERIES					
Preform Date 2006-06-14	Preform By JG	Approved Date 2006-12-12	Approved By GHT	Page 1/6	

Assembly Instruction Cup Holder

Order no.: 315249-XX-X

List of parts that should be in the set

See page 3.

Mounting

Read through these instructions before starting assembly.

1. You can find two mounting slot on the cup holder upholstery. Use one of the holes depending how deep you want to have the upholstery. The upper hole gives a deeper upholstery, and the lower hole gives a shallower upholstery. (Fig. 1).
2. Push the holders bracket through either the upper or lower mounting slot. (Fig. 2).
3. Fold the upholstery and put the lower part of the upholstery through the holders ring. (Fig. 3).
4. Fold the upper part of the upholstery around the holder. (Fig. 4).
5. Assembled cup holder with the Permobil logo turned outwards. (Fig. 5).

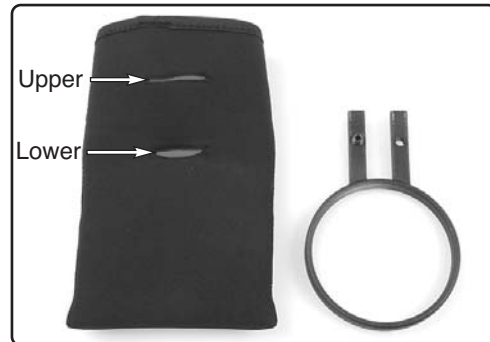
Fig. 1. Upholstery and holder

Fig. 2. Push the holders bracket through any of the slots.

Fig. 3. Put lower part of the upholstery through the holder.

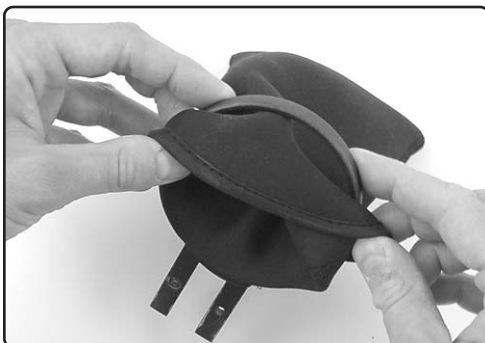
Fig. 4. Fold the upper part of the upholstery around the holder.

Fig. 5. Assembled cup holder.

Assembly Instruction

Document No. 223317	Revision B				
Production state SERIES					
Preform Date 2006-06-14	Preform By JG	Approved Date 2006-12-12	Approved By GHT	Page 2/6	

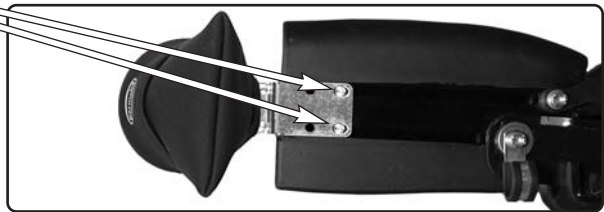
Cup Holder mounted on RS-seat

- A. Detach the rounded front edge on the armrest by loosening the set screw in the mounting tongue, then pull it straight out. Push the cup holder into the armrest unitrack.

If mounting the phone holder:

- B. Position the cup holder and bracket on the side or on the front edge of the armrest. Secure the cup holder by tightening the bolts. See below.


Cup Holder mounted on Corpus-seat

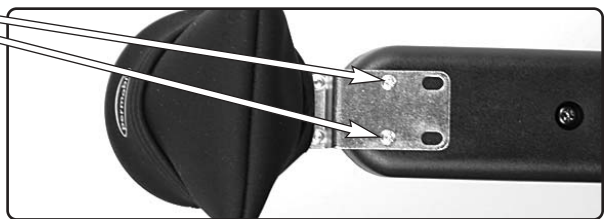- 1. Use the bracket as a template to mark the placement of the screws on the armrest. Drill with a 2 mm drill. Secure the bracket on the armrest with a screw.

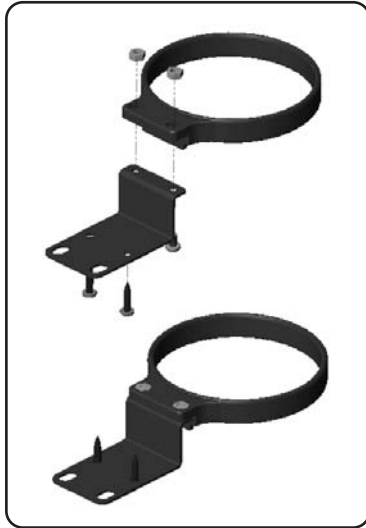
Cup Holder mounted on Captain-seat

- 1. Use the bracket as a template to mark the placement of the screws on the armrest. Drill with a 2 mm drill. Secure the bracket on the armrest with a screw.


Assembly Instruction


Document No. 223317	Revision B				
Production state SERIES					
Preform Date 2006-06-14	Preform By JG	Approved Date 2006-12-12	Approved By GHT	Page 3/6	

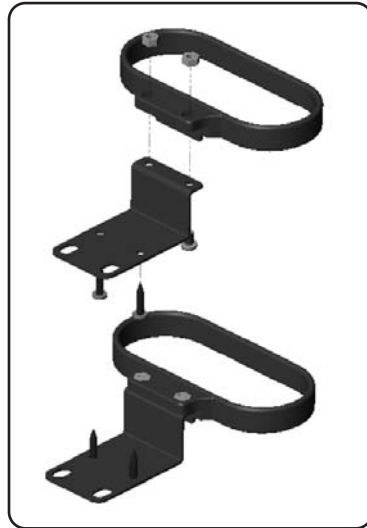
Captain (Cup)

Captain (Phone)

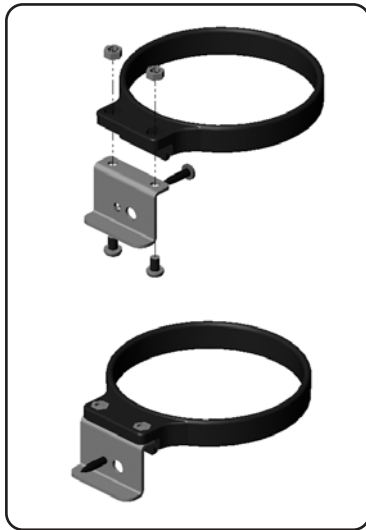
Corpus (Cup)

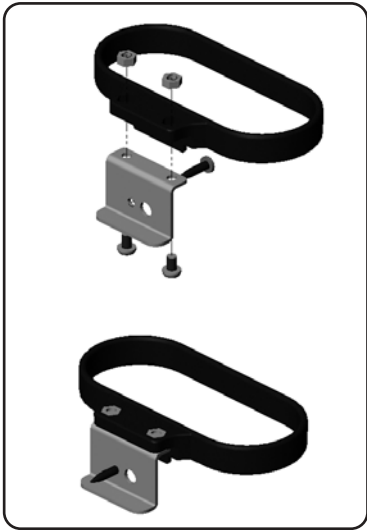Corpus (Phone)

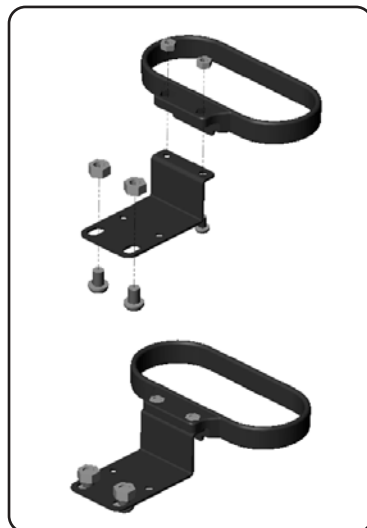RS (Cup) Front edge on armrest

RS (Phone) Side or front edge on armrest

Assembly Instruction



Document No. 223317	Revision B				
Production state SERIES					
Preform Date 2006-06-14	Preform By JG	Approved Date 2006-12-12	Approved By GHT	Page 4/6	

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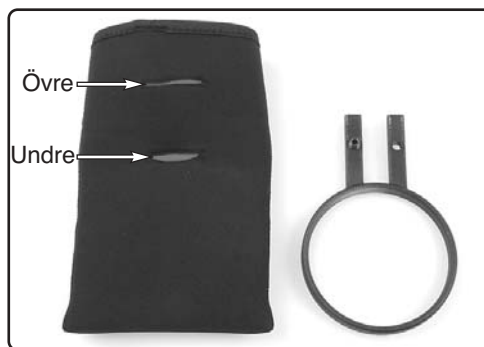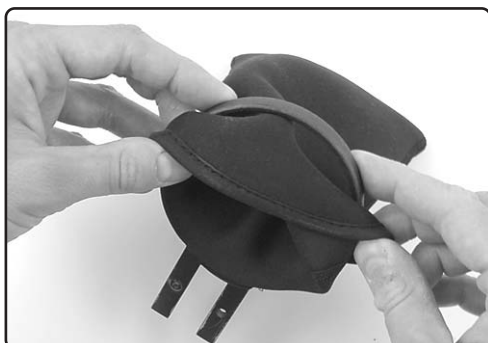Assembly Instruction


Document No. 223317	Revision B				
Production state SERIES					
Preform Date 2006-06-14	Preform By JG	Approved Date 2006-12-12	Approved By GHT	Page 5/6	

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
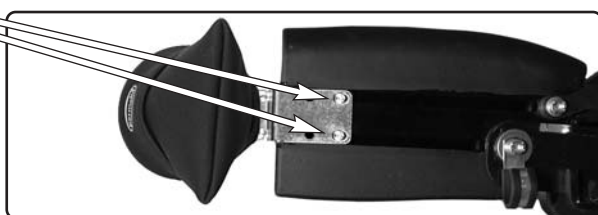
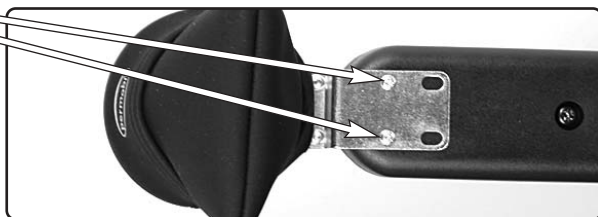
Assembly Instruction


Document No. 223317	Revision B				
Production state SERIES					
Preform Date 2006-06-14	Preform By JG	Approved Date 2006-12-12	Approved By GHT	Page 6/6	

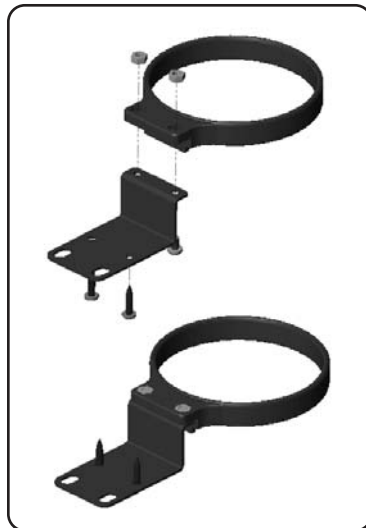
Captain (Mugg)

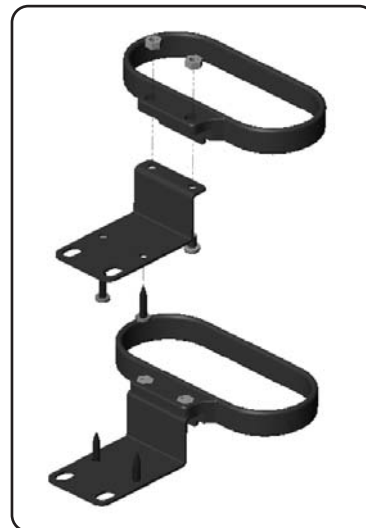Captain (Telefon)

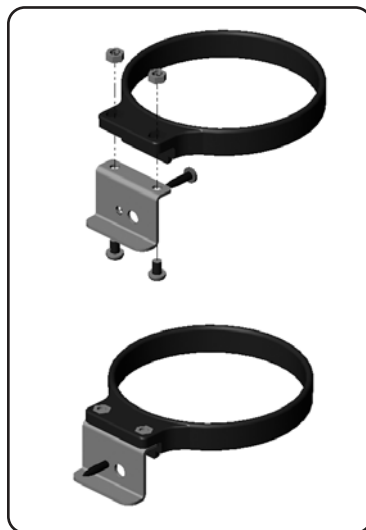
Corpus (Mugg)

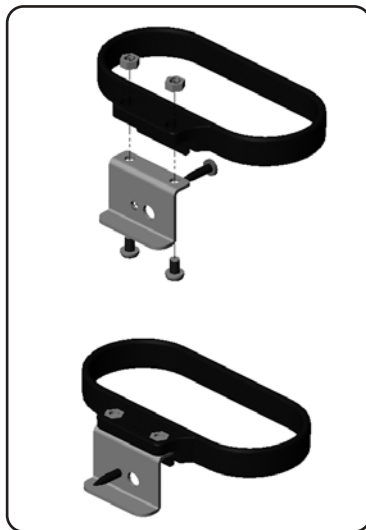Corpus (Telefon)

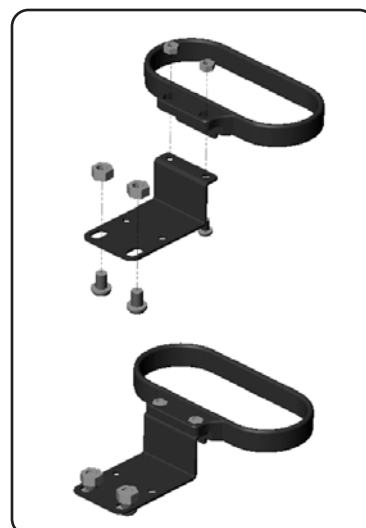RS (Mugg) Framkant på armstöd

RS (Telefon) Framkant el. sida på armstödet