
Assembly Instruction

Document No. 269429	Revision A				permobil <small>THE POWER OF MOBILITY</small>
Production state FOR SERIES					
Preform Date 2015-06-11	Preform By ROHO	Approved Date 7/7/2015	Approved By GHT	Page US1	

Table of contents


Contents..... SE1
 Tools SE1
 Installation..... SE2-3
 Adjustment..... SE4

Contents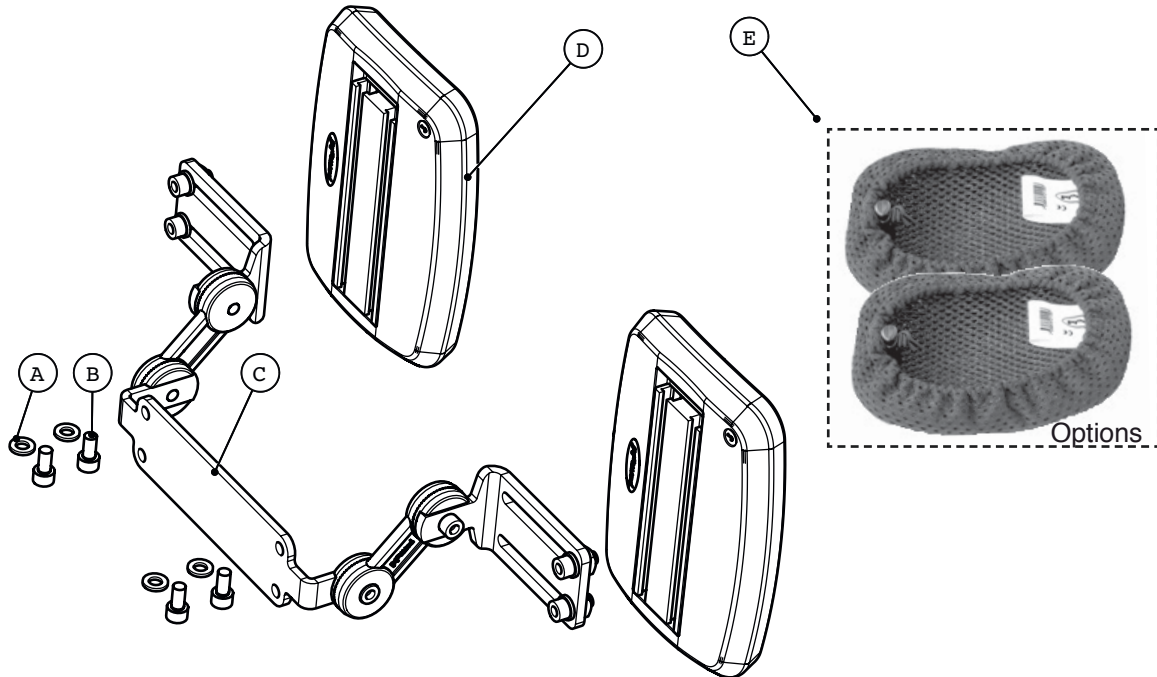


- A Washers 4 pcs
- B Screws 4 pcs
- C Calf support mount 1 pcs
- D Calf supports (available in different variants) 2 pcs
- E Calf support upholstery (optional) 2 pcs

Tools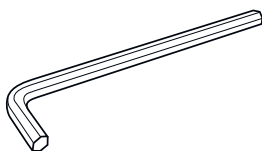

Allen key, 5 mm.

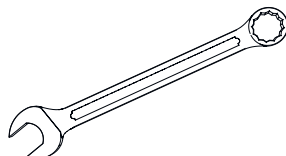
Spanner, 10 mm.


Document No. 269429	Revision A				permobil THE POWER OF MOBILITY
Production state FOR SERIES					
Preform Date 2015-06-11	Preform By ROHO	Approved Date 7/7/2015	Approved By GHT	Page US2	

Assembly


- ① Start by undoing the screws from the slot nuts located on the calf support mount. Use the spanner to hold the slot nuts in place.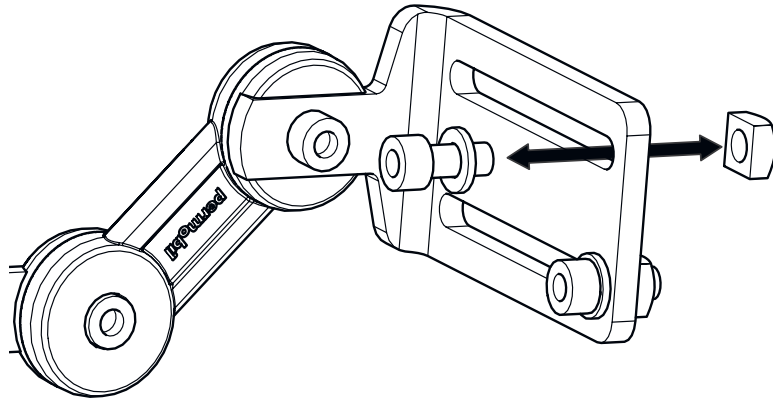

- ② Insert the slot nuts into the slots on the rear of the calf supports.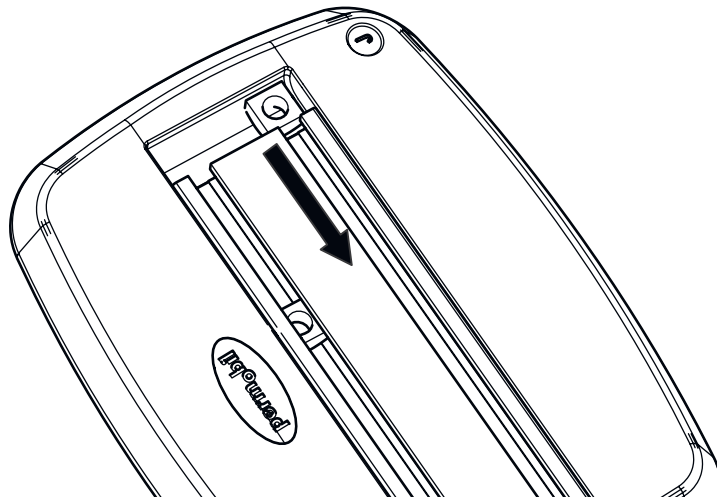


- ③ Screw the calf support mount with washers into the slot nuts.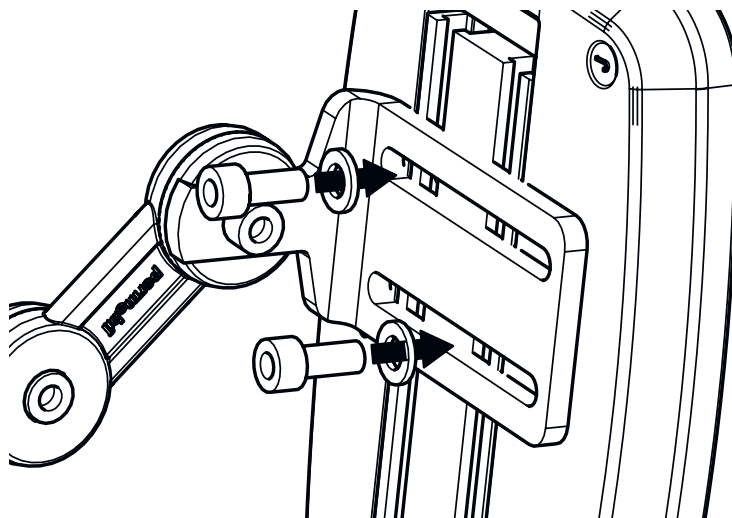


Document No. 269429	Revision A				permobil THE POWER OF MOBILITY
Production state FOR SERIES					
Preform Date 2015-06-11	Preform By ROHO	Approved Date 7/7/2015	Approved By GHT	Page US3	

Assembly


- ④ Screw the calf support mount into the rear of the leg supports.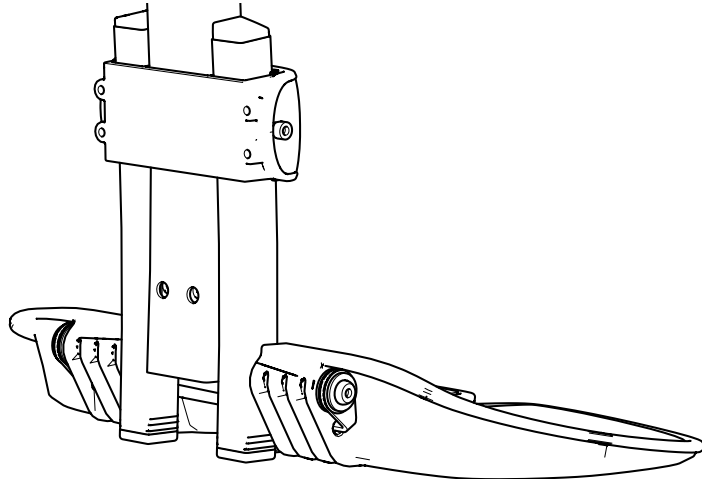

- ⑤ Use the screws and washers supplied in the kit.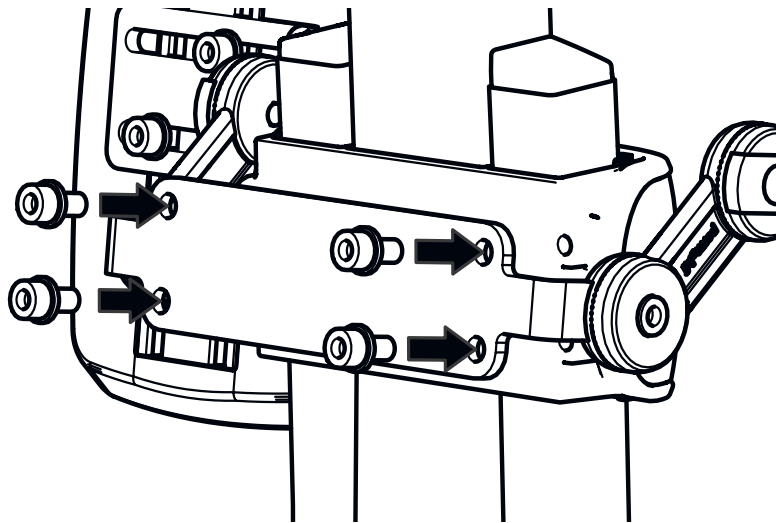


- ⑥ The calf support mount is now in position.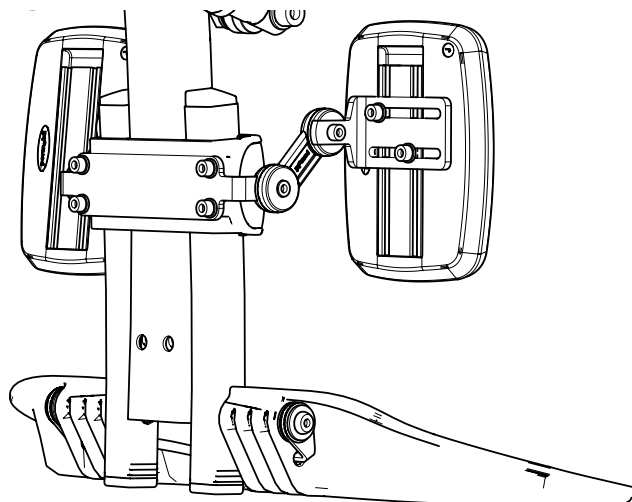


Document No. 269429	Revision A				permobil THE POWER OF MOBILITY
Production state FOR SERIES					
Preform Date 2015-06-11	Preform By ROHO	Approved Date 7/7/2015	Approved By GHT	Page US4	

Adjustment


- ① To adjust the calf support mount vertically and in terms of depth, loosen the screws on the arm.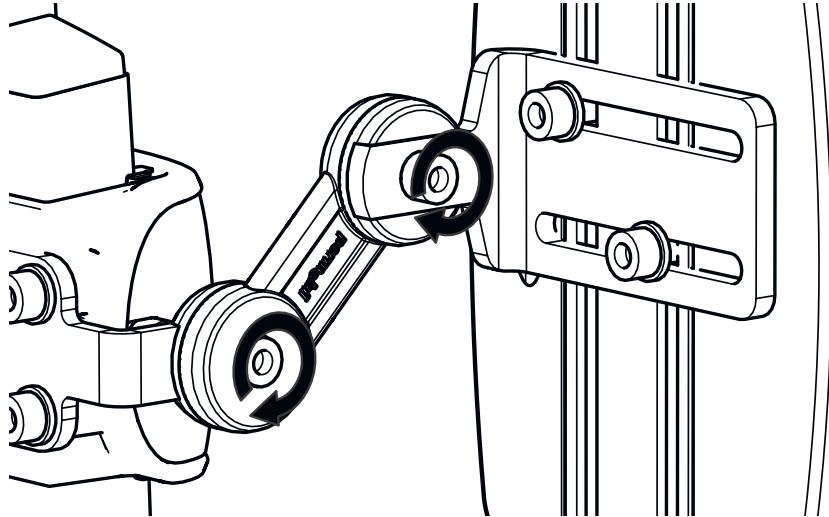

- ② To adjust horizontally and to angle the calf support, loosen the screws in the slot nut.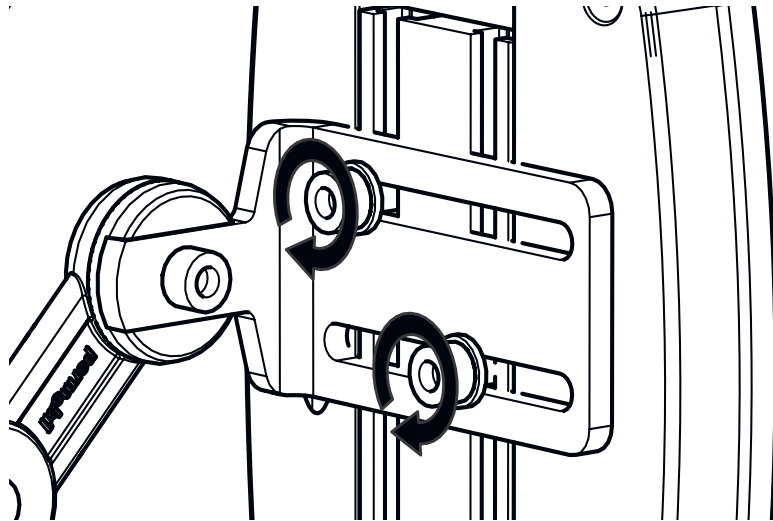

- ③ Adjust until the support feels comfortable, then tighten the screws.

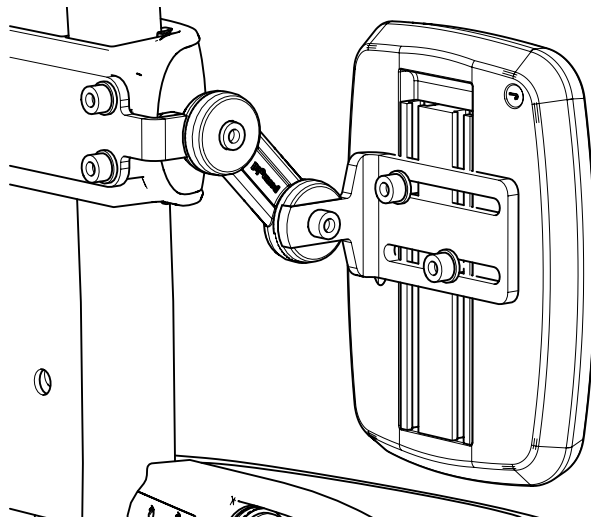Document No. 269429	Revision A				permobil
Production state FOR SERIES					
Preform Date 2015-06-11	Preform By ROHO	Approved Date 7/7/2015	Approved By GHT	Page SE1	

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