**Assembly Instruction**

Document No. <b>228678</b>	Revision <b>E</b>				
Production state <b>SERIES</b>					
Preform Date <b>2007-04-25</b>	Preform By <b>BTA</b>	Approved Date <b>11/18/2013</b>	Approved By <b>GHT</b>	Page <b>1</b>	

## Amputation support

### List of parts that should be in the set

**Always included:**

- 1 pc Bracket.
- 2 pcs Nuts, M6, for mounting cushion.
- 2 pcs Screws, M6x14, for mounting cushion.
- 2 pcs Washers, 6mm, for mounting cushion.

**Depending on model, the following will be included:**

- 1 pc Cushion, A.
- 1 pc Cover for cushion A.
- 1 pc Cushion, D.
- 1 pc Cover for cushion D.
- 1 pc Cushion, H.
- 1 pc Cover for cushion H.
- 1 pc Cushion, J.
- 1 pc Cover for cushion J.

### Read the instructions carefully *before* starting installation

*Fig. 1. Cushion A. The set with order number 103061-99-0 contains cushion and bracket, Fig. 6. Order number 103067-99-0 also contains a cushion cover.*

*Fig. 2. Cushion D. The set with order number 103066-99-0 contains cushion and bracket, Fig. 6. Order number 103071-99-0 also contains a cushion cover.*

*Fig. 3. Cushion H. The set with order number 103065-99-0 contains cushion and bracket, Fig. 6. Order number 103070-99-0 also contains a cushion cover.*

**Assembly Instruction**

Document No. <b>228678</b>	Revision <b>E</b>				
Production state <b>SERIES</b>					
Preform Date <b>2007-04-25</b>	Preform By <b>BTA</b>	Approved Date <b>11/18/2013</b>	Approved By <b>GHT</b>	Page <b>2</b>	


*Fig. 4. Cushion J. The set with order number 103064-99-0 contains cushion and bracket, Fig. 6. Order number 103068-99-0 also contains a cushion cover.*

**Assembly of cushion and amputation support bracket**


The assembly method is the same regardless of the cushion alternative selected.

The cushion can be identified by the letter of the alphabet moulded into it. Figure 5.

The amputation support bracket is universal and used on all combinations. Figure 6.

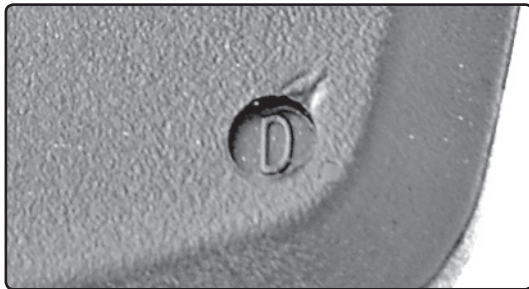
*Fig. 5. The moulded identification letter.*

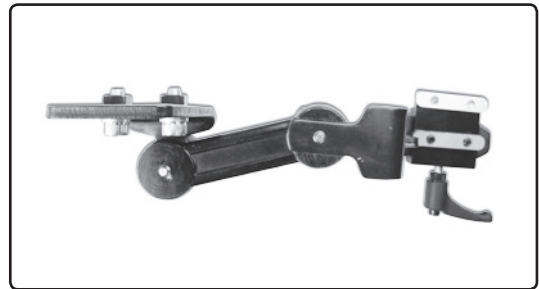*Fig. 6. Bracket for amputation support.*

Mount the two screws on the bracket for accessory attachment. Place a washer between the screw head and the bracket. Screw on the nuts with only a few turns. See figure 6.

Insert the nuts in the two grooves on the rear of the cushion. See figure 7. Slide the bracket in towards the centre of the cushion. Then turn the cushion 90°. Slide the cushion to a position which suits the user. Tighten the fixing screws.

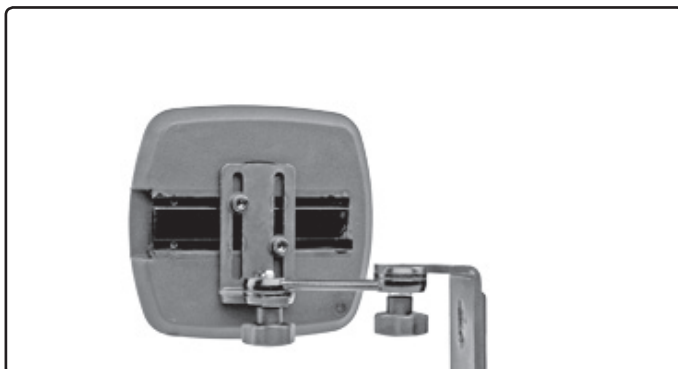*Fig. 7. Assembly of cushion and bracket. Left, the bracket nuts are slid into the cushion grooves. Right, the cushion is turned to the desired position.*

**Assembly Instruction**

Document No. <b>228678</b>	Revision <b>E</b>				
Production state <b>SERIES</b>					
Preform Date <b>2007-04-25</b>	Preform By <b>BTA</b>	Approved Date <b>11/18/2013</b>	Approved By <b>GHT</b>	Page <b>3</b>	

**Attachment of accessory**

Remove the plastic plug from the front edge of the accessory rail, see Fig. 9. Undo the screws on the bracket holder, Fig. 8, so that the metal lip slides easily in the upper groove of the accessory rail. The holder should only be mounted on the top part of the rail, Fig. 9. Replace the plastic plug. Slide the holder forwards towards the plastic plug. Tighten the holder screws.

Undo the screw lever on the bracket holder. Slide the amputation support into position. Tighten the lever. Figure 10.

*Fig. 8. Bracket holder for mounting on the seat's accessory rail.*

*Fig. 9. The bracket holder mounted on the accessory rail.*

**Adjustment of accessory**

**The amputation support is adjusted with the two articulated joints and by moving the cushion both for depth and from side to side.**

The vertical position is adjusted with the two articulated joints. Depth and sideways adjustments are done with the screws holding the cushion. See figure 7.

Loosen the knobs on both the articulated joints and set to the desired position. Fix the position by tightening the knobs. Loosen the screws holding the cushion and adjust to the desired depth and sideways position. Fix the position by tightening the screws again. Figure 10.

*Fig. 10. Adjusting the amputation support to the user's requirements.*

**Assembly Instruction**

Document No. <b>228678</b>	Revision <b>E</b>				
Production state <b>SERIES</b>					
Preform Date <b>2007-04-25</b>	Preform By <b>BTA</b>	Approved Date <b>11/18/2013</b>	Approved By <b>GHT</b>	Page <b>4</b>	

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**Assembly Instruction**

Document No. <b>228678</b>	Revision <b>E</b>				
Production state <b>SERIES</b>					
Preform Date <b>2007-04-25</b>	Preform By <b>BTA</b>	Approved Date <b>11/18/2013</b>	Approved By <b>GHT</b>	Page <b>5</b>	

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