



Link to english

BROMSBLAD TILL S3 BROMS

Monteringsanvisning för:
Bromsblad till S3 broms


art.nr. 4630150, 4630146, 4630120, 4630123
4630124, 4630012

Verktyg som behövs:

Insexnyckel 3 mm




Länk till video

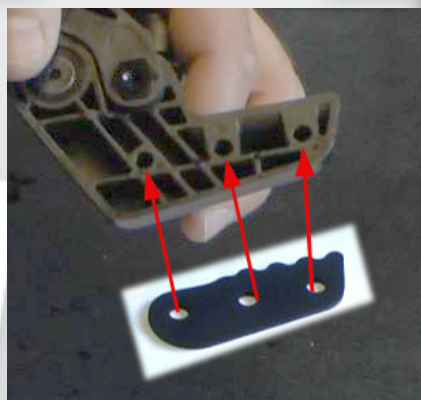

1) Passa in hålen på bromsbladet mot hålen på undersidan av broms S3.



2) Montera första skruven och drag åt den för hand något varv.

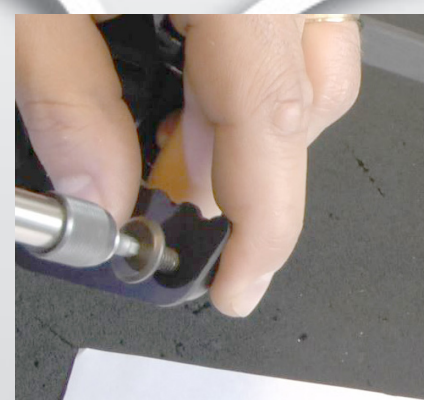

3) Drag åt skruven nästan i botten med skruvdragare eller handverktyg. Insex 3 mm.



4) Fortsätt med de övriga skruvarna på samma sätt.



5) Drag slutligen åt alla skruvar ordentligt i botten.



6) Klart. Upprepa proceduren på den andra bromsen.



BRAKE BLADE FOR S3 BRAKE

Assembly instruction for:
Brake blade for S3 brake


art.no. 4630150, 4630146, 4630120, 4630123
4630124, 4630012

Tools needed:

Allen key 3 mm




[Link to video](#)

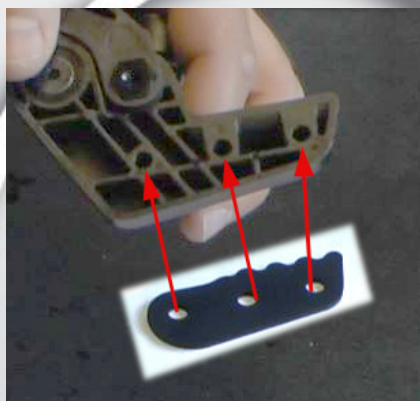

1) Place the holes of the brake blade over the holes on the underside of the S3 brake.



2) Mount the first screw and tighten it a few turns by hand.

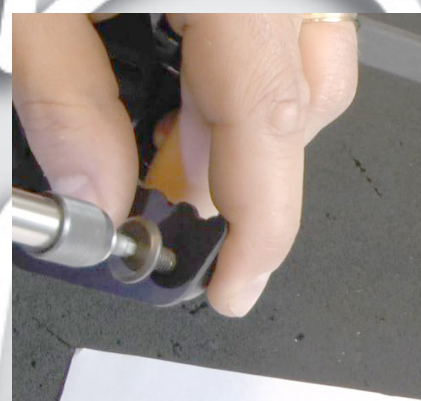

3) Tighten the screw almost to the bottom with a power tool or with a hand tool. Allen key 3 mm.



4) Continue with the rest of the screws..



5) Finally tighten the screws firmly to the bottom.



6) Finished. Repeat the procedure on the other brake.