**Assembly Instructions**


Document no. 228585	Revision A				
Production state <del>Series</del> <b>Series</b>					
Preform date 2007-04-19	Preform by JG	Approved date 2008-06-19	Approved by HNB	Page 1	

**Installation of table bracket, variable inclination**


Read through the instructions *before* starting installation.

1. Install the button that makes it possible to reverse the panel in the correct location using the two screws supplied (*Fig. 1 item 1*).
2. Install the frame with the four screws supplied (*Fig. 1 item 2*).
3. Install the panel on the table with four screws. Ensure that you install the panel with the button box facing the user (*Fig. 1 item 3*).
4. Lay the cables along the frame and the table arm. Fix the cables well using cable ties.
5. The panel can be turned through 180 degrees to produce a flat tabletop. See *Figure 2*.
6. The angle of the table can be adjusted continuously by opening the screws (see arrows in *Figure 1*). Tighten the screws when you have the angle you want.

To connect the button box and panel, see separate installation instructions.

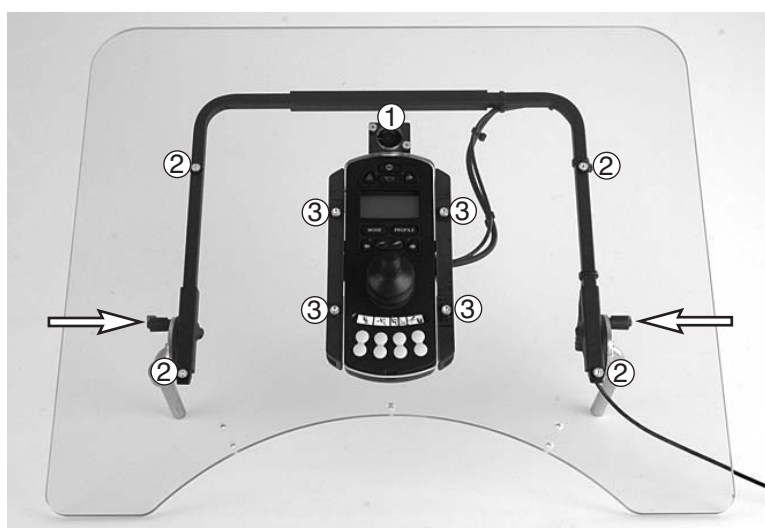*Fig. 1. General picture*

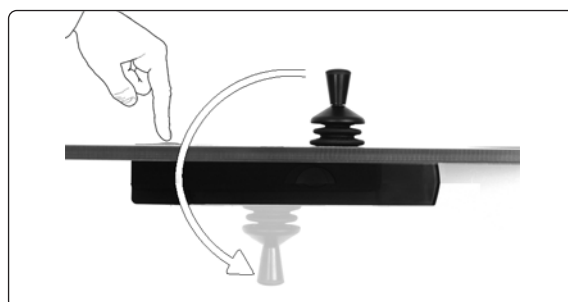*Figure 2. The panel can be reversed. Press the button at the front and turn the panel forwards 180 degrees.*

**Assembly Instructions**


Document no. <b>228585</b>	Revision <b>A</b>				
Production state <b>Series</b>					
Preform date <b>2007-04-19</b>	Preform by <b>JG</b>	Approved date <b>2008-06-19</b>	Approved by <b>HNB</b>	Page <b>2</b>	

**Installation of Miniflex table bracket**


Read through the instructions *before* starting installation.

1. Install the button that makes it possible to reverse the panel in the correct location using the two screws supplied (*Fig. 1 item 1*).
2. Install the frame with the three screws supplied (*Fig. 1 item 2*). **NB** If the smallest tabletop (460x380) is installed, two of the four screws at the front of the frame must be removed to prevent them from touching the button housing.
3. Install the panel on the table with four screws. Ensure that you install the panel with the button box facing the user (*Fig. 1 item 3*).
4. Lay the cables along the frame and the table arm. Fix the cables well using cable ties.
5. The panel can be turned through 180 degrees to produce a flat tabletop. See *Figure 2*.

To connect the button box and panel, see separate installation instructions.

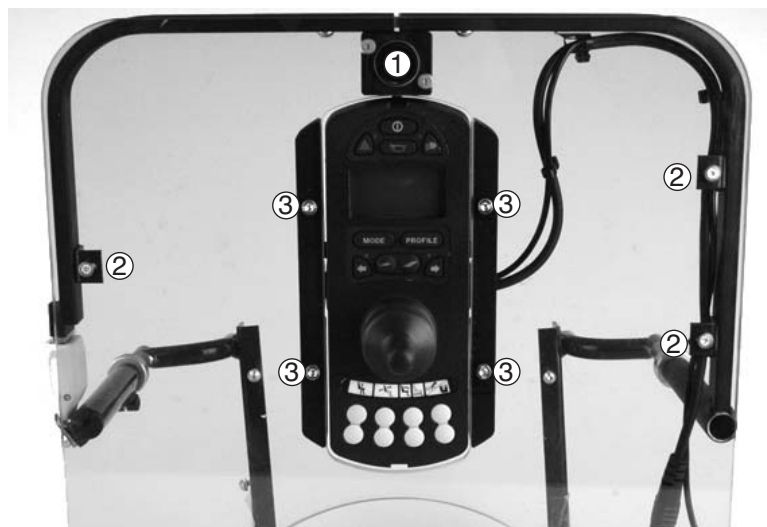*Fig. 1. General picture*

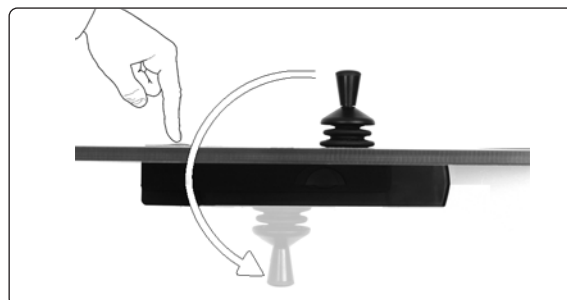*Figure 2. The panel can be reversed. Press the button at the front and turn the panel forwards 180 degrees.*

**Assembly Instructions**

Document no. <b>228585</b>	Revision <b>A</b>				
Production state <b>Series</b>					
Preform date <b>2007-04-19</b>	Preform by <b>JG</b>	Approved date <b>2008-06-19</b>	Approved by <b>HNB</b>	Page <b>3</b>	


**Installation of swiveling table bracket**

Read through the instructions *before* starting installation.


The swiveling table bracket can be installed on the right or left.

1. Install the button that makes it possible to reverse the panel in the correct location using the two screws supplied (*Fig. 1 item 1*).
  2. Install the frame with the four screws supplied (*Fig. 1 item 2*).
  3. Install the panel on the table with four screws. Ensure that you install the panel with the button box facing the user (*Fig. 1 item 3*).
- NB** If the table is to be installed on the left, see point 5. Otherwise go to point 4.
4. Lay the cables along the frame and the table arm. Fix the cables well using cable ties. Go to point 6.
  5. Drill a 3.6 mm diameter hole 10 mm from the bottom of the button housing and 10 mm from the edge of the housing as shown in *Figure 1, item 1* and fix the cables using cable anchors. Fix the cables to the frame using cable anchors too (*Figure 2*).
  6. The panel can be turned through 180 degrees to produce a flat tabletop. See *Figure 3*.


To connect the button box and panel, see separate installation instructions.

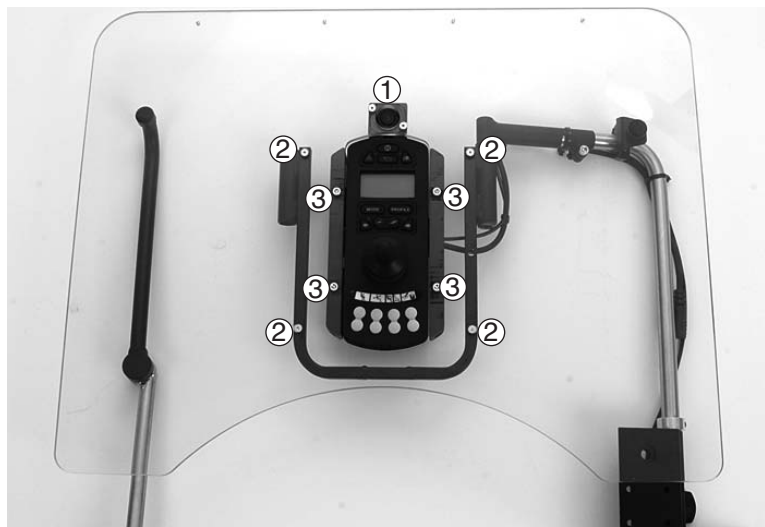*Fig. 1. General picture*

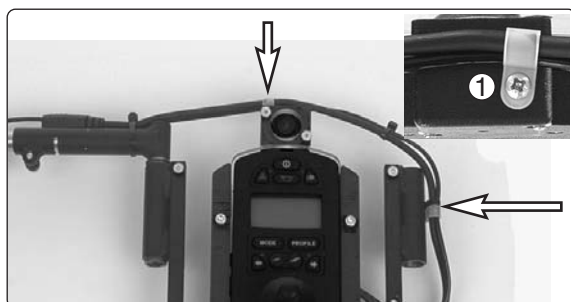*Fig. 2. If the table is installed on the left, the cables must be fixed to the button housing and the frame.*

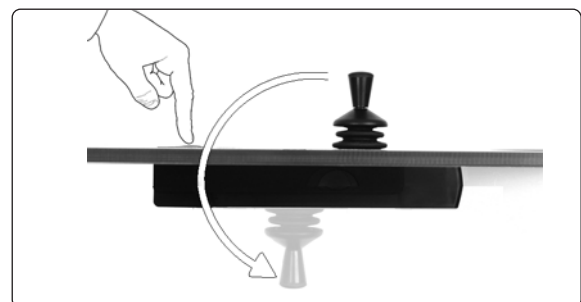*Figure 3. The panel can be reversed. Press the button at the front and turn the panel forwards 180 degrees.*

**Assembly Instructions**

Document no. <b>228585</b>	Revision <b>A</b>				
Production state <b>SERIES</b>					
Preform date <b>2007-04-19</b>	Preform by <b>JG</b>	Approved date <b>2008-06-19</b>	Approved by <b>HNB</b>	Page <b>4</b>	

**Installing a swiveling table frame**

Read through the instructions *before* starting installation.

1. Install the holder underneath the arm rest by unscrewing the balls from both brackets on the arm rest. Loosen the two nuts, item 1, on the arm rest profile and lift off the arm rest.
2. Loosen the two screws, item 2, a little to facilitate installation of the holder. Check that the two stop screws, item 3, are not tightened.
3. Guide the holder towards the arm rest and fix it with the two screws, item 4. Tighten the two stop screws, item 3, and the two screws in item 2.
4. Guide the two arm rest pins in the arm rest profile to the correct position and tighten the two nuts.
5. Screw the balls back on the arm rest brackets.

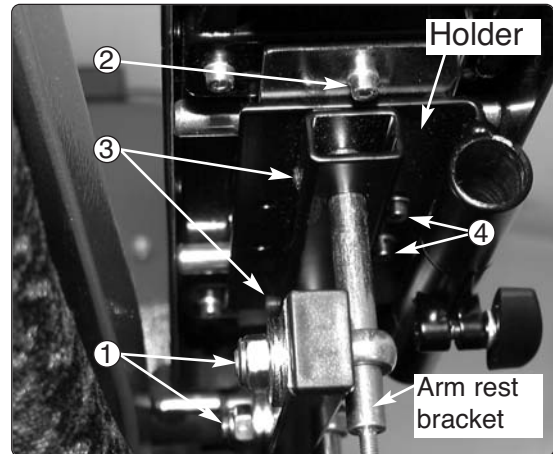
Fig. 7. Fixing the holder

**Installing a table frame in the arm rest**

Read through the instructions *before* starting installation.

1. Remove the rear stop ring on the table frame. Loosen the knob (see Figure 8) underneath the arm rest.
2. Guide the table frame into the folder and adjust the front stop ring so that the table is at a suitable distance from the body.
3. Fit the rear stop ring back on the table frame's tube. See Figure 9.

Tighten the knob that fixes the table frame. See Figure 8.

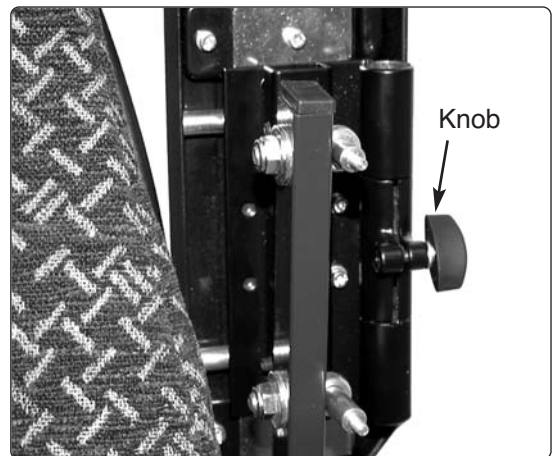
Fig. 8. Knob

Fig. 9. Table frame in holder

**Assembly Instructions**

Document no. 228585	Revision A				
Production state <del>Series</del> Series					
Preform date 2007-04-19	Preform by JG	Approved date 2008-06-19	Approved by HNB	Page 5	

**Fixing a table support for table control/central control.**

1. Install the table support in the panel holder bracket. Tighten the screw in the bracket when the tabletop rests against the support in the down position.
2. Lay the cables along the table frame and strap them in place. Check that the cables are free when the table is folded up. See Figure 10.

Fig. 10. Table support for table control/central control

**Assembly Instructions**

Document no. <b>228585</b>	Revision <b>A</b>				
Production state <b>Series</b>					
Preform date <b>2007-04-19</b>	Preform by <b>JG</b>	Approved date <b>2008-06-19</b>	Approved by <b>HNB</b>	Page <b>6</b>	

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**Assembly Instructions**

Document no. <b>228585</b>	Revision <b>A</b>				
Production state <b>Series</b>					
Preform date <b>2007-04-19</b>	Preform by <b>JG</b>	Approved date <b>2008-06-19</b>	Approved by <b>HNB</b>	Page <b>7</b>	

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**Assembly Instructions**

Document no. <b>228585</b>	Revision <b>A</b>				
Production state <b>SERIES</b>					
Preform date <b>2007-04-19</b>	Preform by <b>JG</b>	Approved date <b>2008-06-19</b>	Approved by <b>HNB</b>	Page <b>8</b>	

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