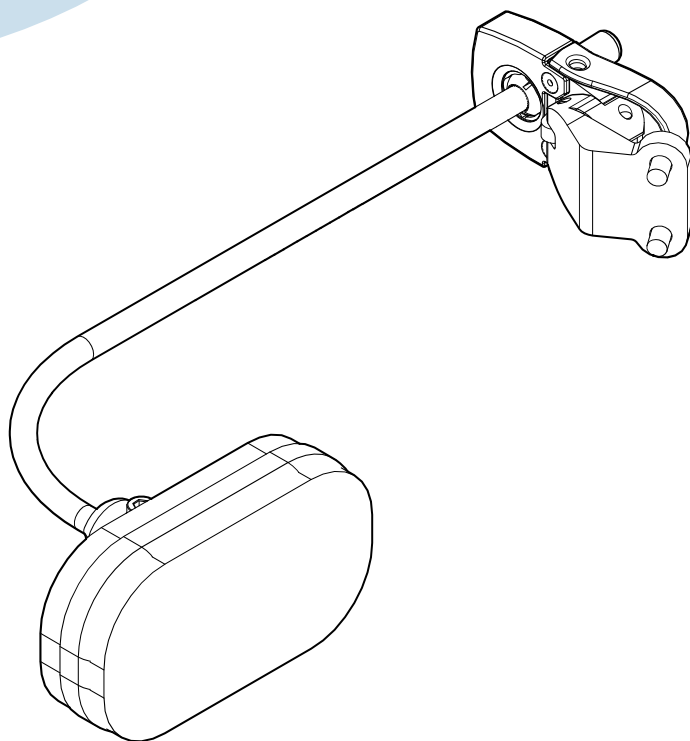
Swing Away Headrest Hardware

Assembly instruction

Thank you for choosing to be part of the Permobil family!

This is the Assembly Instruction of your new accessory. An assembly instruction is not a stand-alone document, but rather a complement to the wheelchair's User's Manual. It can sometimes refer to the User's Manual or the Service Manual.

This Assembly Instruction contains important information, installation instructions, and adjustment instructions. The instructions on how to install the accessory are intended for the service technician only. All other parts of the Assembly Instruction are intended for both the service technician and the user of the wheelchair. In the section Safety information, you find definitions of the admonitions used in this Assembly Instruction. Translations of these definitions can be found in the section Safety instructions. In the section General warnings for this product you find instructions for admonitions used in this Assembly Instruction, if any.

We hope that you are going to enjoy your new accessory. This accessory is intended to be used with a Permobil wheelchair. For more information regarding indoor and outdoor use, and for further information about your wheelchair, please see the wheelchair's User's Manual.

Contents

Important information	4
Safety information	5
Preparation.....	6
General warnings for this product	7
Installing swing away headrest	8
Adjusting swing away headrest	12
Rod shortening	16
Safety instructions.....	18
Contact us	21

Important information

Read this manual

Read this manual, including all admonitions and instructions, before you install, maintain, or use your accessory. Failure to follow these instructions and admonitions can lead to bodily injury or damage to the accessory or wheelchair.

Visually impaired

This document can be downloaded at www.permobil.com. If you are visually impaired, use the magnifying glass in your PDF reader to achieve desired text and picture size.

Order or download a copy of the instructions

If you need another copy of this Assembly Instruction, you can order one from Permobil. Please state the order number specified on the back cover. You can also download a copy from our website: www.permobil.com

Service and repairs

Please refer to the wheelchair's User's Manual for information about service and repairs.

Reporting an incident

Please contact Permobil or your dealer if you want to report an incident. The accessory does not have a serial number. However, if the accessory is attached to the wheelchair, we will ask for the wheelchair's serial number. This serial number is located on the chassis.

Warranty and expected service life

Contact your dealer or Permobil for information about the warranty period of this product. You can find the conditions of the warranty and further information about applicable warranties on our website.

This accessory has an expected service life of five years.

Scrapping and recycling

Contact Permobil for information about scrapping and recycling agreements in force.

Safety information

The following safety labels are used throughout this manual to draw attention to items of significant importance to safety:

Labels

Warning

Signal word used to indicate a hazardous situation which, if not avoided, could result in personal injury or death.

Caution

Signal word used to indicate a hazardous situation which, if not avoided, could result in damage to the product or to other property.

Crushing

Risk of crushing hands or other body parts.

Notice

Signal word used to indicate informative remarks.

Preparation

Necessary tools

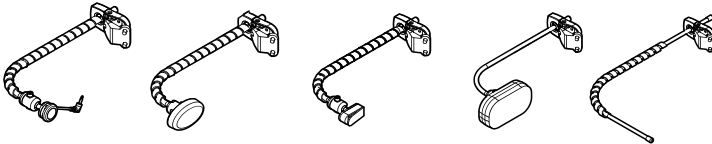
4 mm, 5mm

Assembly kit

The following assemblies are covered by this instruction:

General warnings for this product

① Warning

Swing away headrest should not be used during transport in a motor vehicle.

MUL page 18

① Caution

Avoid driving on rough terrain when using a chin control or when using switches as part of a drive control system.

MUL page 18

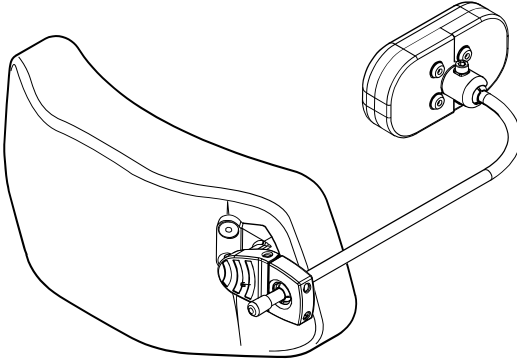
① Notice

Clean the hardware when necessary. Do a regular maintenance checkup on the bolts and nuts.

MUL page 19

Installing swing away headrest


Overview

This assembly instruction shows mounting of swing away headrest hardware with pad. The same mounting procedures apply for all swing away headrest products.

i For service technicians only

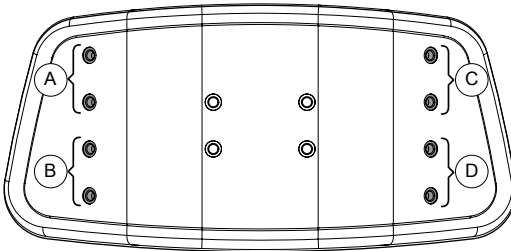
1.


Four possible mounting positions (A, B, C and D)

2.

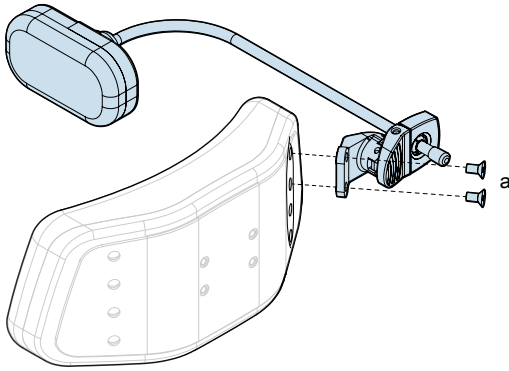
a = 9,8 Nm / 7,2 lbf.ft.

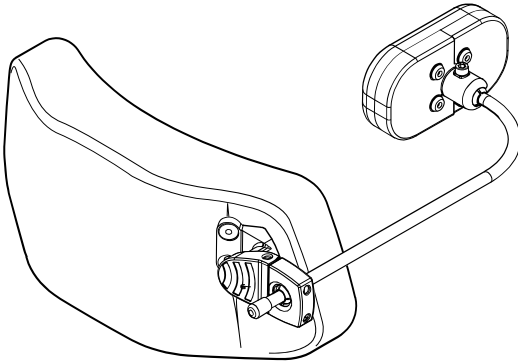
3.

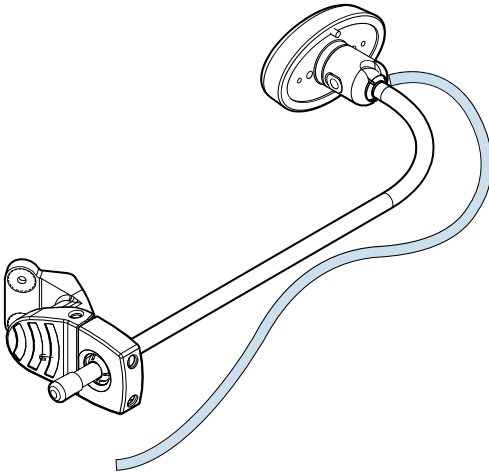
4.

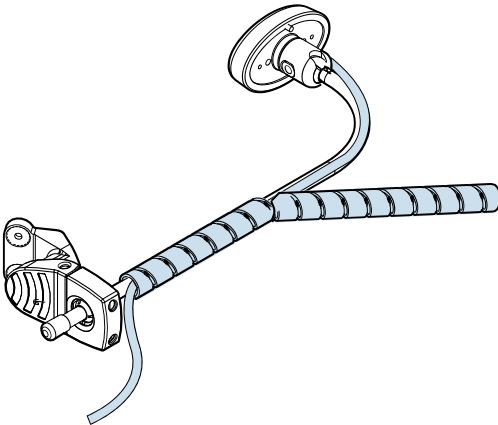5.

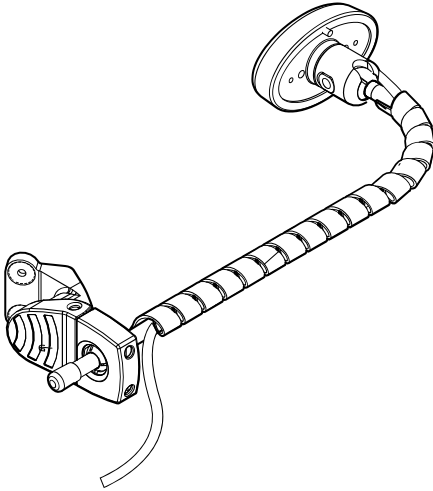
6.


Adjusting swing away headrest

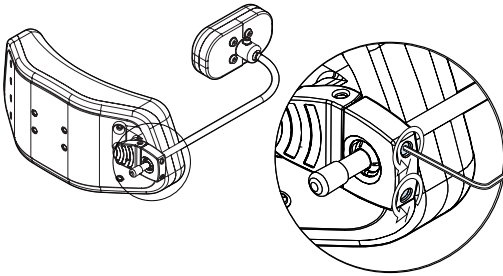1.

This assembly instruction shows adjusting of swing away headrest hardware with pad. The same adjusting procedures apply for all swing away headrest products.

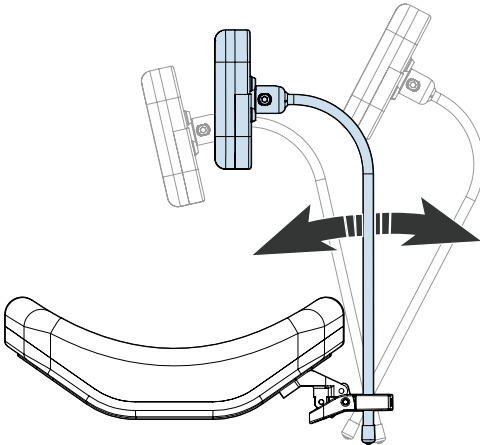2.


① Crushing


Risk of pinching or crushing hands and other body parts.

MUL page 19

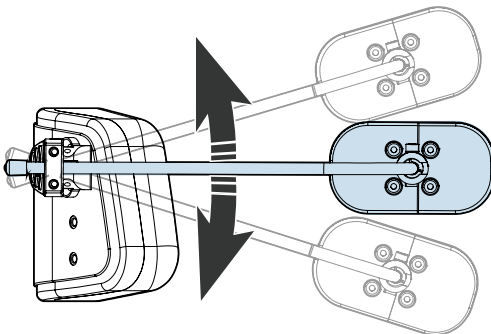3.

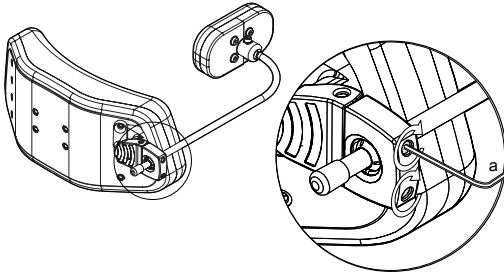
4.


a = 5,7 Nm / 4,2 lbf.ft.

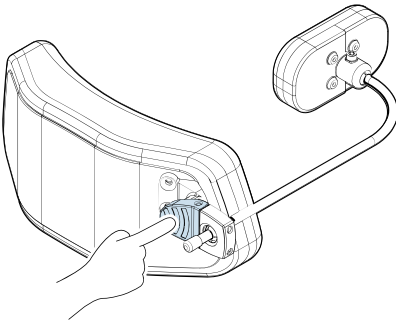
5.

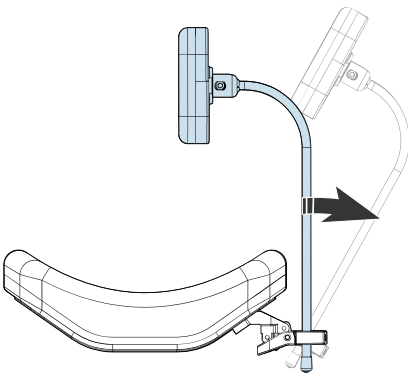6.

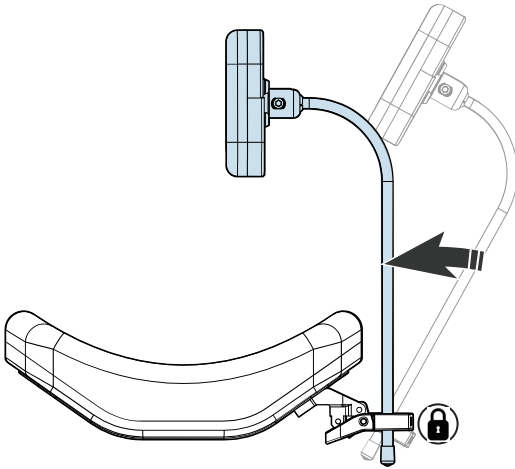7.


① Crushing


Risk of pinching or crushing hands and other body parts.

MUL page 19

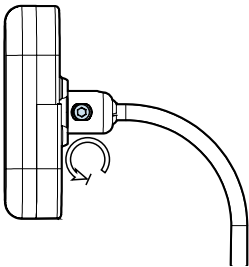8.

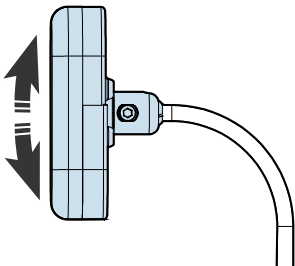
9.

10.

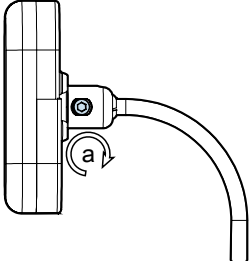
a = 4,7 Nm / 3,7 lbf.ft.


Rod shortening

i For service technicians only

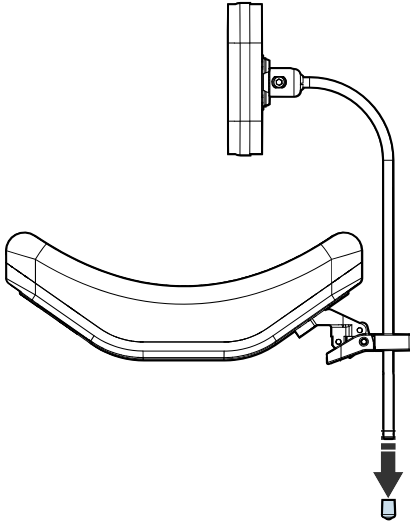1.

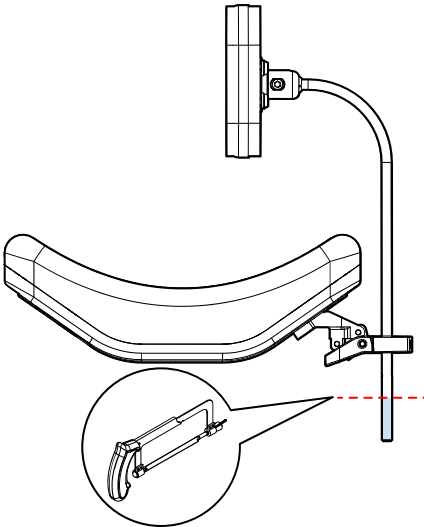
2.


If the rod interferes with any other parts of the wheelchair, the rod may be shortened.

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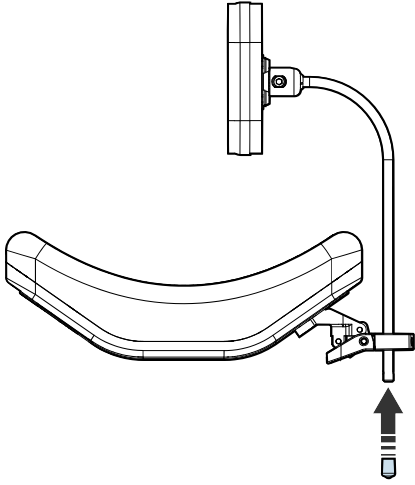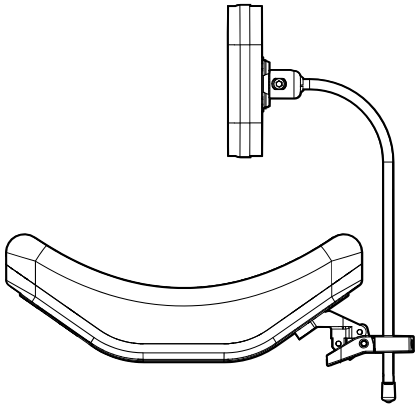Web: www.permobilus.com

permobil

Produced and published by Permobil

Edition: 2

Date: 2018-07-31

Order no: 334561 mul

7 330818 349842

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Permobil AB
Permobil Inc.