Assembly Instruction

Document No. 225199	Revision C				
Production state					
Preform Date 2011-06-16	Preform By UDN	Approved Date	Approved By	Page	

Mounting of Seatbelt.

Best.nr: 305982-99-0 Seatbelt incl. bracket.

The kit includes

- | | |
|---|---|
| <ol style="list-style-type: none"> 1. 1 qt. Belt 2. 4 qt. Three-glide 3. 2 qt. Fixlock brackets 4. 2 qt. Belt holders 5. 2 qt. Spacers | <ol style="list-style-type: none"> 6. 4 qt. M5x16 Screws 7. 2 qt. M8x20 Screws 8. 2 qt. 8,5x15,6x1,2 Washers 9. 4 qt. 5,5x15x1 Washers 10. 4 qt. M5 locking nuts |
|---|---|

Mounting on Corpus II and VS

Read the instructions *before* beginning mounting.

1. Place the fixlock brackets according the measurements in *fig. 1*. Use the bracket as a template to make marks for the drill holes. Drill with $\varnothing 5,2$ mm drill. Mount the two fixlock brackets on the seatback with four M5x16 screws and washers, see *fig 1*. Screw the screws from the front side of the seatback.
2. Mount the belt holder in the rear holes on the seat frame. *Fig. 2*.
3. Draw the belt through the three-glide in through the belt holder. Then draw the belt through the three-glide again. *Fig. 2*.
4. Adjust the length of the belt and fix its position with the fixlock brackets on the seatback. *Fig 3*.

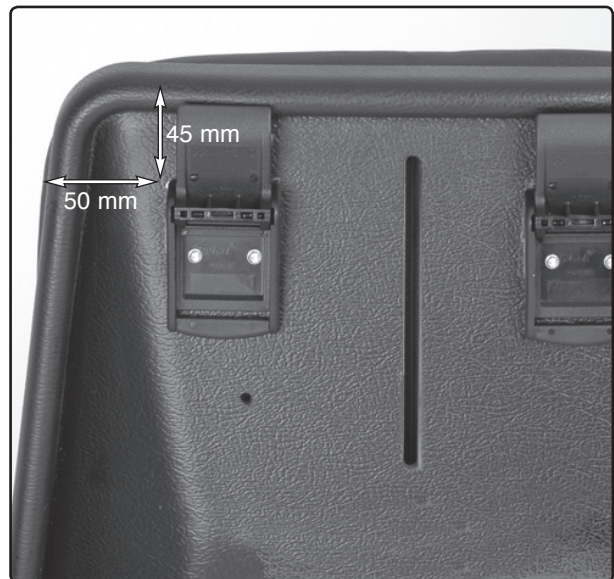
Fig. 1. Mount the fixlock brackets on the seatback

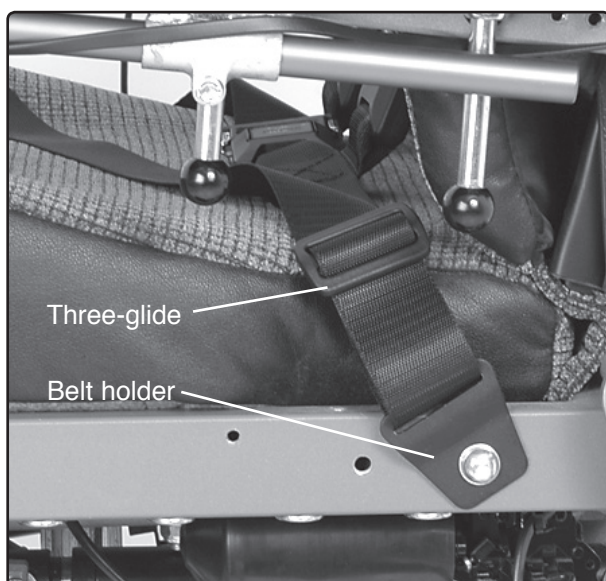
Fig. 2. Belt holder mounted on the seatframe

Fig. 3. Adjust and lock the belt with the fixlock brackets.

Assembly Instruction

Document No. 225199	Revision C				
Production state					
Preform Date 2011-06-16	Preform By UDN	Approved Date	Approved By		Page 2/4

Mounting on Corpus 3G

Read the instructions *before* beginning mounting.

1. Pull the belt through three-glide and through the slots in the backrest plate top. Then pull the belt through the three-glide again. *Fig. 4.*
2. Mount belt clip into the screw holes on back rests lower edge. *Fig. 5 och 6.*
3. Pull the belt through three-glide and through the belt holder. Then pull the belt through the three-glide again. *Fig. 5 och 6.*
4. Adjust the belt length in three-glide at the four attachment points.

Fig. 4. Mount belts on the seat back.

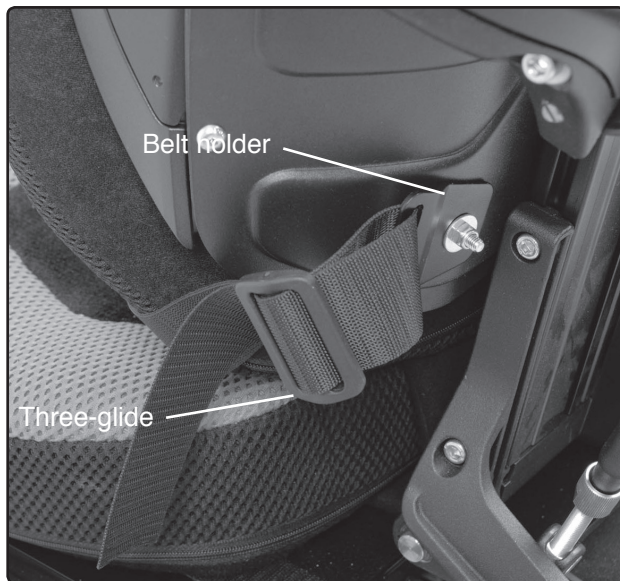
Fig. 5. The belt clip mounted on the rear panel.

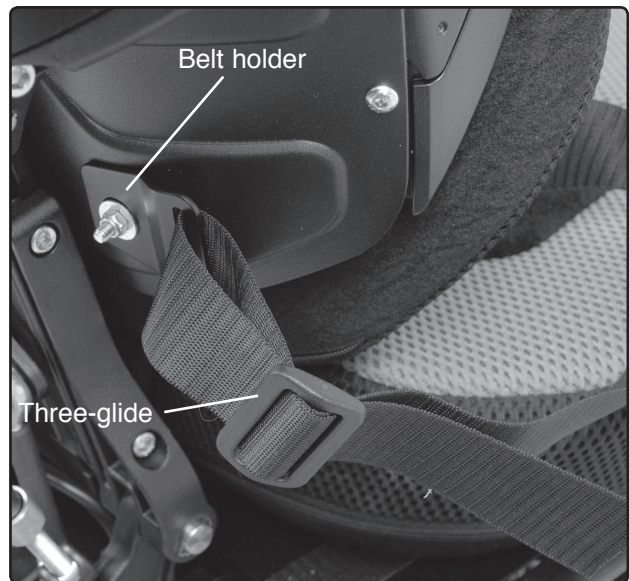
Fig. 6. The belt clip mounted on the rear panel.

Assembly Instruction

Document No. 225199	Revision C				
Production state					
Preform Date 2011-06-16	Preform By UDN	Approved Date	Approved By	Page 3/4	

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Assembly Instruction

Document No. 225199	Revision C				
Production state					
Preform Date 2011-06-16	Preform By UDN	Approved Date	Approved By	Page 4/4	

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