Assembly Instructions

Document No. 232758	Revision C				
Production state FOR SERIES					
Preform Date 2008-01-29	Preform By BTA	Approved Date 2015-10-13	Approved By GHT	Page 1	

Oxygen holder

List of parts in the set

Fig. 1. Parts included in the set.

- 1 pc. Oxygen holder.
- 3 pcs. Screws M6x10.
- 5 pcs. Washers 6 mm
- 3 pcs. Square nuts M6
- 1 pc. Bracket, lower Corpus II.
- 2 pcs. Screws M6x16.
- 2 pcs. Nuts M6.
- 1 pc. Bracket, upper Corpus II.
- 1 pc. Screw M5x12.
- 1 pc. Washer 5 mm
- 1 pc. Spacer.
- 1 pc. Nut, transport protection.
- 1 pc. Bracket CS.
- 2 pcs. Washers 8 mm.
- 2 pcs. Screws M8x16.
- 1 pcs. Bracket for TS.

Read the instructions carefully *before* starting installation

Mounting of bracket for Corpus II on rail

For mounting the oxygen holder on the Corpus II seat, two brackets are used, one above and one below. *Fig. 2.*

Remove the lower plastic plug on the holder.

Dismount the cup from the oxygen holder. Undo the two locking nuts and remove the washers and cup. Slide the screws out of the rail groove.

Slide one of the three square nuts from the kit into each of the two rail grooves. They are for use in mounting the upper bracket.

Mount the upper bracket with two M6x10 screws and two washers. *Fig. 2.* Screw into the two square nuts previously inserted in the rail. Do not screw the holder tightly, it must still be able to slide along the rail.

Fig. 2. Mounting the bracket for Corpus II.

Assembly Instructions

Document No. 232758	Revision C				
Production state FOR SERIES					
Preform Date 2008-01-29	Preform By BTA	Approved Date 2015-10-13	Approved By GHT	Page 2	

Mount the lower bracket with two M6x16 screws, two washers and two locking nuts. Place the M6x16 screws in the two bottom holes of the rail. *Fig. 2*. Place the lower bracket on the screws. Place the two washers on the screws. Screw the bracket in position with the two locking nuts. Tighten the screws.

Slide the screws on the cup back along the rail groove. Mount the cup, two washers and screw on the nuts. Do not fully tighten the nuts yet.

Insert the plastic plug in the rail groove.

Slide the cup so that its base is at the same height as the plastic plug. Tighten both nuts.

Mounting the oxygen holder on Corpus II

Mount the oxygen holder in the three screw holes which hold the plastic cover. *Fig. 3*.

Undo the three screws marked in *Fig. 3*. The two lower holes on the figure are used when mounting the accessory.

Mount the accessory's lower bracket with the two lower screws which hold the plastic cover on the back reinforcement. *Fig. 5*.

Undo the transport nut as shown in *Fig. 4*. It is not used in mounting.

Position the spacer in the recess next to the upper screw hole in the plastic cover. *Fig. 3*.

Slide the upper bracket on the accessory to the same height as the other screw hole. Place a 5 mm washer over the M5x25 screw. Screw the upper bracket in position. Tighten all three screws.

Finally tighten the two M6x10 screws holding the upper bracket in the rail. *Figs. 2 and 5*.

Release the upper part of the holder and adjust to the height of the oxygen cylinder. Fix the holder with the screw at the edge.

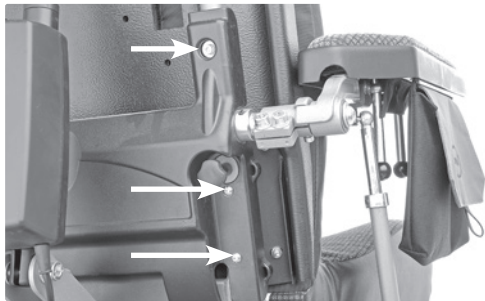
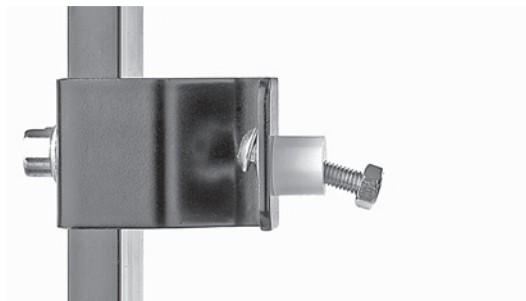
Fig. 3. Undo these three screws.

*Fig. 4. Screw, washer and spacer with transport nut. The spacer and M5x25 screw must be fitted in the upper hole, see *Fig. 3*.*

Fig. 5. The mounted accessory. Three existing screwholes on the back joint are used.

Adjustment of accessory

Check that the basket does not engage with any part of the wheelchair when the backrest is lowered or the seat tilted to their respective extreme positions. If the basket is not completely free in all positions, undo the screws holding it. Slide it in the rail to a position where the accessory will not hinder chair functions. Tighten the screws.

Assembly Instructions

Document No. 232758	Revision C				
Production state FOR SERIES					
Preform Date 2008-01-29	Preform By BTA	Approved Date 2015-10-13	Approved By GHT	Page 3	

Mounting of bracket for CS on rail

For mounting the oxygen holder on the CS seat, the bracket shown on *Fig. 6* is used.

Remove the lower plastic plug on the holder.

Undo the two nuts on the cup, and lift it and the washers out. Slide the two screws out of the rail groove.

Slide in the three square nuts from the kit. Slide two of the nuts in the right-hand groove.

Mount the bracket with three M6x10 screws and three washers. Screw into the square nuts. *Fig. 6*.

Slide the screws on the cup back along the rail and refit the plastic plug. Screw the cup nuts in place with washers. Slide the cup so that its base is at the same height as the plastic plug. Tighten both nuts.

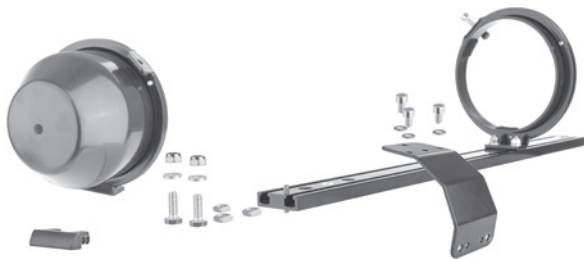
Fig. 6. Mounting of bracket for CS on rail.

Mounting of oxygen holder on the CS seat

The chair's armrest reinforcement has two pre-threaded holes for mounting the accessory. *Fig. 7*.

Place a washer on each of the two M8x16 screws and screw the accessory in place. *Fig. 8*.

The upper edge of the accessory must match the height of the upper edge of the back rest, *Fig. 9*.

If adjustment is required, undo the three M6x10 screws to allow the rail to be slid up or down.

Tighten the screws.

Release the upper part of the holder and adjust to the height of the oxygen cylinder. Fix the holder with the screw at the edge.

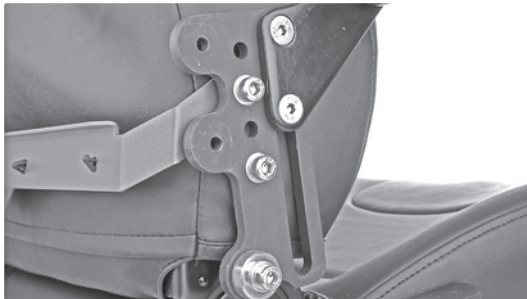
Fig. 7. Fixing point on armrest with mounting holes.

Fig. 8. The accessory mounted on the reinforcement.

Fig. 9. Approximate mounting position.

Assembly Instruction

Document No. 232758	Revision C				
Production state FOR SERIES					
Preform Date 2008-01-29	Preform By BTA	Approved Date 2015-10-13	Approved By GHT	Page 4	

Mounting of Corpus 3G bracket on rail

For mounting the oxygen holder on the Corpus 3G, the bracket shown in *Fig. 10* is used.

Remove the lower plastic plug on the holder.

Loosen the cup's two nuts a couple of turns, pull out the cup with the two screws from the rail groove. Slide the two accompanying tracknuts into the rail, one in each groove.

Mount the bracket with four M6x10 screws and four washers. Screw together with the track nuts. *Fig. 10*.

Slide the screws on the cup back into the rail and refit the plastic plug. Slide the cup so that its base is at the same height as the plastic plug. Tighten both nuts. Slide the bracket against the cup and tighten screws.

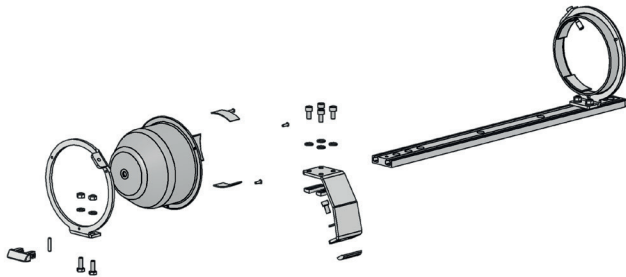
Fig. 10. Mounting the bracket on the rail for right assembly on Corpus 3G.

Mounting the oxygen holder on Corpus 3G

Undo the stop screws holding the cover on the backrest profile and remove the cover by pulling it upwards along the groove. (*Fig. 11*). Mount the bracket on the seat's backrest by loosening the bracket screws and sliding the track nut down into the back profile's outer groove. *Fig. 12*.

Remount the cover and slide the bracket toward the cover and tighten the screws. *fig. 12*.

Release the holder's upper part and adjust to the oxygen cylinder height. Fix the holder with the screw at the edge.

Check that the oxygen holder does not interfere with any other part when the seat is reclined and tilted.

For left-side mounting, the bracket should be reversed.

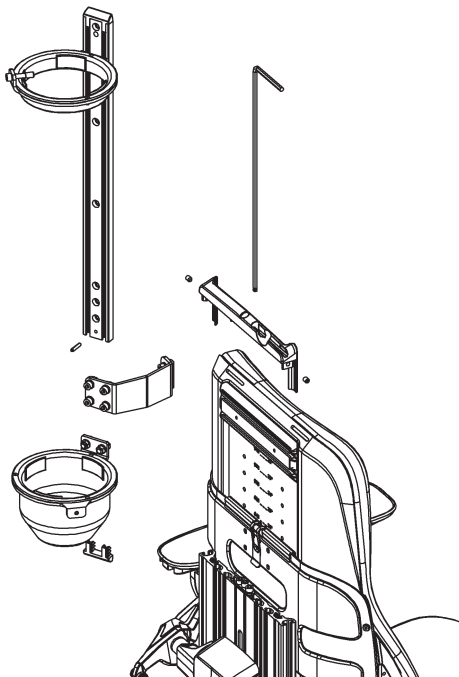
Fig. 11.

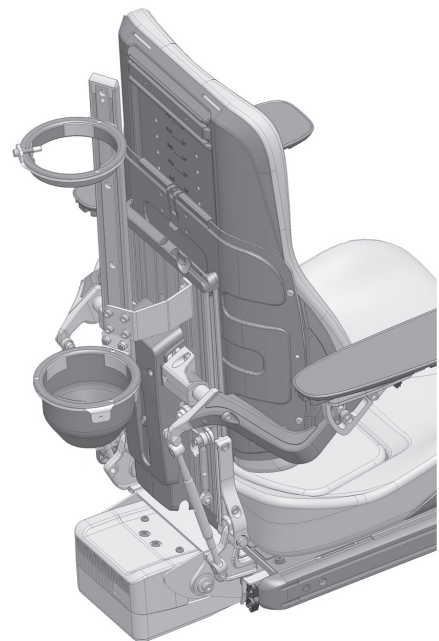
Fig. 12.

Approximate mounting position.

Assembly Instruction

Document No. 232758	Revision C				
Production state FOR SERIES					
Preform Date 2008-01-29	Preform By BTA	Approved Date 2015-10-13	Approved By GHT	Page 5	

Mounting of bracket for TS on rail

For mounting the oxygen holder on the seat, the bracket shown on Fig. 13 is used.

Remove the lower plastic plug on the holder.

Undo the two screws on the basket a few turns so it slides easily in the rail groove.

Completely remove the basket. Slide in the three nuts from the kit.

Slide two of the nuts in the right-hand groove.

Mount the bracket with three M6x12 screws and three washers.

Screw into the nuts. Fig. 13.

Slide the basket back along the rail groove and refit the plastic plug.

Slide the basket so that its base is at the same height as the plastic plug.

Tighten both nuts.

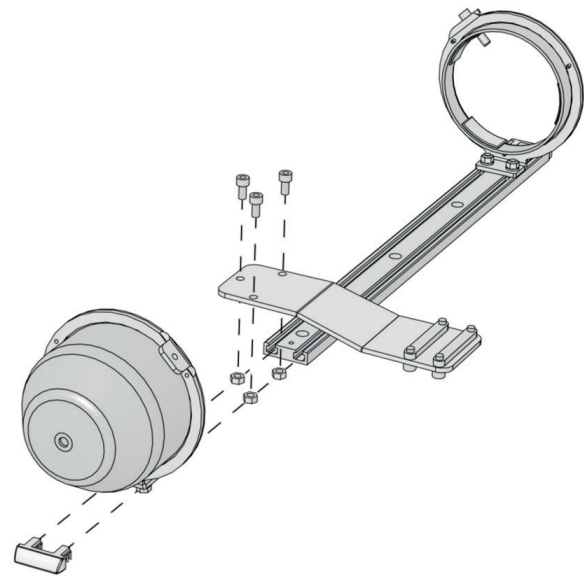
Fig. 13.

Mounting of oxygen holder on the seat

Remove the rubber protection on the backrest frames lower edge. See fig. 14.


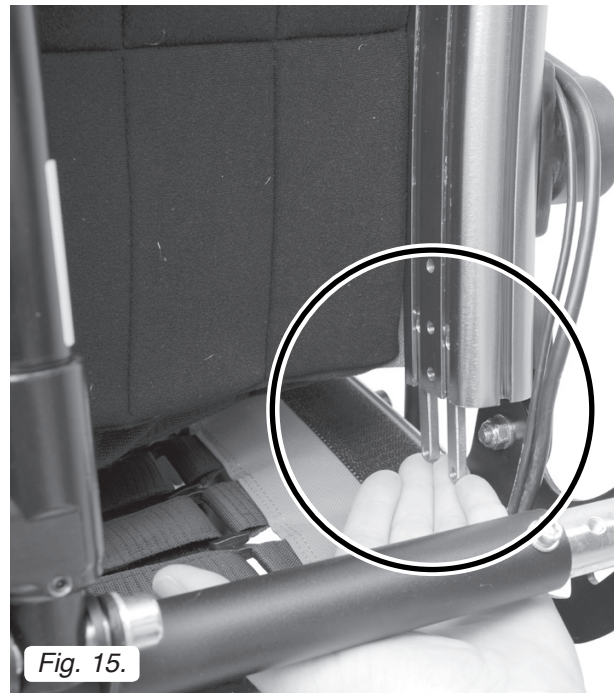
Fig. 14.

Assembly Instruction

Document No. 232758	Revision C				
Production state FOR SERIES					
Preform Date 2008-01-29	Preform By BTA	Approved Date 2015-10-13	Approved By GHT	Page 6	

Fit the slide bar brackets into the grooves of the backrest frame. See fig. 15.

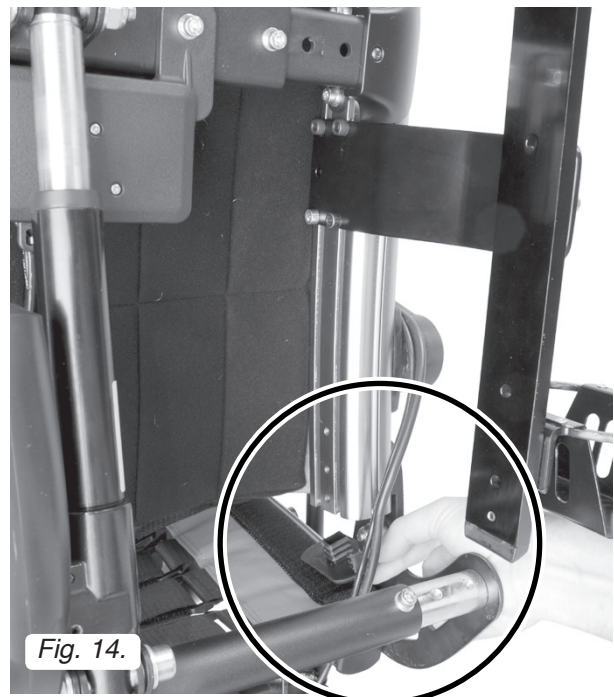Fit the oxygen holder with the supplied screws (M6x12) on the slide bars in the back rest frame. Fit the bracket in a position as high as possible on the back rest frame. See fig. 16.

Make sure the Oxygen holder doesn't collide with the backrest joint when the back rest recline is used. If needed, loosen the three screws holding the oxygen holder (Fig. 13), adjust to desired height and tighten the three screw.

If needed, the position of the cup and the upper holder can be adjusted. Loosen its two screws and adjust the position. Fix into desired position with the two screws.

Assembly Instruction

Document No. 232758	Revision C				
Production state FOR SERIES					
Preform Date 2008-01-29	Preform By BTA	Approved Date 2015-10-13	Approved By GHT	Page 7	

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