**Assembly Instruction**

Document No. 247826	Revision A				
Production state SERIES					
Preform Date 2011-05-25	Preform By TSN	Approved Date 2011-05-25	Approved By GHT	Page 1	

## Mounting of Luggage box


### List of parts in the set.

Qty. 1. Luggage box incl. undercarriage  
 Qty. 1. Chassis bracket  
 Qty. 1. Screw ISO 4762 M6x100 8.8 Fe/Zn 5 C1  
 Qty. 2. Screw ISO 4762 M6x10 8.8 Fe/Zn 5 C1  
 Qty. 3. Washer ISO 7089 6 200 HV Fe/Zn 5 C1  
 Qty. 2. Threaded spacer

Qty. 1. Luggage box holder  
 Qty. 4. Screw ISO 4762 M5x12 8.8 Fe/Zn 5 C1  
 Qty. 4. Screw ISO 4762 M6x16 8.8 Fe/Zn 5 C1  
 Qty. 4. Washer 6 Fe/Zn 5 C1 (SRB 6,4x14x1,5)  
 Qty. 4. Washer ISO 7093-1 5 200 HV Fe/Zn 5 C1


### Read the instructions carefully before starting installation.

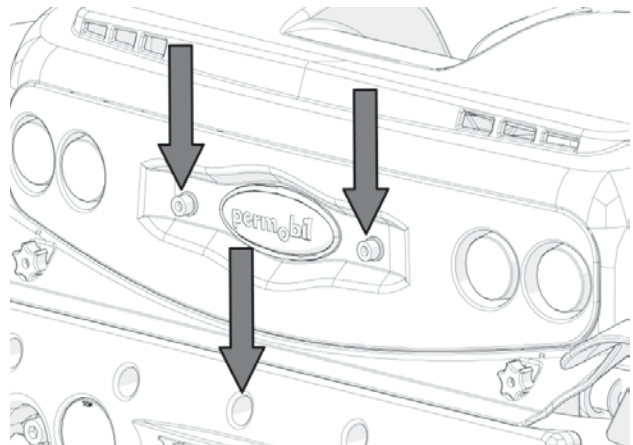1. Remove the two screws holding the rear lights and the screw at the centre of the lower cover. See fig.

*Remove existing screws.*

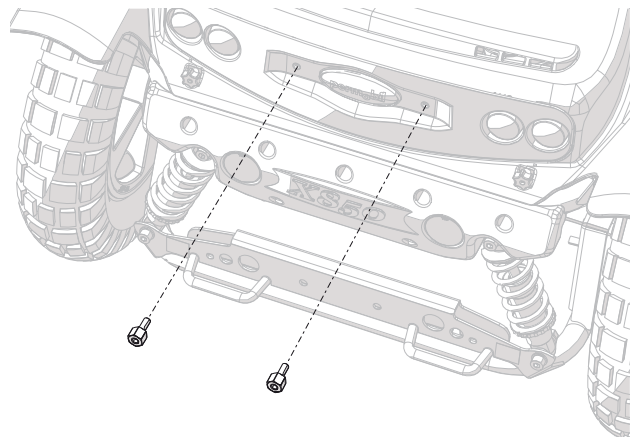2. Mount the attached threaded spacers. See fig.


*Mount the two threaded spacers.*

**Assembly Instruction**

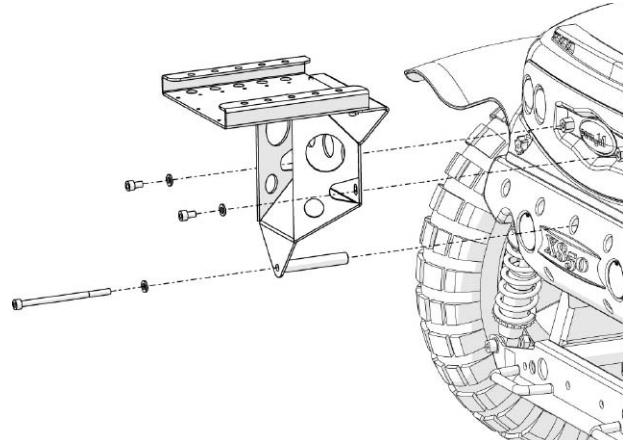
Document No. <b>247826</b>	Revision <b>A</b>				
Production state <b>SERIES</b>					
Preform Date <b>2011-05-25</b>	Preform By <b>TSN</b>	Approved Date <b>2011-05-25</b>	Approved By <b>GHT</b>	Page <b>2</b>	

3. Mount the chassis bracket with attached screws M6x12 and washers 6,4x12x1,6 on the threaded spacers and screw M6x100 with washer 6,4x12x1,6 at the centre of the lower cover. See fig.
4. Verify how far forward the luggage box can be mounted by measuring the distance between the rear edge of the chassis bracket to the rear of the back rest. Have in mind the seat functions like seat swivel, seat tilt and back rest recline will have effect on the distance. Use all the seat functions and measure the distance when it's the shortest.

*Mount the chassis bracket.*

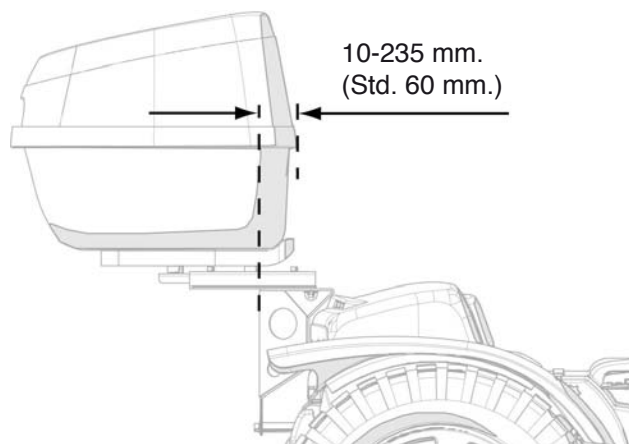
When mounting the luggage box in standard position, the distance between the rear end of the chassis brackets to the front end of the luggage box is approx. 60 mm. The distance can be adjusted from approx. 10 to 235 mm. by mounting in different positions. See paragraph 5-8.

*The position of the luggage box can be adjusted.*

5. Mount the luggage box holder using the attached screws (M6x16). As standard, the luggage box holder is mounted with its hole pattern corresponding with the hole pattern of the chassis bracket. See fig.

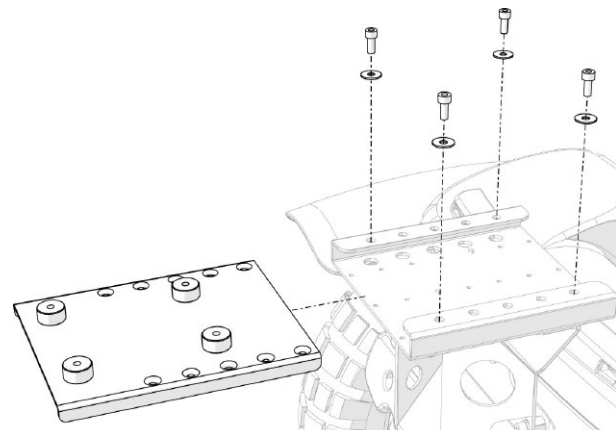To move the position of the luggage box forward or backward, move the luggage box holder 1-3 steps forwards/backwards in the hole pattern. Each step moves the position of the luggage box 25 mm. The distance between the front and rear screws decreases with the same amount of steps as the luggage box holder is moved.


*Mount the luggage box holder on the chassis bracket.*

**Assembly Instruction**

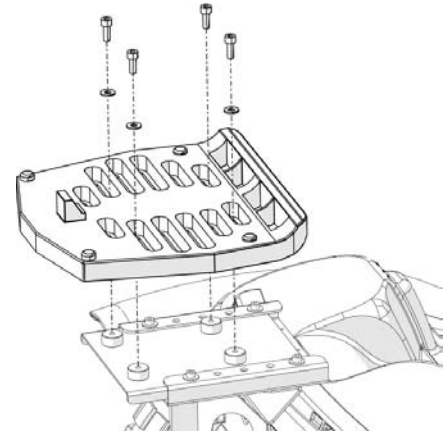Document No. <b>247826</b>	Revision <b>A</b>				
Production state <b>SERIES</b>					
Preform Date <b>2011-05-25</b>	Preform By <b>TSN</b>	Approved Date <b>2011-05-25</b>	Approved By <b>GHT</b>	Page <b>3</b>	


6. Mount the undercarriage on the bracket using the attached screws (M6x16) and washers. As standard the under carriage is mounted in its front holepattern and with the curved part pointing forwards with positions the luggage box with its opening pointing backwards.

If wanted, mounting can be performed using the rear hole pattern of the undercarrage with moves the position of the luggage box 25 mm. If wanted, the undercarrage can also be mounted with its curved part pointing backwards with positions the luggage box with its opening pointing forwards.


*Mount the undercarriage on the bracket.*

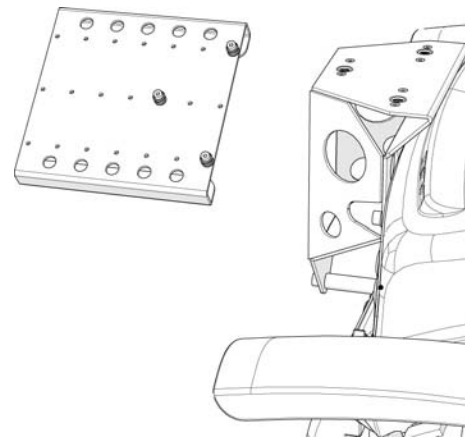7. If needed, the position of the luggage box can be moved even more forwards. This is done by moving the quick lock pins. As standard these are mounted in their most front position. To move the position of the luggage box forward, move the position of the locking pins 1-4 steps in the holepattern. Each step moves the position of the luggage box 25 mm.

*If needed, the position of the luggage box can be moved even more forwards by moving the quick lock pins.*

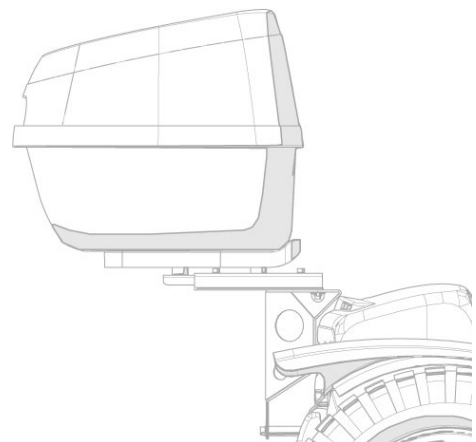8. Mount the luggage box on the undercarrage. First slide the front end of the luggage box into the curved part of the undercarrage, then secure it by pressing the rear end of the luggage box towards the undercarrage until its locked in place. If needed, the luggage box can be release by pressing the red button at the rear.

*Luggage box mounted.*

**Assembly Instruction**

Document No. 247826	Revision A				
Production state SERIES					
Preform Date 2011-05-25	Preform By TSN	Approved Date 2011-05-25	Approved By GHT	Page 4	

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
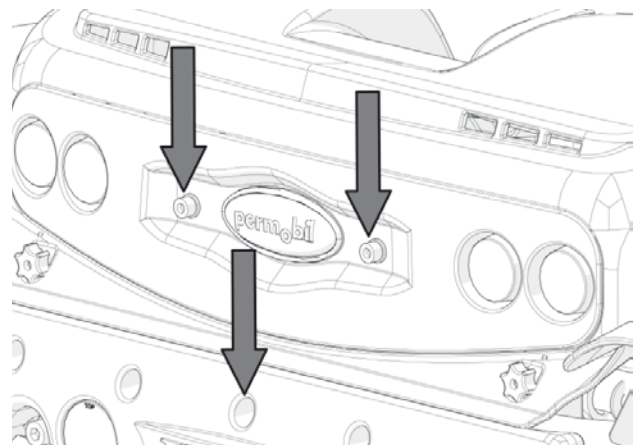
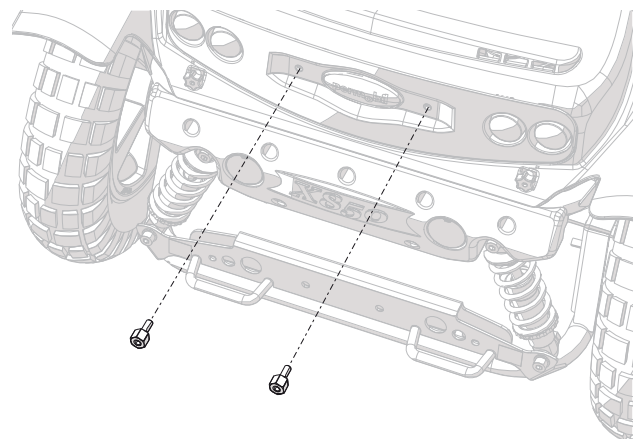
**Assembly Instruction**

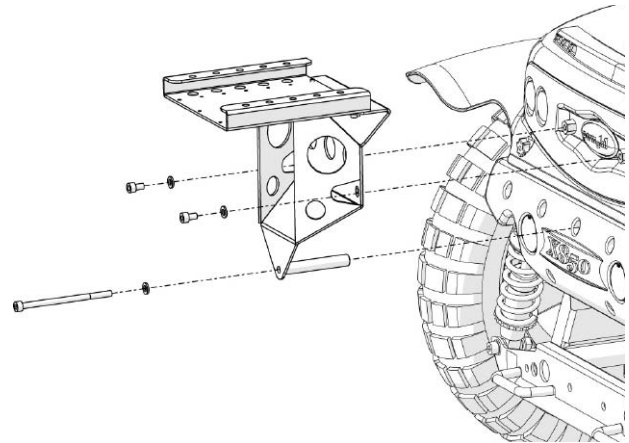Document No. <b>247826</b>	Revision <b>A</b>				
Production state <b>SERIES</b>					
Preform Date <b>2011-05-25</b>	Preform By <b>TSN</b>	Approved Date <b>2011-05-25</b>	Approved By <b>GHT</b>	Page <b>5</b>	

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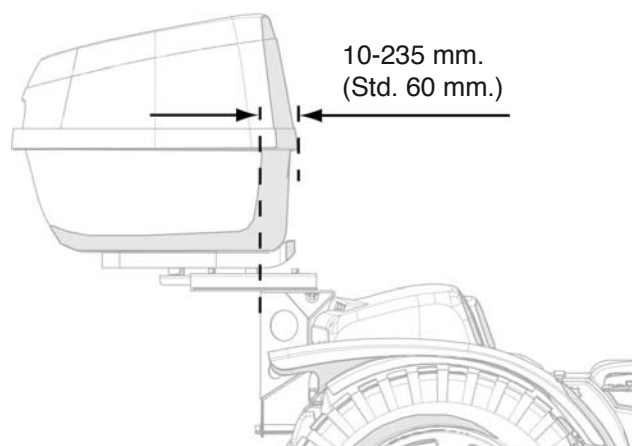
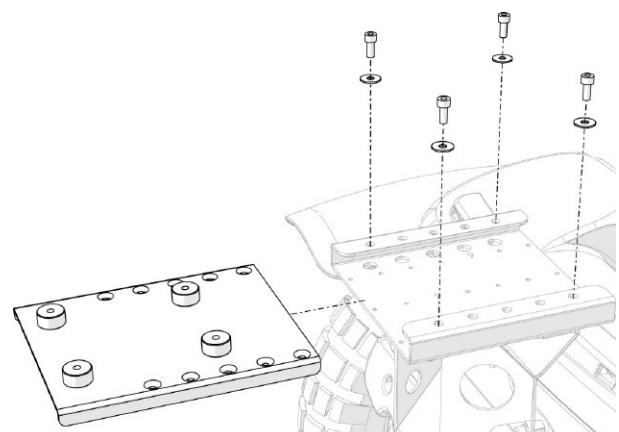
**Assembly Instruction**

Document No. <b>247826</b>	Revision <b>A</b>				
Production state <b>SERIES</b>					
Preform Date <b>2011-05-25</b>	Preform By <b>TSN</b>	Approved Date <b>2011-05-25</b>	Approved By <b>GHT</b>	Page <b>6</b>	

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