

Assembly Instructions

Document No. 249408	Revision A					
Production state SERIES						
Preform Date 2011-12-13	Preform By JOF	Approved Date 12/15/2011	Approved By GHT		Page 1	

Mounting Footplate heightening

List of parts in the set

*Fig. 1. Parts included in the set.*

1 pcs Footstrap kit.

Read the instructions carefully before starting installation

Mounting

1. Attach the straps to the footplate by first pulling the strap through the outer hole of the footplate, then continue under the footplate and attached the velcro, see fig.3. Then pull the strap up through the center hole of the footplate (do the same on both right and left side), see fig 2.
2. Place the footplate heightening on the footplate (one for 30 mm and two for 60 mm height) while the two straps is pulled through the middle hole of the footplate heightening, see fig. 4. Pull the straps through the buckles and fixate at desired length.

*Fig. 2. Mount the straps.**Fig. 3. Velcro under the footplate .**Fig. 4. Place the footplate heightening och fixate the straps .*

Assembly Instructions

Document No. 249408	Revision A					
Production state SERIES						
Preform Date 2011-12-13	Preform By JOF	Approved Date 12/15/2011	Approved By GHT		Page 2	

Montering av Footstrap kit

Lista över ingående delar.



Bild. 1. Ingående delar.

1 st Uppsättning Fotremmar

Läs noga igenom anvisningen *innan* montering påbörjas

Montering

1. Montera fotremmarna till fotstödsplattan genom att först dra remmen genom det yttre hålet i fotplattan, fortsätt sedan under fotplattan där det fästs med kardborre, se bild 3. Dra därefter upp remmen i fotplattans mittersta hål (detta gäller både höger och vänster fotrem), se bild 2.
2. Placera fotinlägget (en för 30 mm och två st för 60 mm höjd) på fotplattan samtidigt som de två fotremmarna dras upp genom fotinläggets hål, se bild 4. Spänn sedan fast remmarna i dess spännen och fixera vid önskad längd.



Bild. 2. Montera fotremmarna.



Bild. 3. Kardborrfäste undet fotstödet .



Bild. 4. Placera fotinlägg och spänn remmarna.