Assembly Instruction

| | | | | | |
|----------------------------|-------------------|---------------|-------------|--|---|
| Document No. 246555 | Revision A.01 | | | |  |
| Production state | | | | | |
| Preform Date 2011-01-12 | Preform By UDN | Approved Date | Approved By | | Page 1 |

Mounting Co-Pilot for R-net

Item no.: **317694***Co-pilot R-net*

List of parts in the set.

1 pcs. handle
4 pcs. screws ISO 4762 M5x12 Fe/Zn

1 pcs. R-net cable 1.5m
5 pcs. cable ties

Mounting

Read the instructions carefully *before* starting installation.

1. Mount the handle on the bracket using 4 x M5x10 screws. See *Fig. 1*.
2. Fit one end of the cable to the panel and the other to the terminal strip for the seat/chassis. For detailed information about the location of the strip, see the seat/chassis manual.
3. Secure the cable along the bracket and seat using cable ties located at approximately 150 mm intervals. Leave plenty of spare cable at the join points so that the cable is not stretched and broken. Remember that it must be possible to adjust the handle lengthwise.

Fig. 1. Mounting the handle.

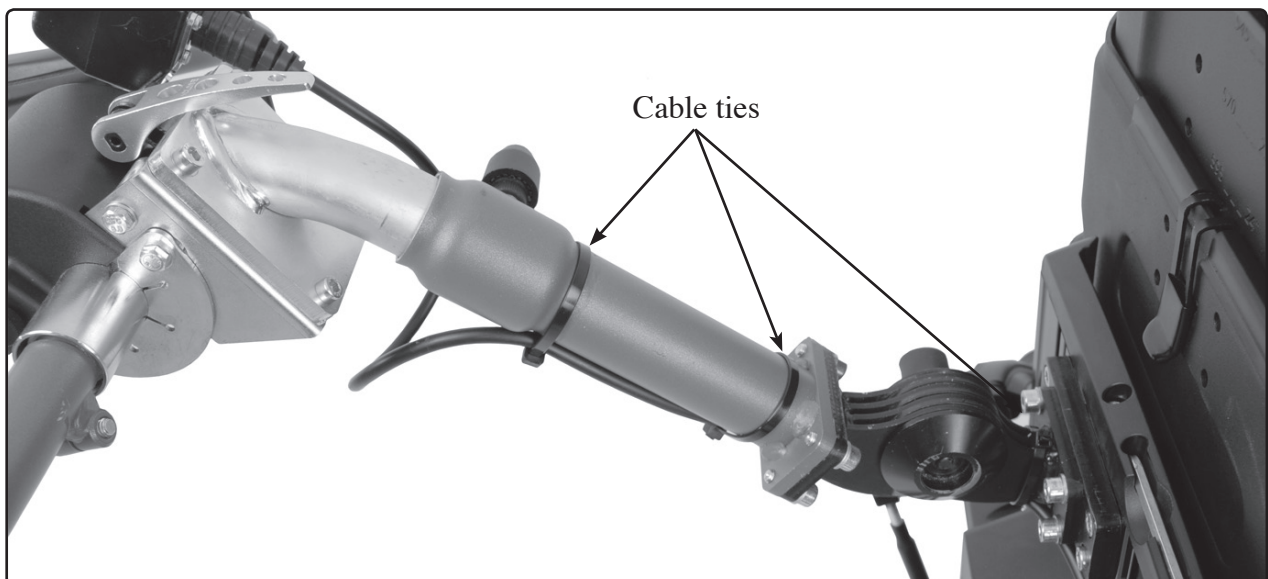
Fig. 2. Cabling secured with cable ties.

Assembly Instruction

| | | | | | |
|----------------------------|-------------------|---------------|-------------|-----------|---|
| Document No. 246555 | Revision A.01 | | | |  |
| Production state | | | | | |
| Preform Date 2011-01-12 | Preform By UDN | Approved Date | Approved By | Page 2 | |

NB! Check that all moving parts can move to their minimum and maximum position without stretching the cabling.

4. Check that all the covers are replaced. For detailed information, see the chair's service manual.
5. Load the Co-pilot program.
6. Test-run all the functions.

Fig. 3. Cabling secured with cable ties.

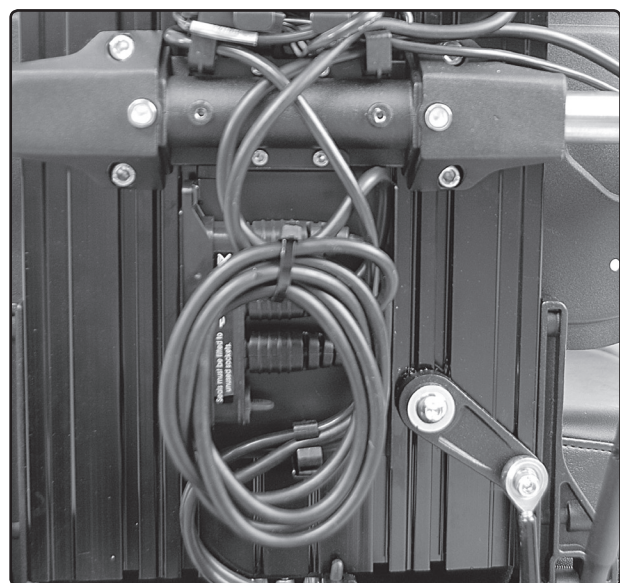
Fig. 4. Example of cable linked to chair's terminal strip.

Assembly Instruction

| | | | | | |
|----------------------------|-------------------|---------------|-------------|------|---|
| Document No. 246555 | Revision A.01 | | | |  |
| Production state | | | | | |
| Preform Date 2011-01-12 | Preform By UDN | Approved Date | Approved By | Page | 3 |

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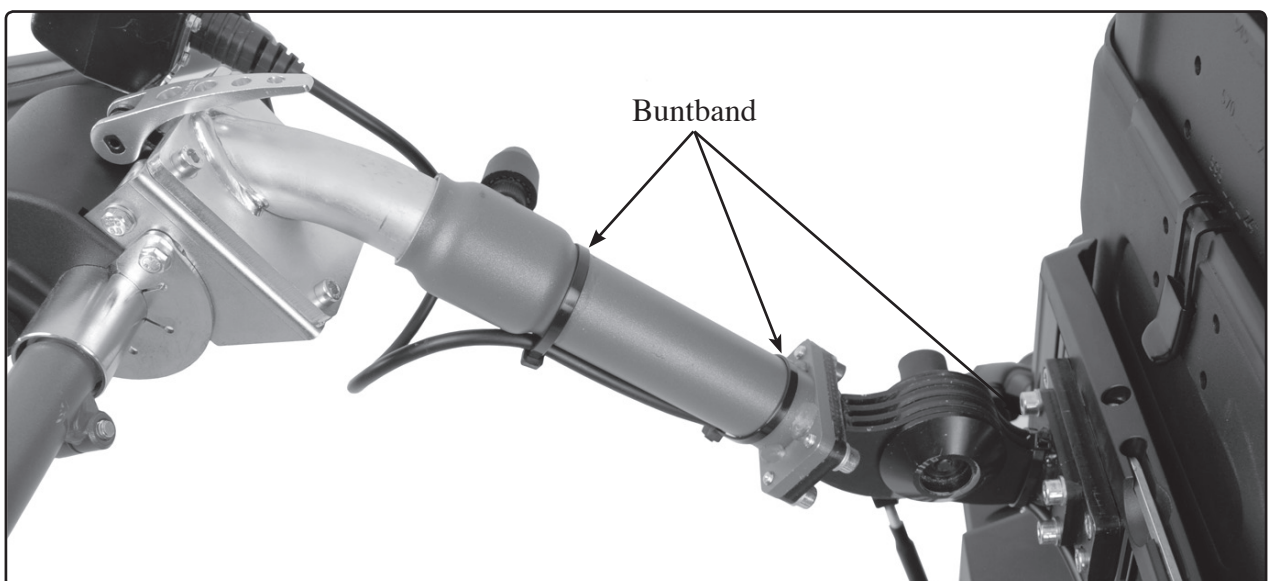
Assembly Instruction

| | | | | | |
|-----------------------------------|--------------------------|---------------|-------------|------------------|---|
| Document No. 246555 | Revision A.01 | | | |  |
| Production state | | | | | |
| Preform Date 2011-01-12 | Preform By UDN | Approved Date | Approved By | Page 4 | |

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