Assembly Instruction

Document No. 269511	Revision B				permobil
Production state SERIES					
Preform Date 2015-06-17	Preform By JOF	Approved Date 2016-02-10	Approved By GHT	Page EN 1	

Installation instructions for armrest pad, Corpus

Removing the armrest pad, Corpus	<i>Page 1</i>
Installing the armrest pad, Corpus	<i>Page 2</i>
Cleaning the armrest pad, Corpus	<i>Page 3</i>

Contents

1 set Armrest pad


Tools (not included):

- 4 mm ball-end hex key or similar

Removing the armrest pad, Corpus

1. Raise the armrest for easier access to the anchor points.
2. Remove the existing armrest pad by pressing in the anchor points with the aid of a tool (e.g. hex key or screwdriver). Once they are all pressed in, carefully remove the pad.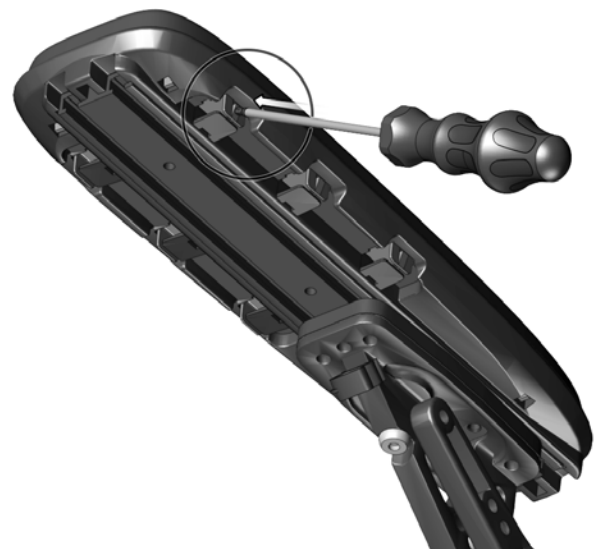


Assembly Instruction

Document No. 269511	Revision B				permobil
Production state SERIES					
Preform Date 2015-06-17	Preform By JOF	Approved Date 2016-02-10	Approved By GHT	Page EN 2	

Installing the armrest pad, Corpus


1. Install the new armrest pad by moving it straight down and locating it so that all of its anchor points arrive in the correct positions (1).
Press the pad into place by hand (2).

NOTE: Be sure that all anchor points attach properly.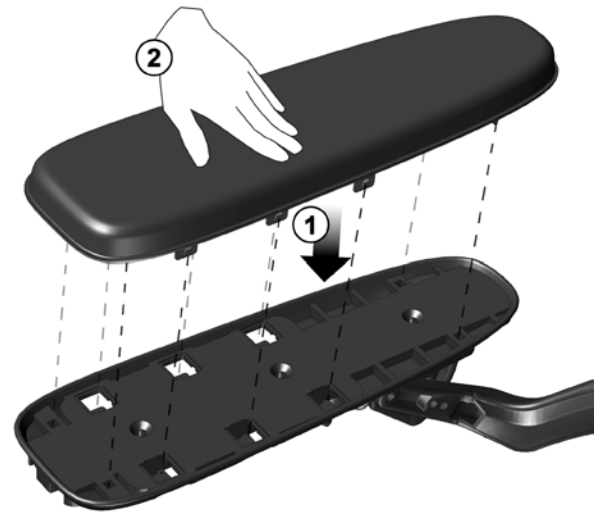

2. Use a 4 mm ball-end hex key (or similar narrow round tool) to press the excess fabric under the armrest trim.

NOTE: Do NOT use sharp instruments as these may damage the fabric.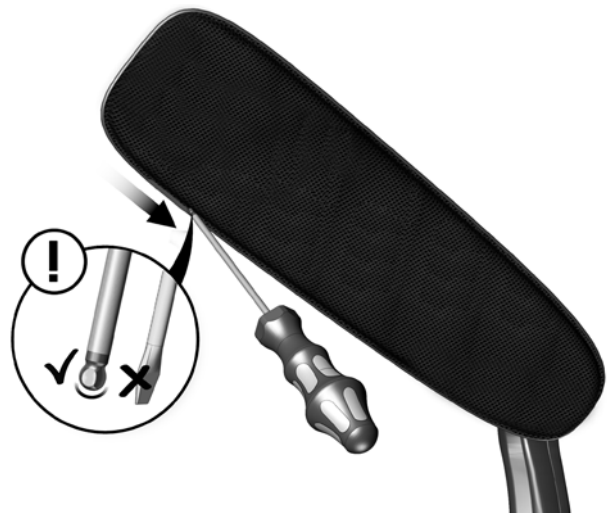


Assembly Instruction

Document No. 269511	Revision B				permobil
Production state SERIES					
Preform Date 2015-06-17	Preform By JOF	Approved Date 2016-02-10	Approved By GHT	Page EN 3	

Cleaning the armrest pad, Corpus

1. Clean the armrest pad with a cloth moistened with hot water.

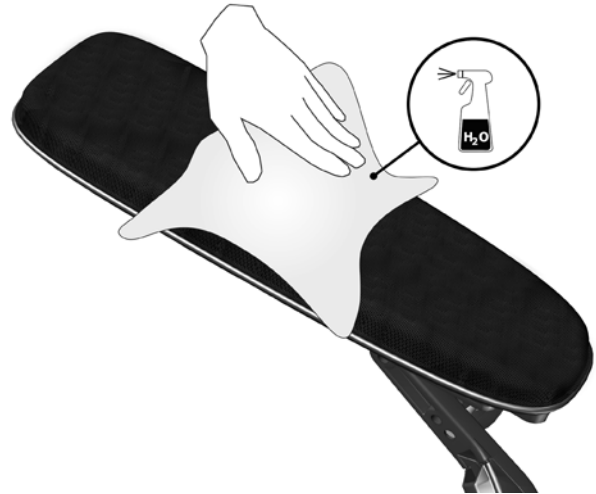Assembly Instruction

Document No. 269511	Revision B				permobil
Production state SERIES					
Preform Date 2015-06-17	Preform By JOF	Approved Date 2016-02-10	Approved By GHT	Page SE 1	

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