


Assembly Instruction

Document No. 240058	Revision A.02				
Production state					
Preform Date 2009-11-26	Preform By TSN	Approved Date	Approved By	Page 1/2	

Mounting of alternative joystick grip on X850

Order no.: 100962-99-0, 101165-99-0, 100961-99-0, 100965-99-0

List of parts in the set

Qty. 1. Alternative joystick grip

Read the instructions carefully *before* starting installation**Removing the standard grip.**

1. Loosen the screw holding the standard grip.
See fig.



The standard grip is fitted with a screw.

2. Pull the standard grip off the joystick. See fig.


Mounting the alternative grip.

Push the alternative grip on to the joystick. Make sure the grip is firmly fitted.



Remove the standard grip.

Assembly Instruction

Document No. 240058	Revision A.02				
Production state					
Preform Date 2009-11-26	Preform By TSN	Approved Date	Approved By	Page 2/2	

Montering av alternativt styrspaksgrepp på X850

Best.nr: 100962-99-0, 101165-99-0, 100961-99-0, 100965-99-0

Förteckning över de detaljer som ingår i satsen

1 st. Alternativt styrspaksgrepp

Läs noga igenom anvisningen *innan* montering påbörjas**Demontering av standard-grepp**

1. Lossa skruven som håller standard-greppet, se bild.



Standard-greppet sitter monterat med en skruv.

2. Dra standard-greppet av joysticken, se bild.

Montering av alternativt grepp.

Tryck fast det alternativa greppet på styrspaken.
Försäkra dig om att greppet sitter stadigt monterat.



Demontera standard-greppet.