Assembly Instruction

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Assembly Instruction

Document No. 263612	Revision C				permobil
Production state SERIES					
Preform Date 2014-09-04	Preform By JOF	Approved Date 5/26/2015	Approved By GHT	Page EN 1	

Installation instruction, USB Charger

Corpus 3G *Page. 2 - 5* VS *Page. 8 - 9* Miniflex *Page. 13 - 15*
 CII *Page. 6 - 7* PSjunior *Page. 10 - 12* K450 *Page. 16 - 18*

**Important Information****USB charger**

- Input: 18 - 75V DC
- Output: 5V / 1.5 A DC

- Only chargeable when the wheelchair is switched on.

- Not chargeable while the wheelchair is charging.

- DO NOT install the USB charger with the output facing up.
- The USB charger may ONLY be installed on Permobil products.

- To prevent water and dirt entering the USB charger, the safety cover MUST be put in place on completion of charging.


- For optimal charging, we recommend use of the manufacturer's OEM charging cable.

Table of contents

1. Charger
2. Splitter
3. Straps
4. Bolt, M6x25
5. Bolt, M6x22
6. Adapter cable
7. Nut
8. Locking nut
9. Washer

Tool (not supplied):

- Allen key
- Wrench

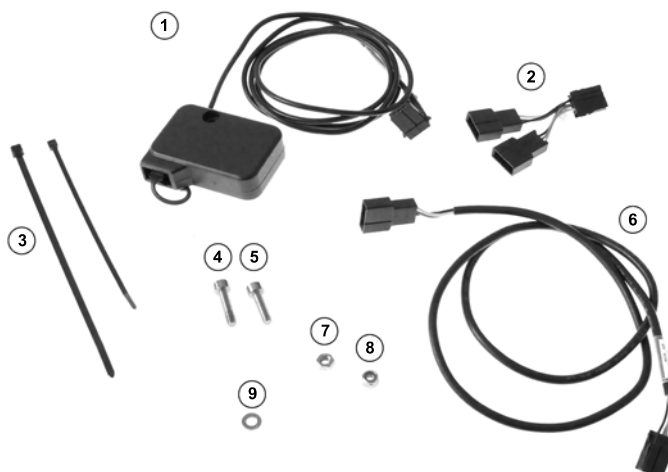Assembly Instruction

Document No. 263612	Revision C				permobil
Production state SERIES					
Preform Date 2014-09-04	Preform By JOF	Approved Date 5/26/2015	Approved By GHT	Page EN 2	

Installation instruction, USB Charger**Corpus 3G****Parts included, Corpus 3G**

1. Choose a side for the USB charger. Mount it beneath the left or right arm-rest. Choose the side depending on panel location as the charger should sit on the opposite side.

2. Fit the USB charger bolt and loosely tighten the nut so that 4 mm remains.

Assembly Instruction

Document No. 263612	Revision C				permobil
Production state SERIES					
Preform Date 2014-09-04	Preform By JOF	Approved Date 5/26/2015	Approved By GHT	Page EN 3	

Corpus 3G

3. Carefully slide the USB into the uni-track rail (the locating pin in one track and the bolt in the other) with the socket facing forward/out.

4. With the charger in the correct position, torque the bolt to 9.8 Nm.

5. Remove the armrest pad by clicking the attachments underneath free.

Assembly Instruction

Document No. 263612	Revision C				permobil
Production state SERIES					
Preform Date 2014-09-04	Preform By JOF	Approved Date 5/26/2015	Approved By GHT	Page EN 4	

Corpus 3G

6. Secure the cable beneath the armrest; follow items 1 - 3. In item 4, unscrew the two protective cover bolts and place the cable under the cover. Then bolt the cover back in place.

7. Remove the three backrest cover retaining bolts (for access to wiring).


Assembly Instruction


Document No. 263612	Revision C				permobil
Production state SERIES					
Preform Date 2014-09-04	Preform By JOF	Approved Date 5/26/2015	Approved By GHT	Page EN 5	

Corpus 3G

8. Follow items 1-3 for running the cable on the back of the backrest. Gather the cable and secure it under the cable clamp.

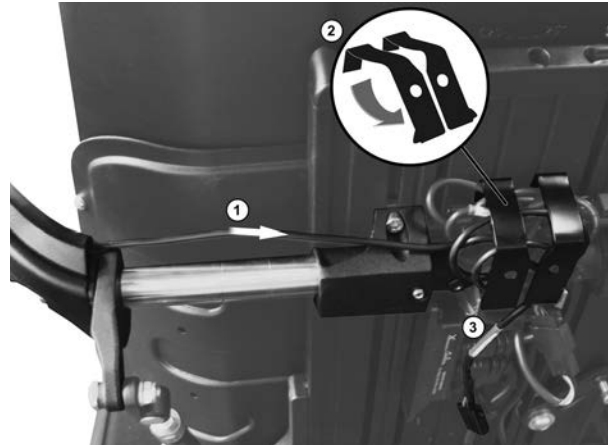
9. Use a splitter to connect the wiring. Then secure the remaining wiring and splitter under the cable clamp. Put the cover back in place and fasten it.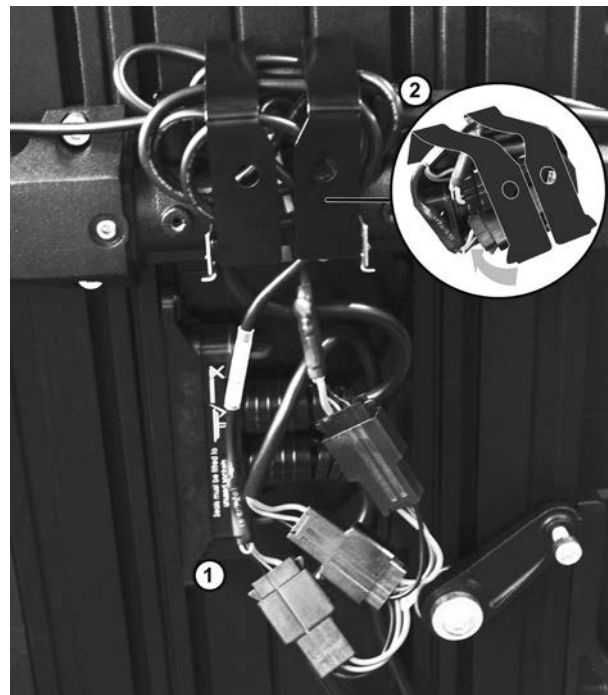

Assembly Instruction

Document No. 263612	Revision C				permobil
Production state SERIES					
Preform Date 2014-09-04	Preform By JOF	Approved Date 5/26/2015	Approved By GHT	Page EN 6	

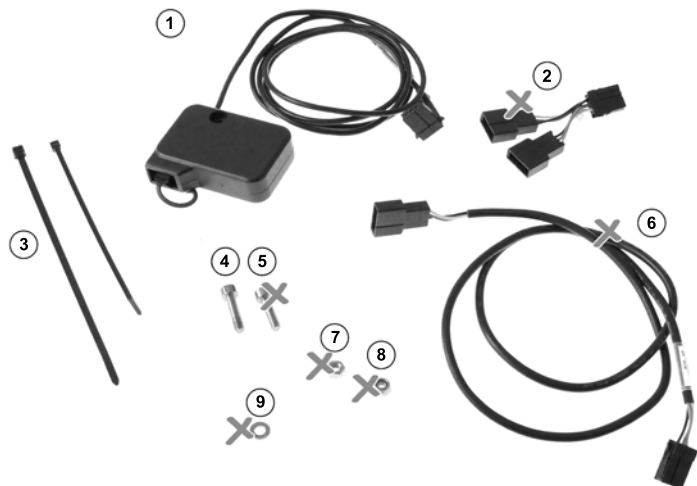
Installation instruction, USB Charger**CII****Parts included, CII**

1. Install the USB charger under the left armrest with the socket facing down.

2. Fasten the USB charger using an M6x25 bolt in the threaded hole at the back of the armrest rod.

Tightening torque 9.8 Nm.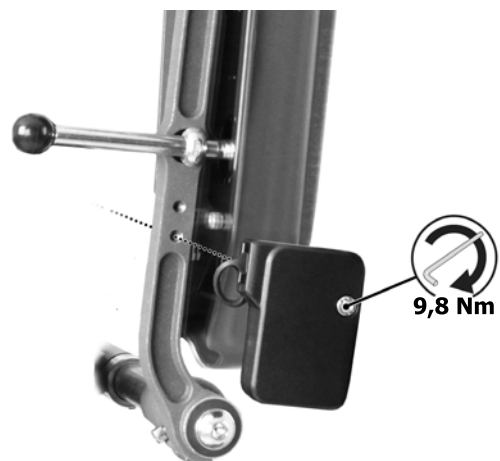


Assembly Instruction


Document No. 263612	Revision C				permobil
Production state SERIES					
Preform Date 2014-09-04	Preform By JOF	Approved Date 5/26/2015	Approved By GHT	Page EN 7	

CII


3. Remove the backrest plate by undoing the two knobs (1). Then slide the upper lock (2) to the left and lift away the backrest plate.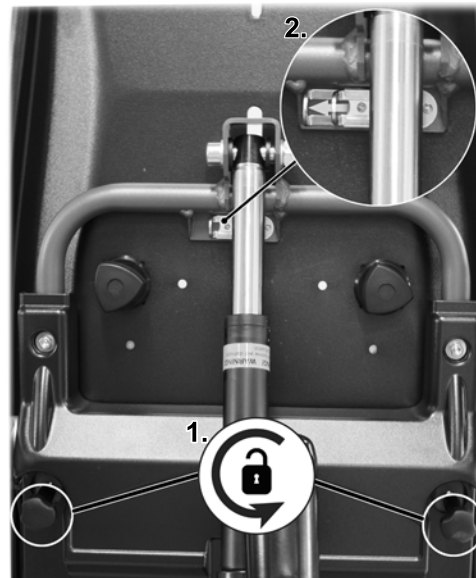

4. Run the cable from the USB charger along the left side of the back frame following positions 1-3. Then run the cable in behind the back frame to the centre of the front of the backrest.

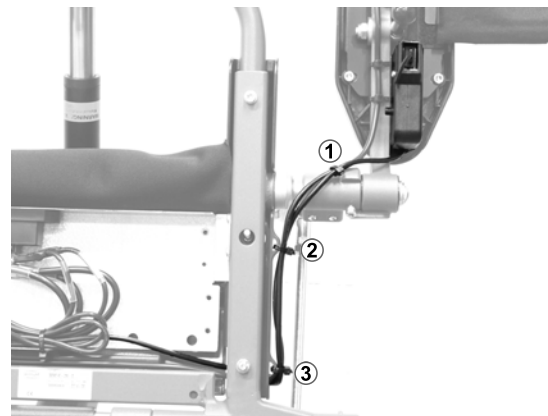
5. Gather the cable and secure it using a cable tie at position 4, then connect the wiring.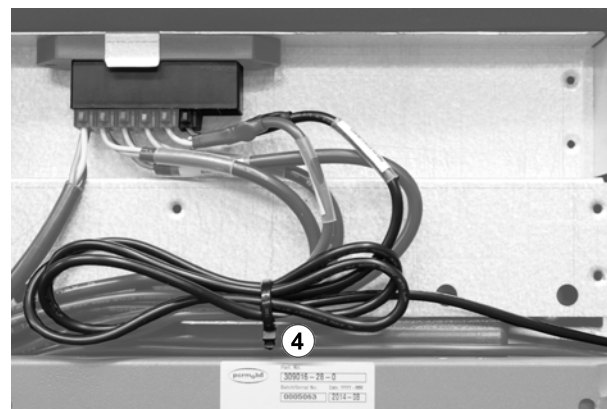

Assembly Instruction

Document No. 263612	Revision C				permobil
Production state SERIES					
Preform Date 2014-09-04	Preform By JOF	Approved Date 5/26/2015	Approved By GHT	Page EN 8	

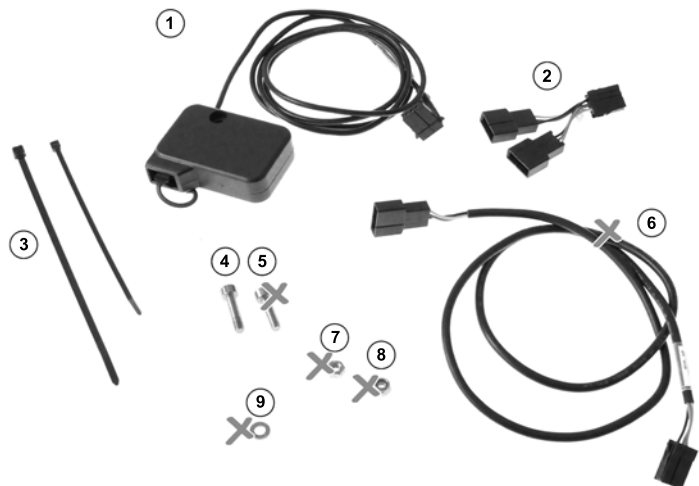
Installation instruction, USB Charger**VS****Parts included, VS**

1. Install the USB charger under the left armrest with the socket facing down.

2. Fasten the USB charger using an M6x25 bolt in the threaded hole at the back of the armrest rod.

Tightening torque 9.8 Nm.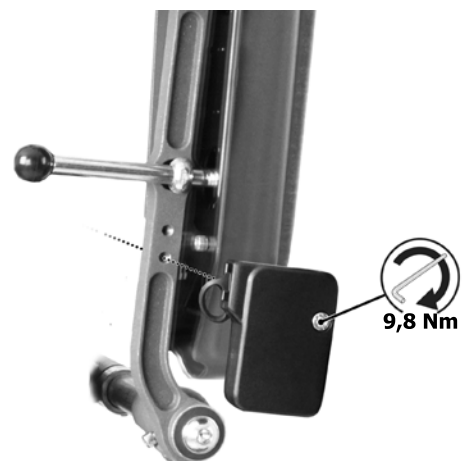


Assembly Instruction


Document No. 263612	Revision C				permobil
Production state SERIES					
Preform Date 2014-09-04	Preform By JOF	Approved Date 5/26/2015	Approved By GHT	Page EN 9	

VS

3. Run the cable along the armrest rod and continue behind the backrest following positions 1-3. Fashion a bend in the cable and secure it using a cable tie in position 3. Continue down toward the actuator on the right side following positions 4-9.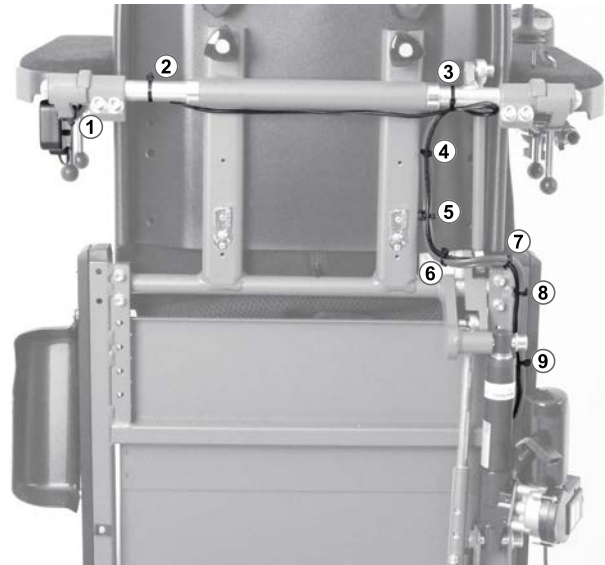


4. Connect the wiring using a splitter behind the actuator and secure it using a cable tie. The upper actuator bracket can be removed to facilitate access for connection.

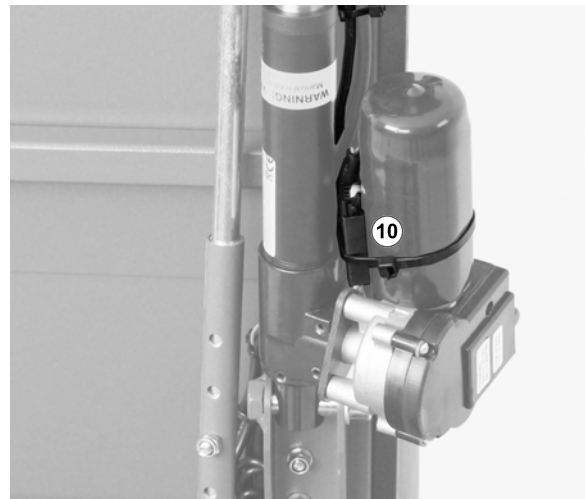**Important Information**


Secure the backrest when removing the upper actuator bracket, as the latter also attaches the backrest.

Assembly Instruction


Document No. 263612	Revision C				permobil
Production state SERIES					
Preform Date 2014-09-04	Preform By JOF	Approved Date 5/26/2015	Approved By GHT	Page EN 10	

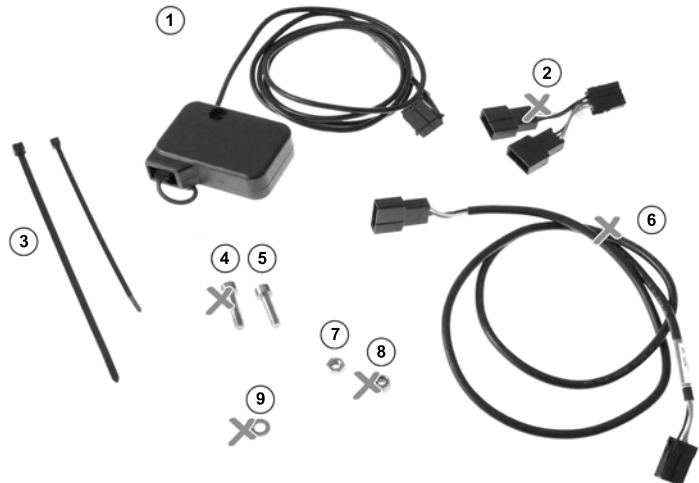Installation instruction, USB Charger**PS junior****Parts included, PSjunior**

1. Choose a side for the USB charger. Install it beneath the left or right armrest. Choose the side depending on panel location as the charger should sit on the opposite side.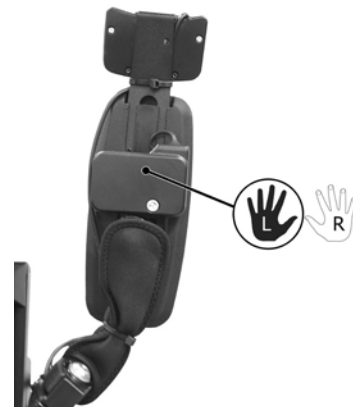

2. In cases where the button panel is located in the armrest, remove both the panel and its bracket from the armrest. Begin by removing the two bolts that hold it in place, then lift the panel away. Then remove the two bolts that hold the panel bracket in place and use rocking movements to slide the bracket out of the unitrack bracket.

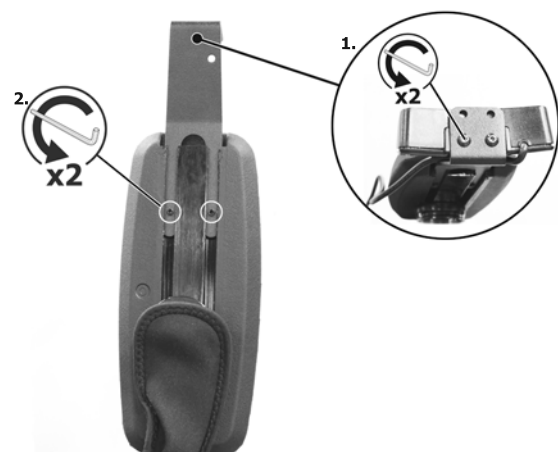Assembly Instruction


Document No. 263612	Revision C				permobil
Production state SERIES					
Preform Date 2014-09-04	Preform By JOF	Approved Date 5/26/2015	Approved By GHT	Page EN 11	

PS junior

3. Fit the USB charger bolt and loosely tighten the nut so that 4 mm remains.

4. Carefully slide the USB into the uni-track rail (the locating pin in one track and the bolt in the other) with the socket facing forward/out.

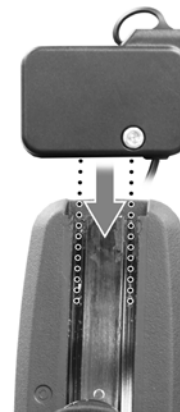5. Reinstall the button panel bracket and then adjust the charger into its proper position and torque the bolt to 9.8 Nm.

Assembly Instruction


Document No. 263612	Revision C				permobil
Production state SERIES					
Preform Date 2014-09-04	Preform By JOF	Approved Date 5/26/2015	Approved By GHT	Page EN 12	

PS junior


6. Run the cable between the armrest and the arm following positions 1-2.

7. Continue the cable run behind the back following positions 3-7. Then run the cable in between the seat base and seat frame.

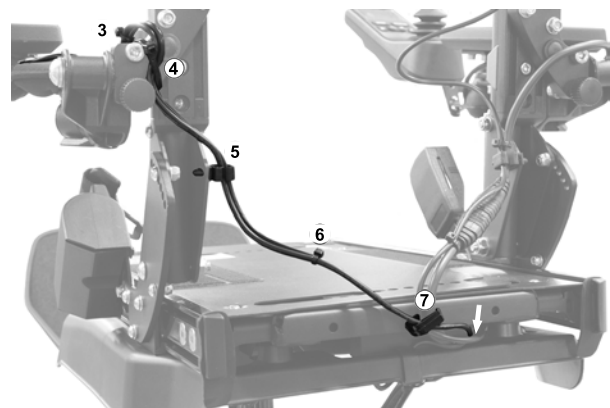
8. Continue to run the cable under the seat. Gather the cable in position 8, and secure it using a cable tie. Connect the wiring.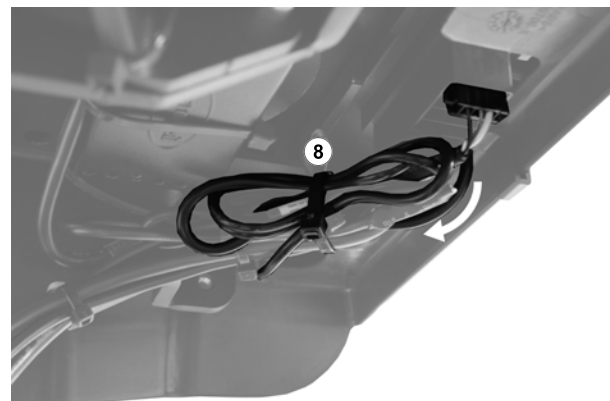

Assembly Instruction

Document No. 263612	Revision C				permobil
Production state SERIES					
Preform Date 2014-09-04	Preform By JOF	Approved Date 5/26/2015	Approved By GHT		Page EN 13

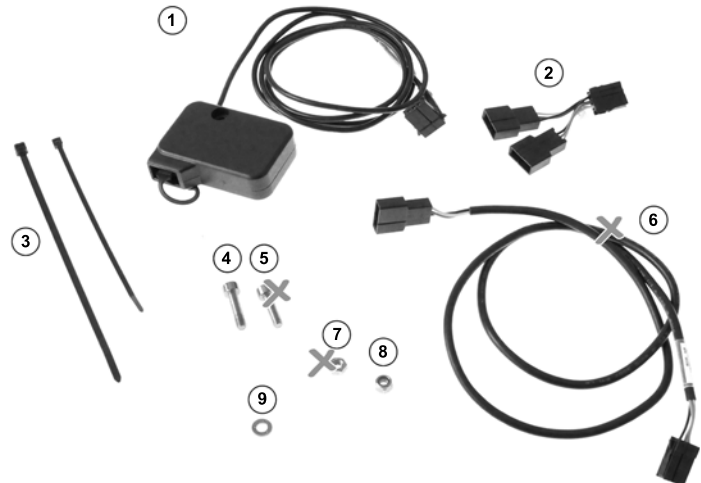
Installation instruction, USB Charger**Miniflex****Parts included, Miniflex**

1. Choose a side for the USB charger. Install it beneath the left or right armrest. Choose the side depending on panel location as the charger should sit on the opposite side.

2. Attach the USB charger to the bracket under the armrest with a M6x25 bolt, washer and locking nut.

Tightening torque 9.8 Nm.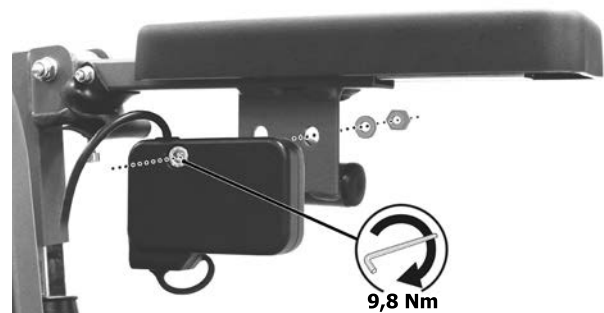


Assembly Instruction


Document No. 263612	Revision C				permobil
Production state SERIES					
Preform Date 2014-09-04	Preform By JOF	Approved Date 5/26/2015	Approved By GHT	Page EN 14	

Miniflex

3. Raise the armrest for ease of cable installation. Follow positions 1 and 2 with strap; allow a generous length of cable at the armrest hinge to avoid cable wear when the armrest is raised.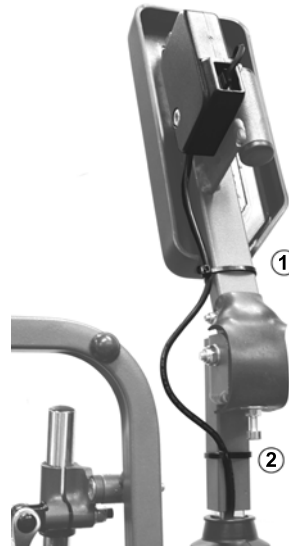

4. Remove the armrest by undoing the two bolts on the rear of the backrest and then slide off the pleated plastic protection from the armrest rod.

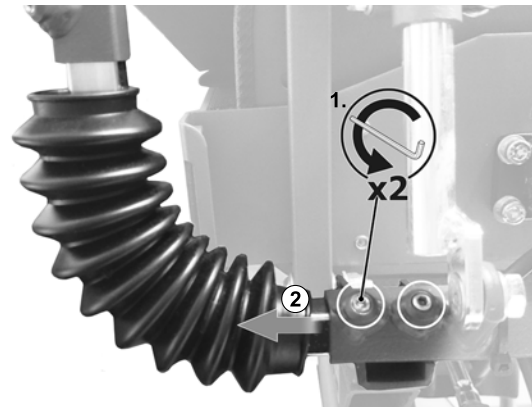5. Run the cable through the pleated plastic protection and re-install the plastic protection on the armrest rod. Then re-install the armrest.


Assembly Instruction


Document No. 263612	Revision C				permobil
Production state SERIES					
Preform Date 2014-09-04	Preform By JOF	Approved Date 5/26/2015	Approved By GHT	Page EN 15	

Miniflex

6. Run the cable on the inside of the seat frame, behind the back rest. Fashion a bend in the cable and secure it using a strap in position 1. Then follow positions 2-7 to continue the cable run.

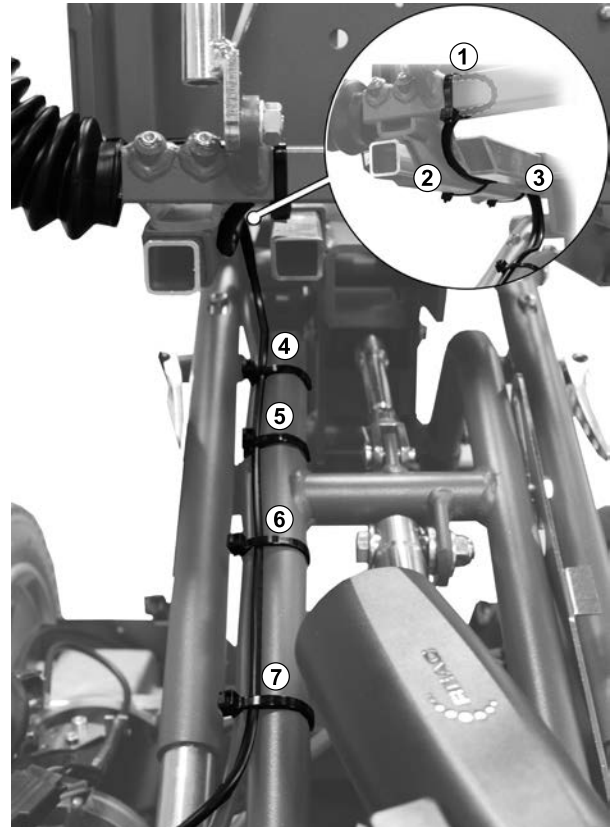
7. Continue to run the cable to the ICS box located on the left side of the chassis. Connect using the splitter provided.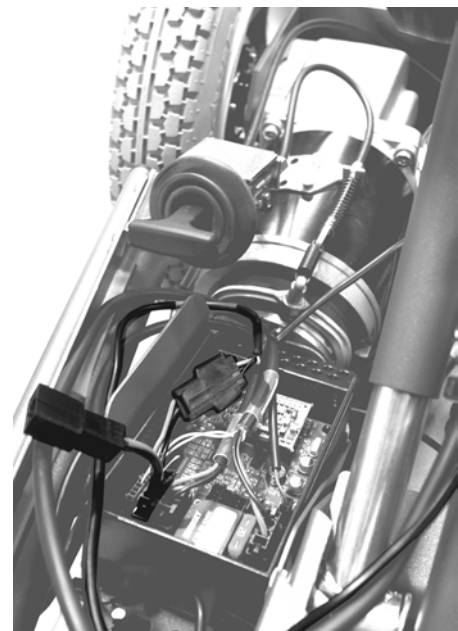

Assembly Instruction

Document No. 263612	Revision C				permobil
Production state SERIES					
Preform Date 2014-09-04	Preform By JOF	Approved Date 5/26/2015	Approved By GHT	Page EN 16	

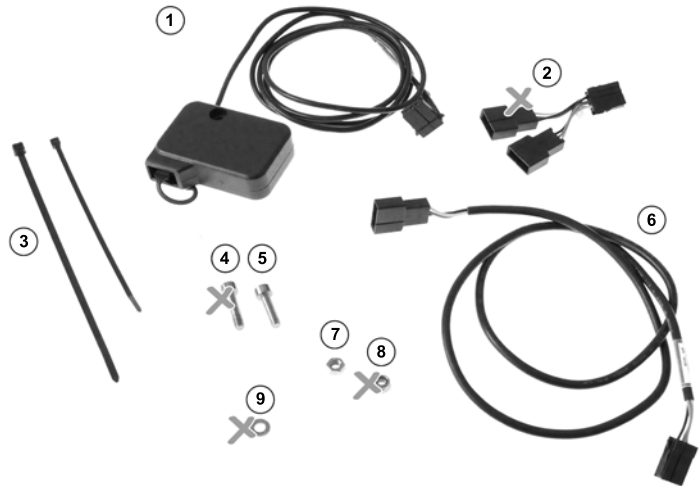Installation instruction, USB Charger**K450****Parts included, K450**

1. Choose a side for the USB charger. Install it beneath the left or right armrest. Choose the side depending on panel location as the charger should sit on the opposite side.

2. Fit the nut into the left track in the unitrack underneath the armrest, then insert the M6x22 bolt into the USB charger and fit the latter into the unitrack with the guide pin in the right track and the bolt in the nut with the socket facing forward/outward. Then torque the bolt to 9.8 Nm.

Assembly Instruction


Document No. 263612	Revision C				permobil
Production state SERIES					
Preform Date 2014-09-04	Preform By JOF	Approved Date 5/26/2015	Approved By GHT	Page EN 17	

K450


3. Run the cable to the rear between the armrest padding and the armrest. Follow positions 1 and 2 using cable ties, then run the cable down and in below the back attachment.

4. Unlock the knobs on either side of the backrest attachment (1) and tilt the seat forward for easier access to the cable clamps. Follow positions 3-8 and secure the cable using the existing clamps. When the cable is securely clamped at all six points, run it through the lowest hole and to the rear.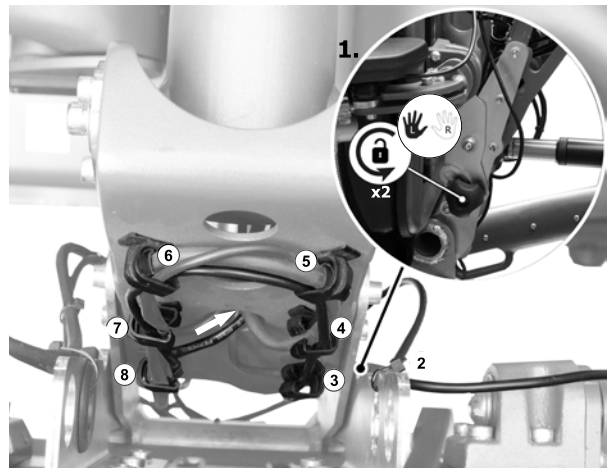

5. Continue to secure the cable in the existing cable clamps along the lifting arm, follow positions 9-11.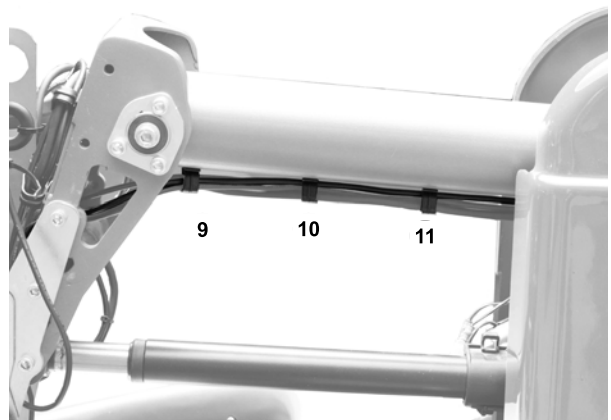


Assembly Instruction


Document No. 263612	Revision C				permobil
Production state SERIES					
Preform Date 2014-09-04	Preform By JOF	Approved Date 5/26/2015	Approved By GHT	Page	EN 18

K450

6. Continue to run the cable to the end of the lifting arm; secure it in the existing cable cable clamps in positions 12-13. Connect the extension cable to the USB cable and secure it in the existing cable clamp in position 14. Coax the extension cable through the hole at the bottom of the rear plate (follow the existing cable run).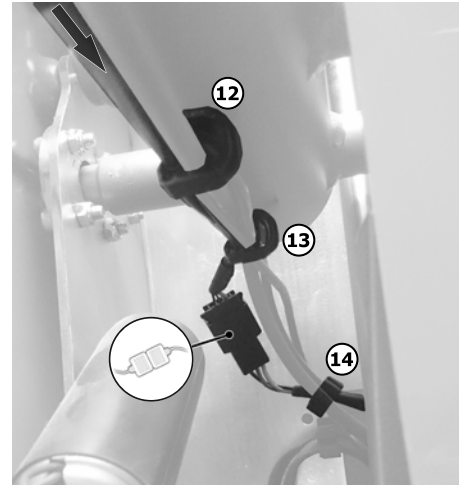

7. Remove the two bolts that secure the back cover. Remove the cover and then undo the two bolts that secure the ICS attachment plate.

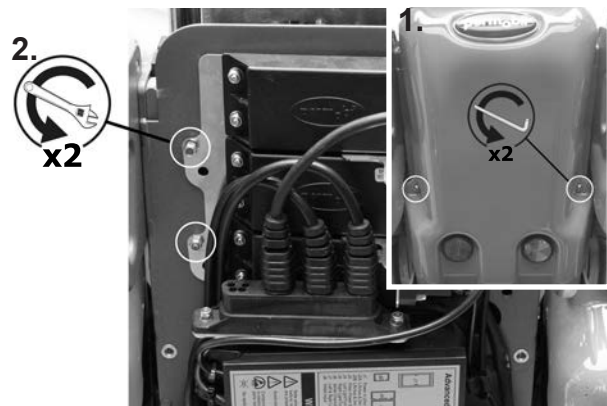8. Pull out the cable that was threaded through the hole at the bottom of the rear plate. Because space in this location is extremely limited, secure the cable in the existing cable clamps (positions 14-19) by using the holes in the ICS/cable attachment plate. Run the cable through the big hole at the top right and then connect the wiring to the ICS box. Re-install the attachment plate and back cover.