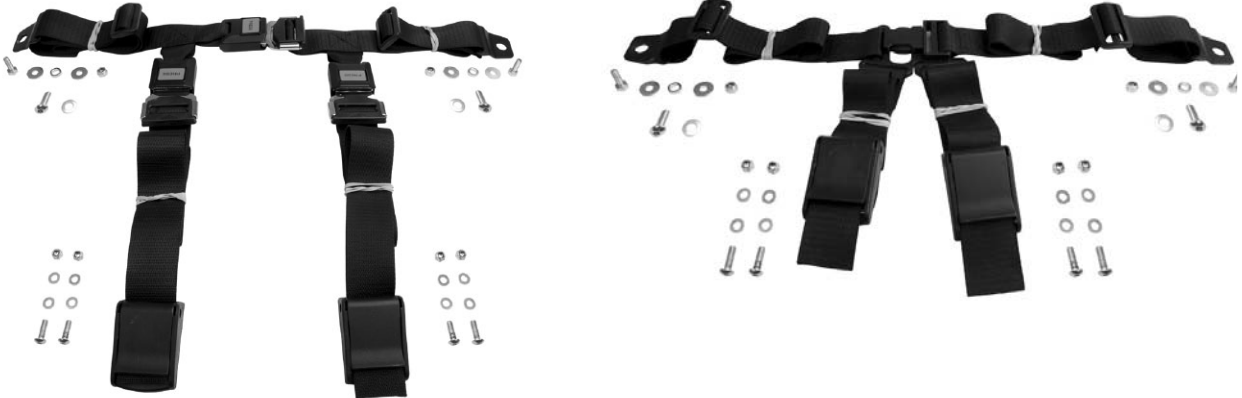
Assembly Instruction

Document No. 236439	Revision B				
Production state SERIES					
Preform Date 2008-12-19	Preform By TSN	Approved Date 5/15/2012	Approved By GHT	Page 1/8	

Assembly of 4-point belt**List of parts that should be in the set.**

Qty. 1. Belt	Qty. 2 Screw M8x20 K6S
Qty. 2 Three slide	Qty. 2 Washer, BRB 8,4x16x1,6
Qty. 2 Bracket, Fix-lock	Qty. 2 Spacer
Qty. 4 Screw, M5x16 K6S	Qty. 2 Screw M6x16
Qty. 8 Washer 5,3x10x1	Qty. 4 Washer 6,4x18x1,6
Qty. 4 Locking nut, M5	Qty. 2 Locking nut M6
Qty. 2 Belt tongues	

Read the instructions carefully *before* starting assembly.

Mounting on PS Junior

1. Mount the two Fix Lock Brackets on the upper edge on the back of the back rest plate, use the two accompanying screws (M5x12), washers and nuts, Mount the brackets in the lower hole pattern. See fig. below to the left.
2. Mount the upper part of the belt in the Fix Lock brackets. See fig. below to the right.
3. Adjust the length of the belt.

Mount the two Fix Lock Brackets on the upper edge on the back of the back rest plate.

Mount the upper part of the belt in the Fix Lock brackets.

Assembly Instruction

Document No. 236439	Revision B				
Production state SERIES					
Preform Date 2008-12-19	Preform By TSN	Approved Date 5/15/2012	Approved By GHT	Page 2/8	

4. Mount the belt tongues on the lower parts of the belt using the three-slide.


The belt tongues are mounted with a three-slide. Ensure that the shorter part of the belt is lowest so that it is clamped in place.


5. Loosen the three nuts, on the left and right hand of the seat, holding the back rest. See fig.
6. Remove the plastic plugs on the rear end of the accessory rail, then remove the back rest by pulling it out of the rail.
7. Mount one of the accompanying screws (M6x16) in the upper groove of the rail on each side of the seat. Remount the back rest.
8. Mount the belt on the screw earlier mounted in the rail. Use accompanying washers and nuts. See fig. below.

 **CAUTION**

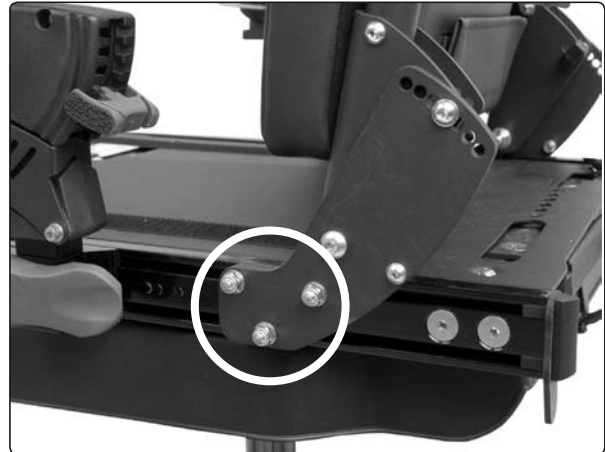Ensure that you mount the belt brackets in the right direction so that they can rotate freely.

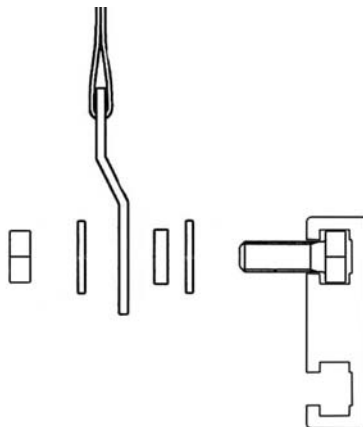The back rest is mounted with three screws and nuts on the left and right side of the seat.


The belt is mounted on the accessory rail.

Assembly Instruction

Document No. 236439	Revision B				
Production state SERIES					
Preform Date 2008-12-19	Preform By TSN	Approved Date 5/15/2012	Approved By GHT	Page 3/8	


Mounting on MiniFlex

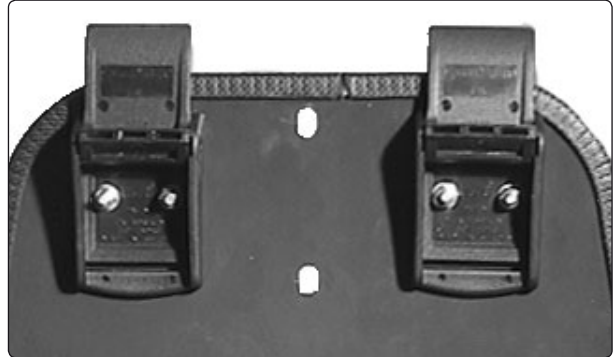1. Mount the two Fix Lock Brackets on the upper edge on the back of the back rest plate, use the two accompanying screws (M5x12), washers and nuts, See fig.

Mount the two Fix Lock Brackets on the upper edge on the back of the back rest plate.

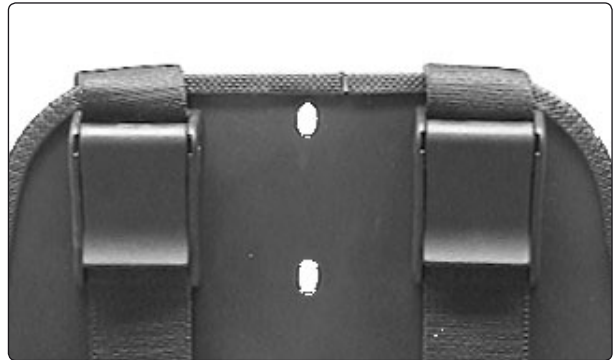2. Mount the upper part of the belt in the Fix Lock brackets. See fig.
3. Adjust the length of the belt.

Mount the upper part of the belt in the Fix Lock brackets.

4. Mount the belt tongues on the two lower parts of the belt. Secure the belt tongues with the three slides. See fig.

The belt tongues are mounted with a three-slide. Ensure that the shorter part of the belt is lowest so that it is clamped in place.


Assembly Instruction

Document No. 236439	Revision B				
Production state SERIES					
Preform Date 2008-12-19	Preform By TSN	Approved Date 5/15/2012	Approved By GHT	Page 4/8	


5. Mount te belt with belt tongues on the seat using the accompanying screw (M8x16), washers and spacers. See fig.

 **CAUTION**

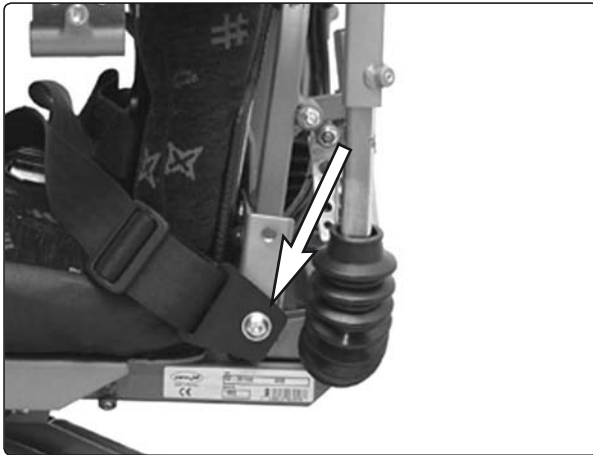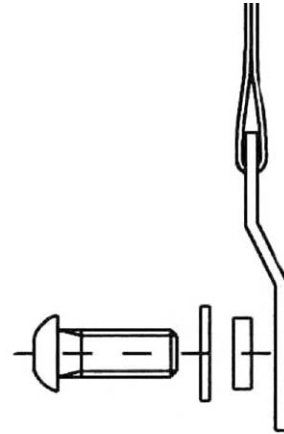Ensure that you mount the belt brackets in the right direction so that they can rotate freely.

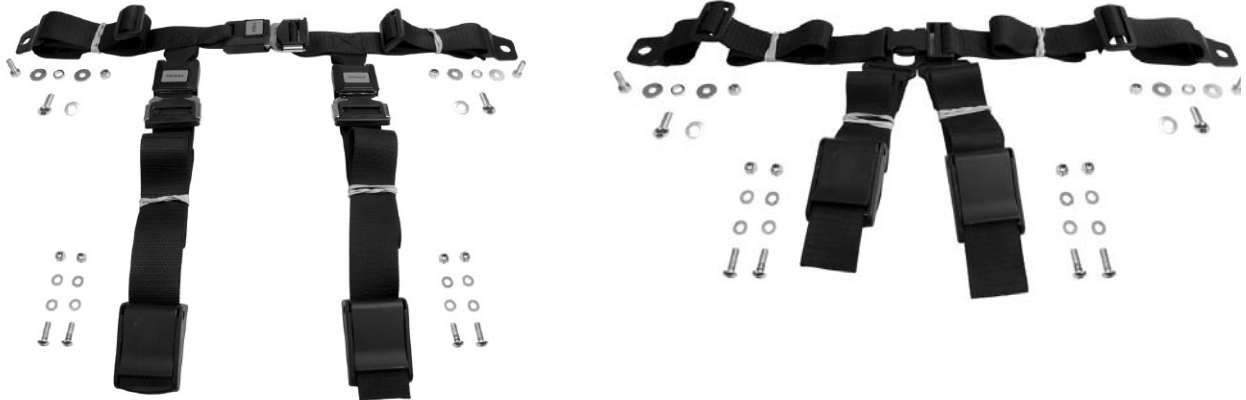
The belt is mounted on the seat.

The belt is mounted with accompanying screws, washers and spacers.

Assembly Instruction

Document No. 236439	Revision B				
Production state SERIES					
Preform Date 2008-12-19	Preform By TSN	Approved Date 5/15/2012	Approved By GHT	Page 5/8	

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Assembly Instruction

Document No. 236439	Revision B				
Production state SERIES					
Preform Date 2008-12-19	Preform By TSN	Approved Date 5/15/2012	Approved By GHT	Page 6/8	

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
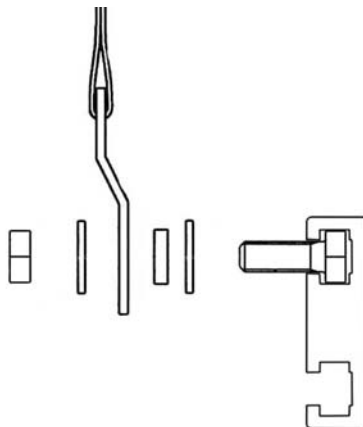
Assembly Instruction

Document No. 236439	Revision B				
Production state SERIES					
Preform Date 2008-12-19	Preform By TSN	Approved Date 5/15/2012	Approved By GHT	Page 7/8	

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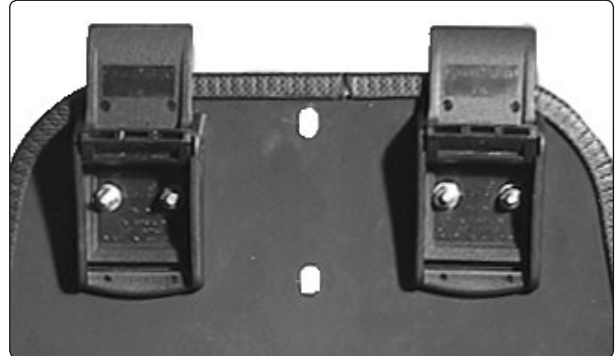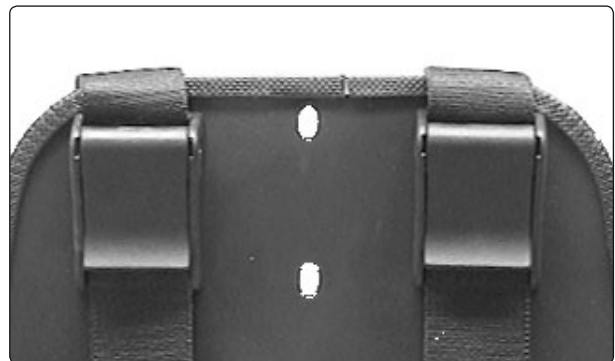
Assembly Instruction

Document No. 236439	Revision B				
Production state SERIES					
Preform Date 2008-12-19	Preform By TSN	Approved Date 5/15/2012	Approved By GHT	Page 8/8	

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