
Permobil Adjustable Tray

Assembly instruction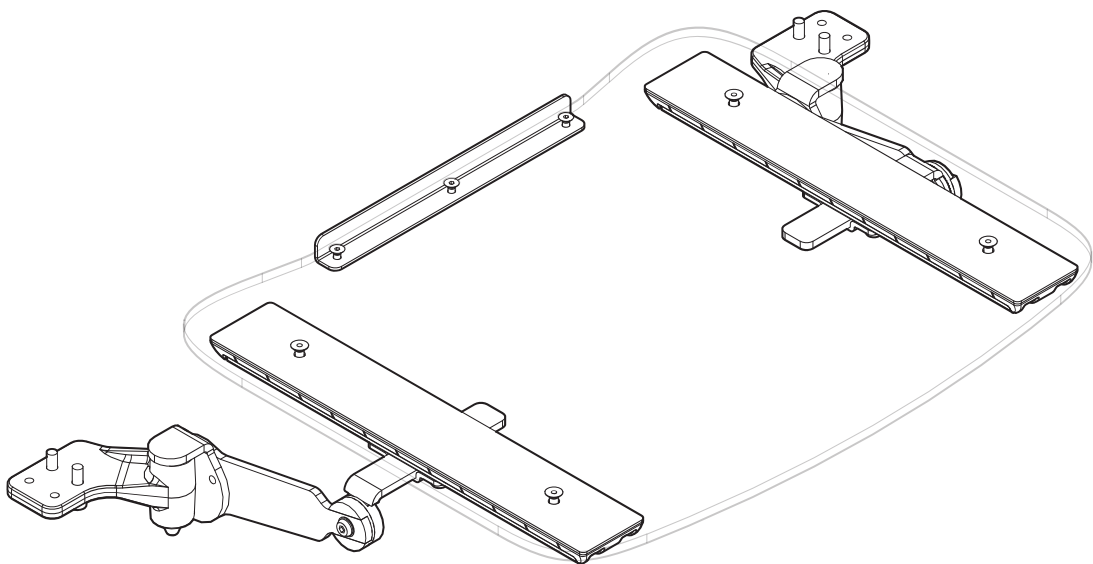

Thank you for choosing to be part of the Permobil family!

This is the Assembly Instruction of your new accessory. An assembly instruction is not a stand-alone document, but rather a complement to the wheelchair's User's Manual. It can sometimes refer to the User's Manual or the Service Manual.

This Assembly Instruction contains important information, installation instructions, and adjustment instructions. The instructions on how to install the accessory are intended for the service technician only. All other parts of the Assembly Instruction are intended for both the service technician and the user of the wheelchair. In the section Safety information, you find definitions of the admonitions used in this Assembly Instruction. Translations of these definitions can be found in the section Safety instructions. In the section General warnings for this product you find instructions for admonitions used in this Assembly Instruction, if any.

We hope that you are going to enjoy your new accessory. This accessory is intended to be used with a Permobil wheelchair. For more information regarding indoor and outdoor use, and for further information about your wheelchair, please see the wheelchair's User's Manual.

Contents

Important information	4
Safety information	5
Preparation.....	6
General warnings for this product	7
Installation.....	8
Height adjustment.....	14
Angle adjustment	16
Depth and side adjustment	18
Safety instructions.....	20
Contact us	25

Important information

Read this manual

Read this manual, including all admonitions and instructions, before you install, maintain, or use your accessory. Failure to follow these instructions and admonitions can lead to bodily injury or damage to the accessory or wheelchair.

Visually impaired

This document can be downloaded at www.permobil.com. If you are visually impaired, use the magnifying glass in your PDF reader to achieve desired text and picture size.

Order or download a copy of the instructions

If you need another copy of this Assembly Instruction, you can order one from Permobil. Please state the order number specified on the back cover. You can also download a copy from our website: www.permobil.com

Service and repairs

Please refer to the wheelchair's User's Manual for information about service and repairs.

Reporting an incident

Please contact Permobil or your dealer if you want to report an incident. The accessory does not have a serial number. However, if the accessory is attached to the wheelchair, we will ask for the wheelchair's serial number. This serial number is located on the chassis.

Warranty and expected service life

Contact your dealer or Permobil for information about the warranty period of this product. You can find the conditions of the warranty and further information about applicable warranties on our website.

This accessory has an expected service life of five years.

Scrapping and recycling

Contact Permobil for information about scrapping and recycling agreements in force.

Safety information

The following safety labels are used throughout this manual to draw attention to items of significant importance to safety:

Labels

Warning

Signal word used to indicate a hazardous situation which, if not avoided, could result in personal injury or death.

Crushing

Risk of crushing hands or other body parts.

Notice

Signal word used to indicate informative remarks.


Preparation

Necessary tools

Assembly kit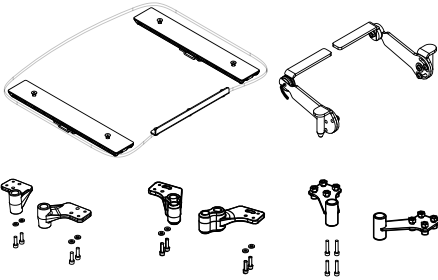

General warnings for this product

① Warning

Do not use the tray during transit. It should be removed and stowed.

MUL page 20

② Warning

Do not overload the tray. It can support up to 10 kg (22 lbs).

MUL page 20

③ Warning

Do not operate the chair with items on the tray.

MUL page 21

⑤ Warning

Do not use the tray for repositioning.

MUL page 21


⑥ Warning


Use of the tray, especially when loaded, can degrade the stability and maneuverability of the wheelchair.

MUL page 22

Installation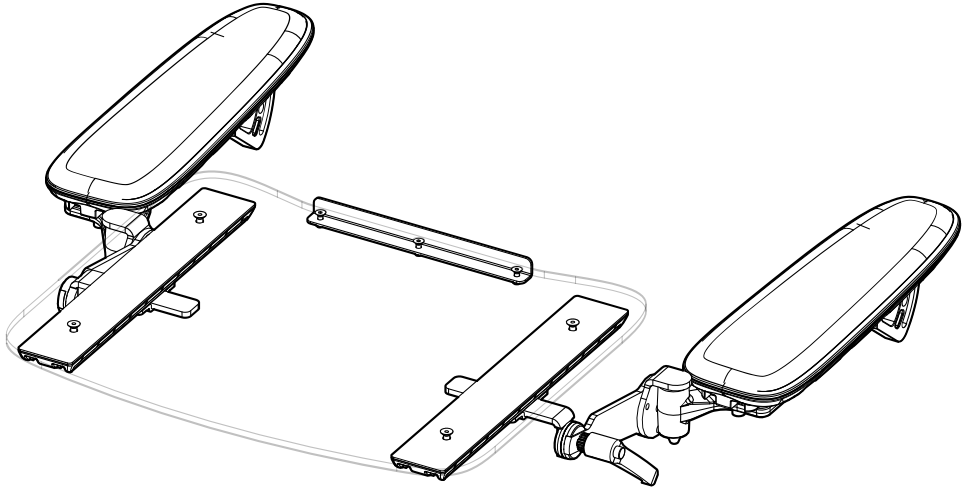


1.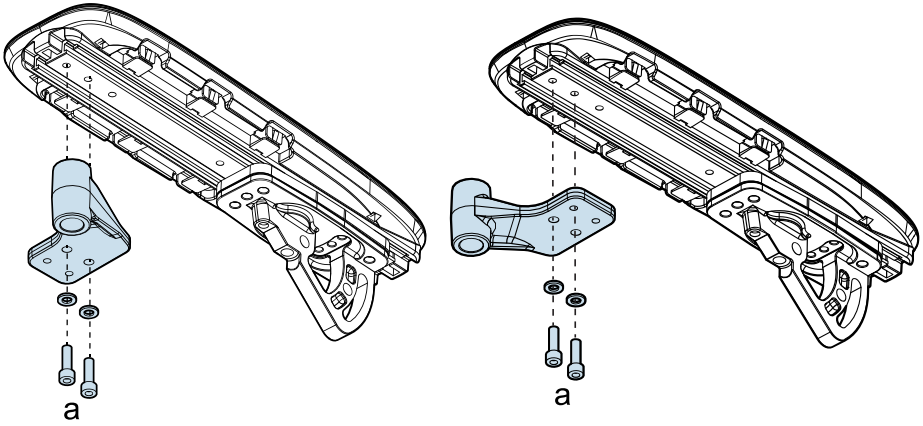

Installing Corpus, PS Jr and MX receivers.

a = 9,8 Nm


2.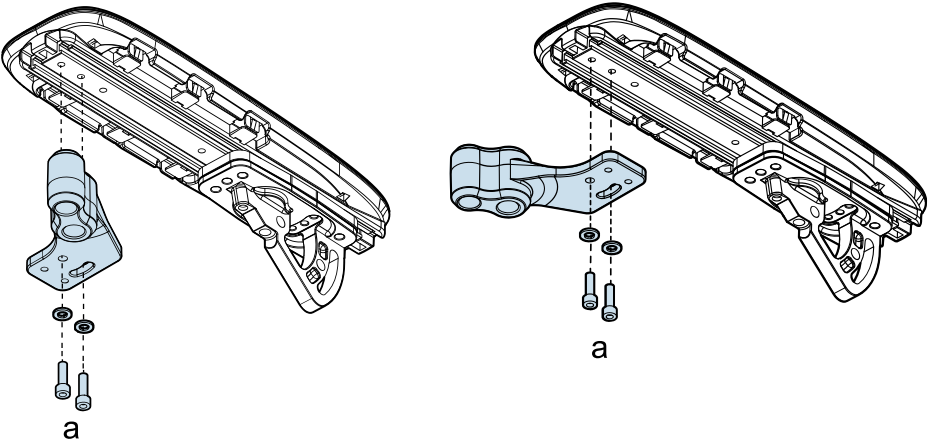

Installing of Corpus VS receivers.

a = 9,8 Nm

3.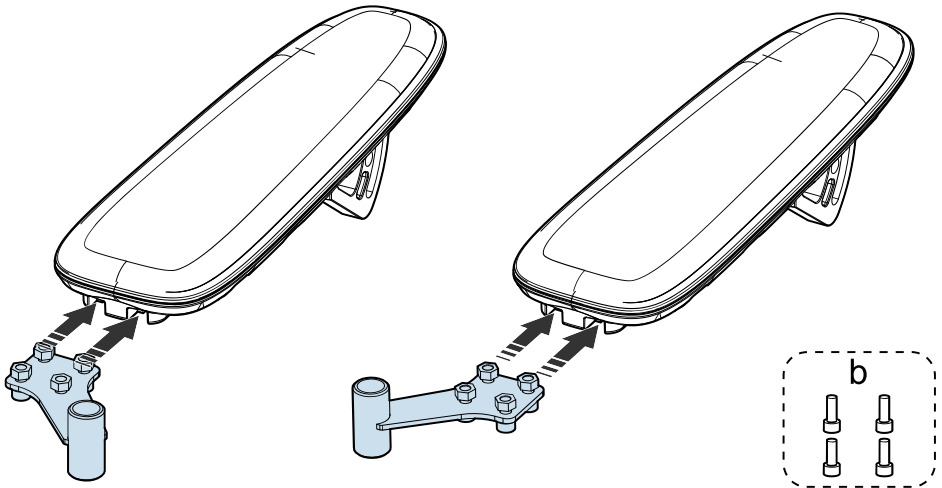


Installing M1 receivers.


a = 9,8 Nm

b: Use longer M6x16 bolts if the receivers are mounted with a Fixed Joystick Mount. In this scenario, the M1 receiver should be flush with the UniTrack rail and the Fixed Joystick Mount should be attached directly underneath, with the M6x16 bolts going through both brackets.


4.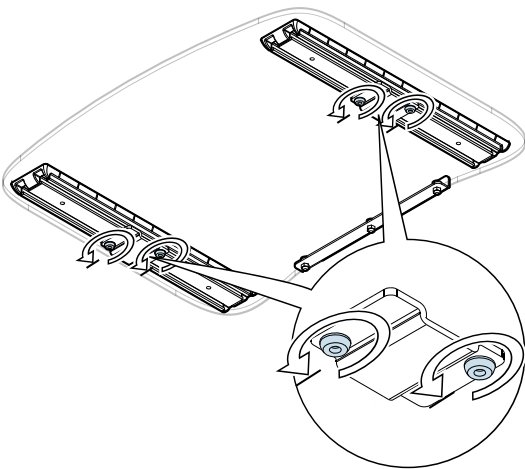

5.

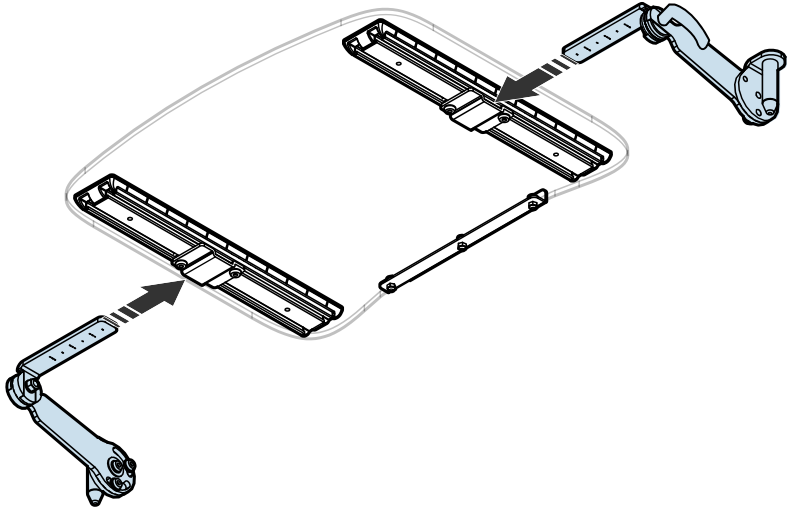
6.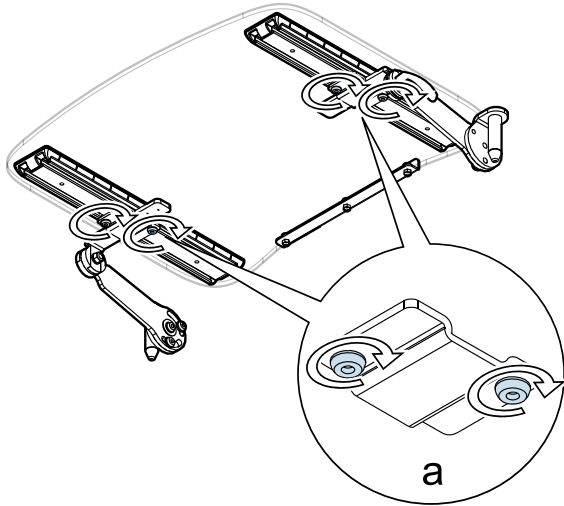

a = 9,8 Nm

7.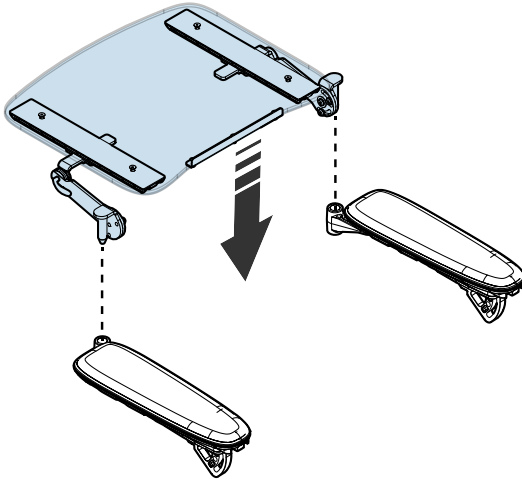


① Crushing

Keep hands and legs clear from the hardware when fitting the tray into the receivers.

MUL page 23

8.

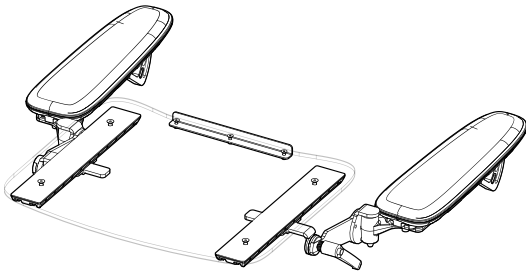④ Warning

Do not raise the armrests when the tray is installed.

MUL page 21

⑦ Warning

Make sure that the user is clear from the receiver before lifting the armrest. Not doing so can cause injuries to the upper body or the face.

MUL page 22


① Notice

Check hardware at least once every 3 months and tighten bolts when needed.


MUL page 24

Height adjustment


1.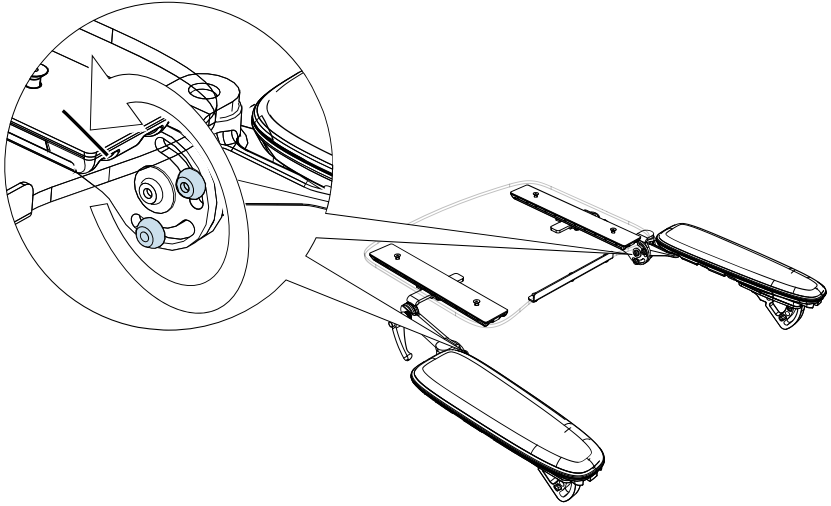

2.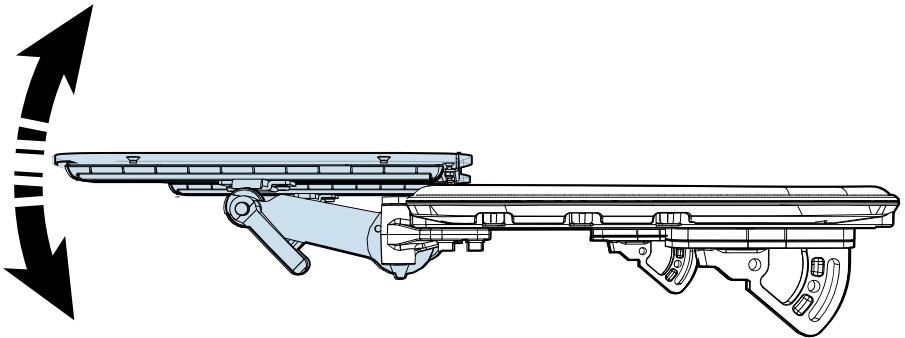


3.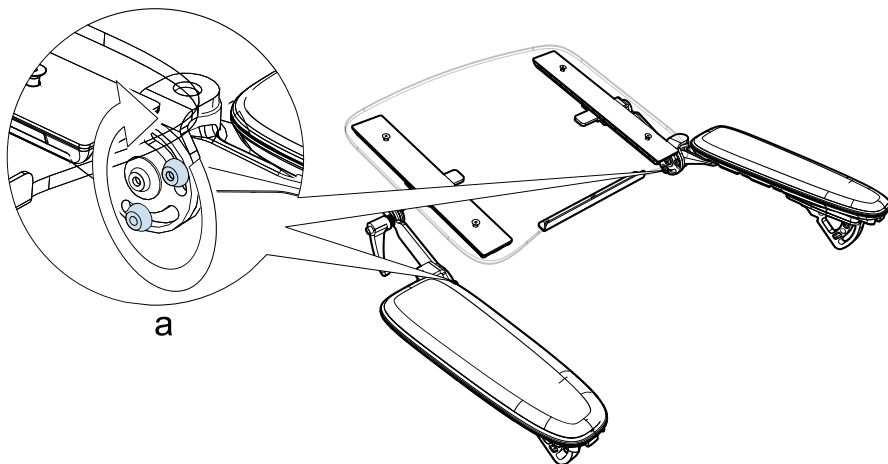


a = 9,8 Nm

Angle adjustment


1.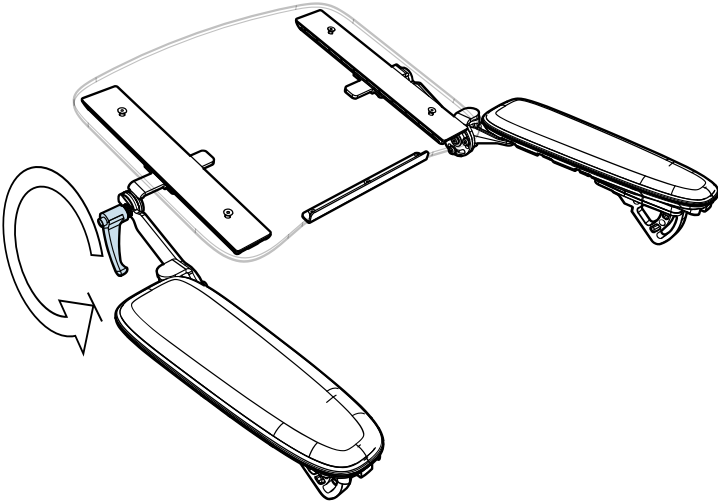

2.

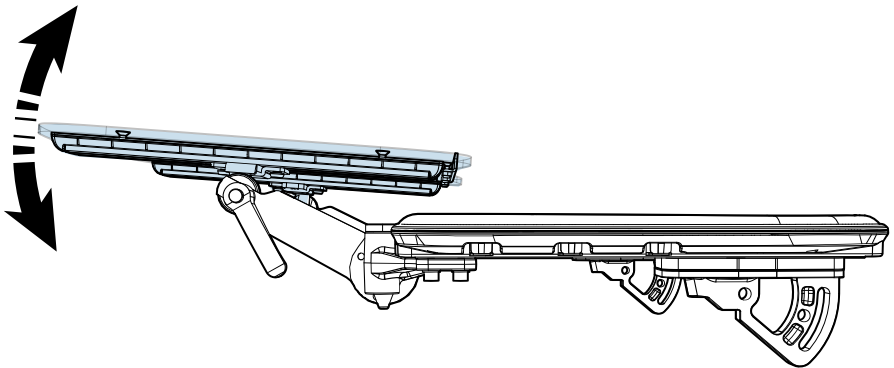
3.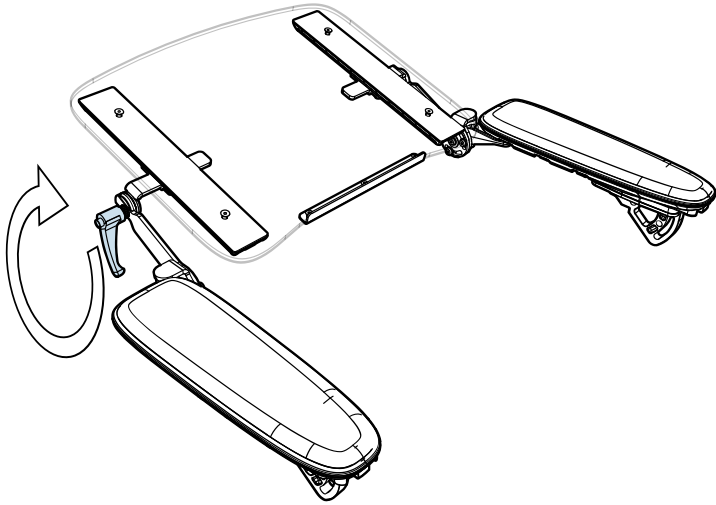


Depth and side adjustment


1.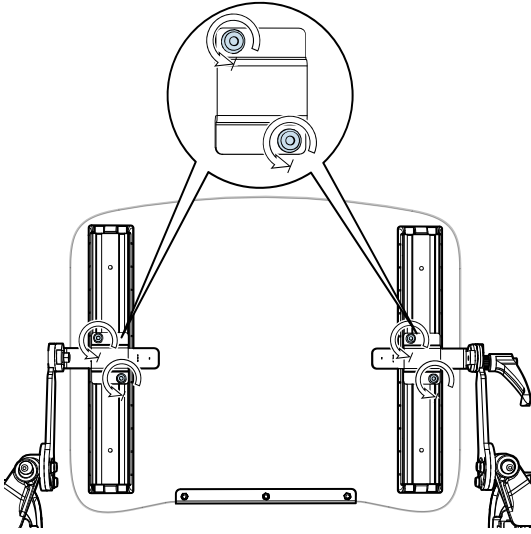

2.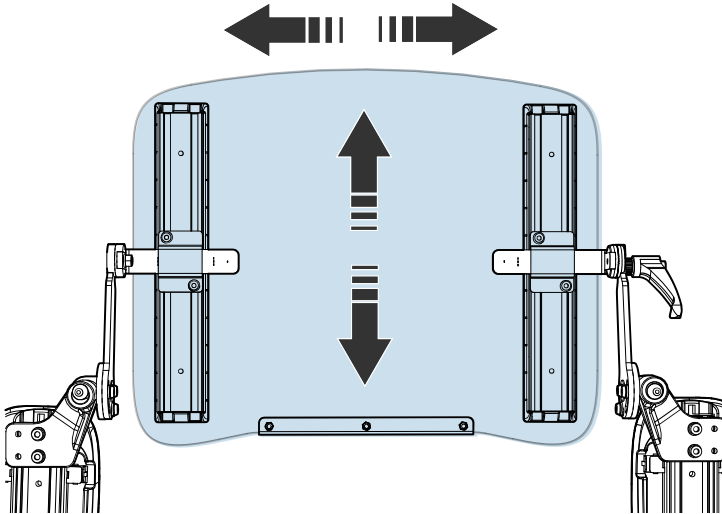

3.

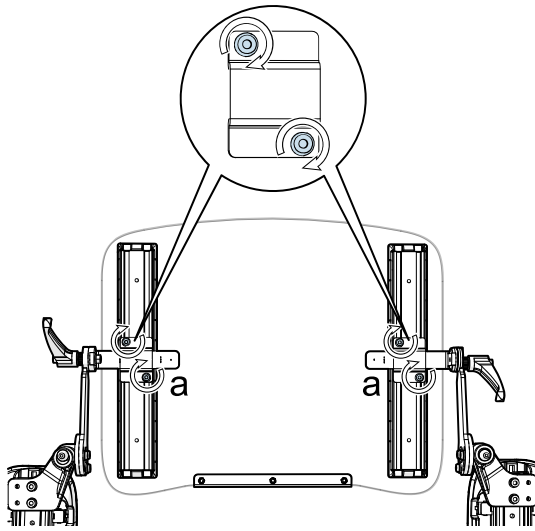$a = 9,8 \text{ Nm}$

Safety instructions

Warning

① Warning

- US:** Do not use the tray during transit. It should be removed and stowed.
- DE:** Verwenden Sie den Tisch nicht bei der Fahrt. Er sollte entfernt und verstaut werden.
- UK:** Do not use the tray during transport. It should be removed and stowed.
- CA:** N'utilisez pas le plateau pendant le transport. Il doit être retiré et rangé.
- FR:** Ne pas utiliser le plateau pendant les déplacements. Celui-ci doit être retiré et rangé.
- NL:** Gebruik het blad niet tijdens vervoer. Het dient te worden verwijderd en opgeborgen.
- NB:** Bruk ikke brettet under transport. Det må tas av og pakkes ned.
- US:** Não usar a bandeja durante o transporte. Deve ser removido e estivado.
- ES:** No utilice la bandeja durante el transporte. Se debe retirar y almacenar.
- SV:** Använd inte brickan under transport. Den ska tas bort och stivas undan.

② Warning

- US:** Do not overload the tray. It can support up to 10 kg (22 lbs).
- DE:** Den Tisch nicht überladen. Er kann mit bis zu 10 kg belastet werden.
- UK:** Do not overload the tray. It can support up to 10 kg (22 lbs).
- CA:** Ne surchargez pas le plateau. Il peut supporter jusqu'à 10 kg (22 lb).
- FR:** Ne jamais surcharger le plateau. Il peut supporter jusqu'à 10 kg.
- NL:** Belaad het blad niet te zwaar. Het kan maximaal 10 kg aan.
- NB:** Brettet må ikke overbelastes. Det tåler en belastning på opptil 10 kg.
- US:** Não sobrecarregar a bandeja. Suporta até 10 kg (22 lb).
- ES:** No sobrecargue la bandeja. Puede soportar hasta 10 kg (22 libras).
- SV:** Överbelasta inte brickan. Den klarar en belastning på upp till 10 kg.

③ Warning

US: Do not operate the chair with items on the tray.

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UK: Do not operate the chair with items on the tray.

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④ Warning

US: Do not raise the armrests when the tray is installed.

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UK: Do not raise the armrests when the tray is installed.

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⑤ Warning

US: Do not use the tray for repositioning.

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UK: Do not use the tray for repositioning.

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⑧ Warning

US: To prevent forward tipping, do not anteriorly tilt the seat or angle the backrest forward when the tray is in use.

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UK: To prevent forward tipping, do not tilt the seat or angle the backrest forward when the tray is in use.

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Crushing

① Crushing

US: Keep hands and legs clear from the hardware when fitting the tray into the receivers.

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UK: Keep hands and legs clear from the hardware when fitting the tray into the receivers.

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Notice

① Notice

US: Check hardware at least once every 3 months and tighten bolts when needed.

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UK: Check hardware at least once every 3 months and tighten the bolts when needed.

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permobil

Produced and published by Permobil

Edition: 2

Date: 2018-07-31

Order no: 334555 mul

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Permobil AB
Permobil Inc.