Assembly Instructions

Document No. 248625	Revision A.00				
Production state					
Date performed 2011-10-06	Performed TSN	Date Approved	Approved By	Page 1	

Attendant Control Panel


List of the parts included in the set

Qty. 1. Attendant Control Panel


Read the instructions carefully *before* starting work

Assembly on Corpus 3G

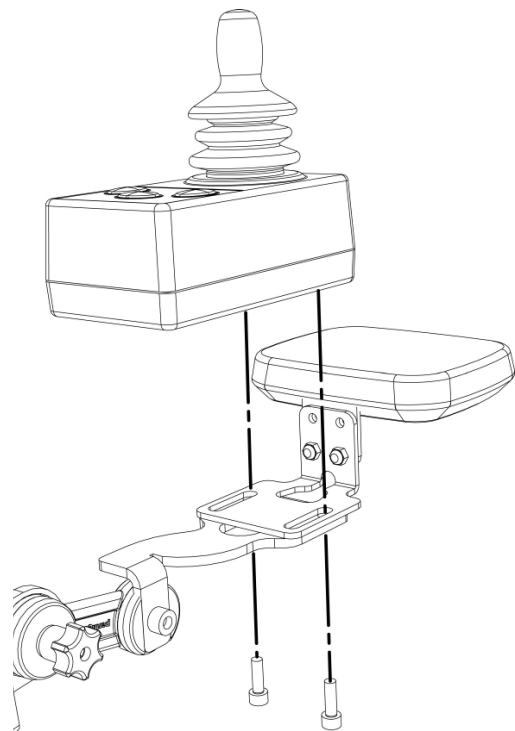1. Fit the control panel on the control panel bracket using the associated screws. See the illustration to the right.
2. Loosen the two screws holding the plastic cover to the back of the backrest. Twist the plastic cover downwards. See the illustration below.
3. Connect the BUS-cable to the contact block on the back of the back rest. See the illustration below.
4. Refit the plastic cover using the two screws. See the illustration below.

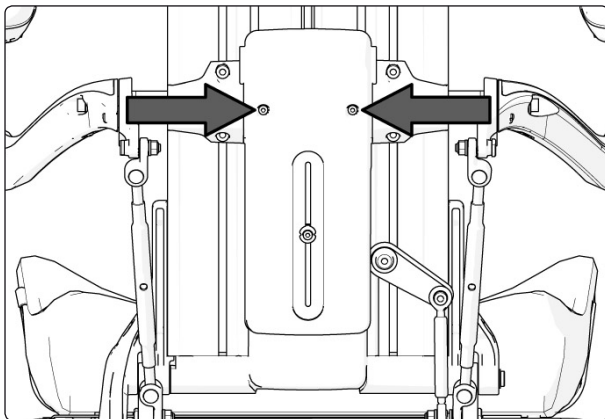Fit the control panel on the control panel bracket using the associated screws.

The plastic cover is fixed using two screws.

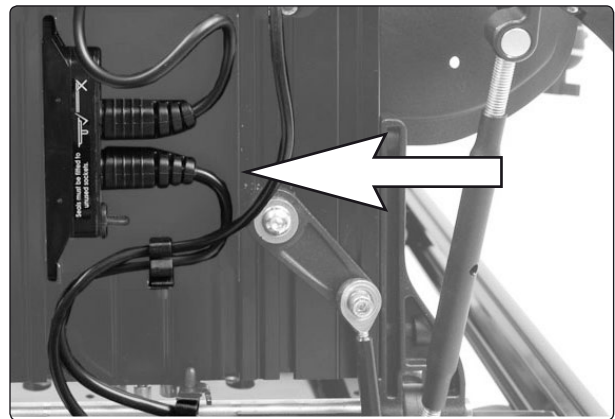
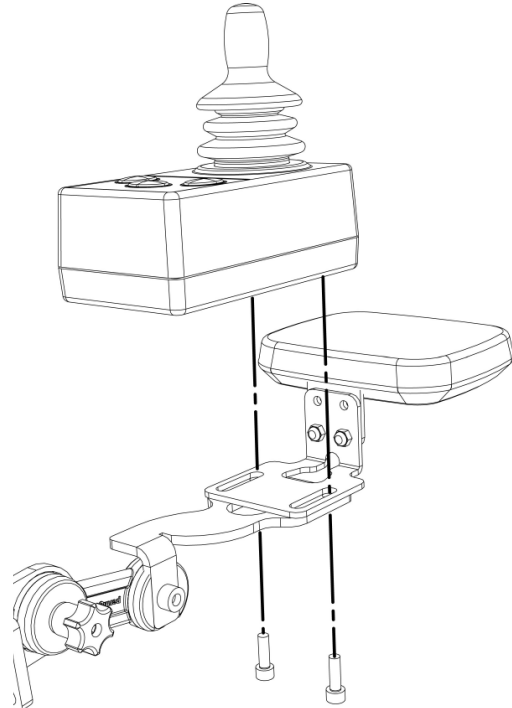
Fig 7. Connect the BUS-cable to the contact block on the back of the back rest.

Assembly Instructions


Document No. 248625	Revision A.00				
Production state					
Date performed 2011-10-06	Performed TSN	Date Approved	Approved By		Page 2

Assembly on C400/C500


1. Fit the control panel on the control panel bracket using the associated screws.
2. Remove the chassis cover, it's fitted with two knobs at the front edge and one knob at the back. See the illustration below.
3. Connect the BUS-cable to the contact block in the chassis. See the illustration below.
4. Fit the BUS cable together with existing cables between the chassis and the seat using cable ties.
5. Refit the cover using the three knobs. See the illustration below.

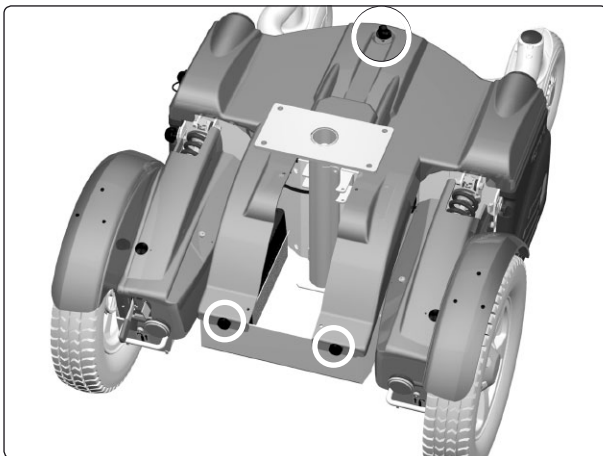Fit the control panel on the control panel bracket using the associated screws.

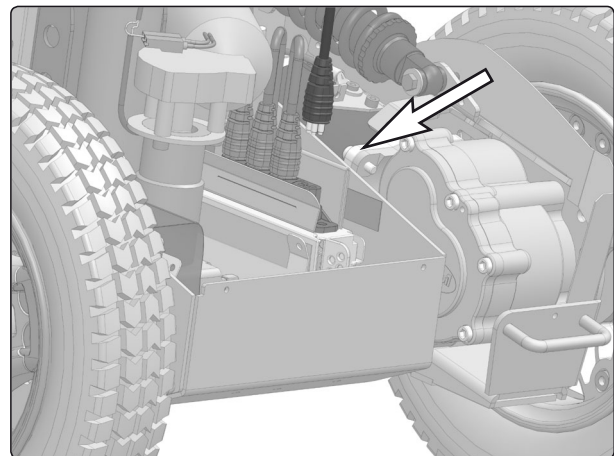The chassis cover is mounted with two knobs at the front edge and one knob at the back.

Connect the BUS-cable to the contact block in the chassis.

Assembly Instructions

Document No. 248625	Revision A.00				
Production state					
Date performed 2011-10-06	Performed TSN	Date Approved	Approved By	Page 3	

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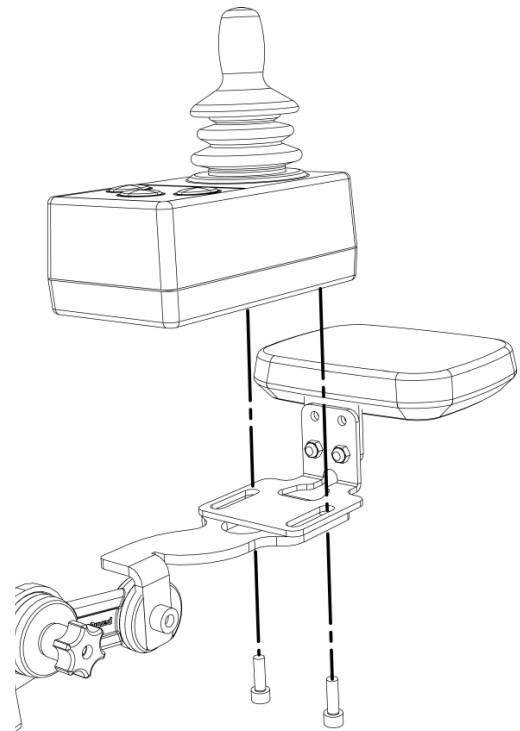
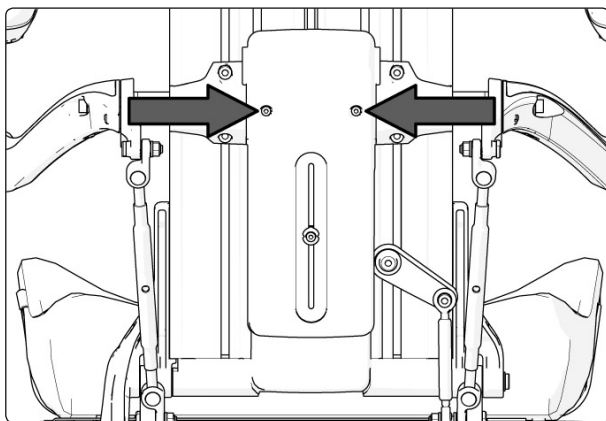
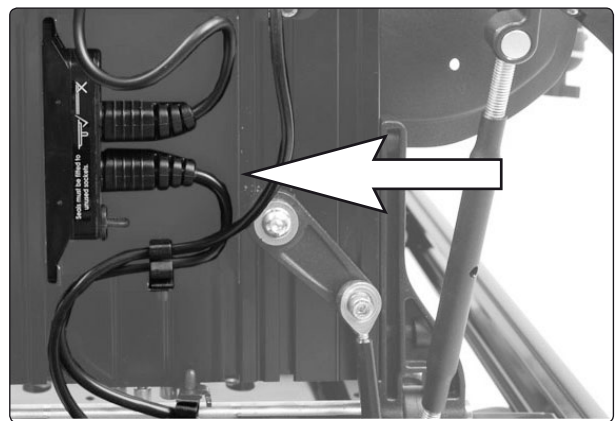
The plastic cover is fixed using two screws.

Connect the BUS-cable to the contact block on the back of the back rest.

Assembly Instructions

Document No. 248625	Revision A.00				
Production state					
Date performed 2011-10-06	Performed TSN	Date Approved	Approved By	Page 4	

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