**Assembly Instruction**

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**Assembly Instruction**

Document No. 228685	Revision C				
Production state <b>SERIES</b> (S, B, C, D)					
Preform Date 2007-04-25	Preform By BTA	Approved Date 2011-01-20	Approved By GHT	Page 4	

**Thigh support***Order no.: 103086-99-0**Order no.: 103091-99-0**Order no.: 102976-99-0**Order no.: 103090-99-0**Order no.: 103092-99-0**Order no.: 103095-99-0**Order no.: 102977-99-0**Order no.: 103094-99-0***List of parts that should be in the set****Always included:**

- 2 pcs Brackets.
- 4 pcs Nuts, M6.
- 4 pcs Screws, M6x10.

**Depending on model, the following will be included:**

- 2 pcs Cushions, B.
- 2 pcs Covers for cushion B.
- 2 pcs Cushions, C.
- 2 pcs Covers for cushion C.
- 2 pcs Cushions, G.
- 2 pcs Covers for cushion G.
- 2 pcs Cushions, J.
- 2 pcs Covers for cushion J.

**Read the instructions carefully *before* starting installation**

*Fig. 1. Cushion B. The set with order number 103086-99-0 contains cushions and brackets, Fig. 6. Order number 103092-99-0 also contains cushion covers.*

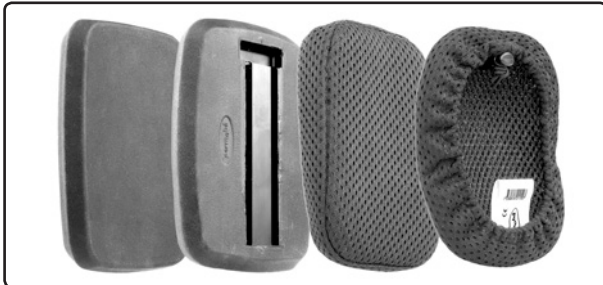
*Fig. 2. Cushion C. The set with order number 103091-99-0 contains cushions and brackets, Fig. 6. Order number 103095-99-0 also contains cushion covers.*

*Fig. 3. Cushion G. The set with order number 102976-99-0 contains cushions and brackets, Fig. 6. Order number 102977-99-0 also contains cushion covers.*

**Assembly Instruction**

Document No. <b>228685</b>	Revision <b>C</b>				
Production state <b>SERIES</b>					
Preform Date <b>2007-04-25</b>	Preform By <b>BTA</b>	Approved Date <b>2011-01-20</b>	Approved By <b>GHT</b>	Page <b>5</b>	


*Fig. 4. Cushion J. The set with order number 103090-99-0 contains cushions and brackets, Fig. 6. Order number 103094-99-0 also contains cushion covers.*

**Assembly of cushion and hip support bracket**

The assembly method is the same regardless of the cushion alternative selected.

The cushion can be identified by the letter of the alphabet moulded into it. Figure 5.

The thigh support bracket is universal and used on all combinations. Figure 6.

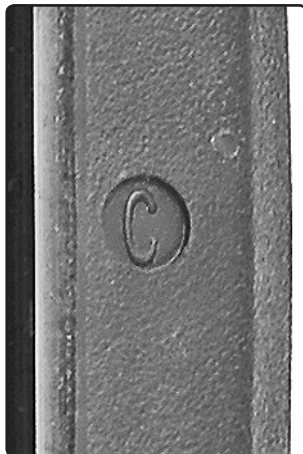*Fig. 5. The moulded identification letter.*

*Fig. 6. Brackets for thigh supports.*

Mount the two screws on the bracket for accessory attachment. Screw in the nuts only a few turns. See figure 6.

Remove the plastic plug mounted on some of the cushions. Slide the nuts into the two grooves. See figure 7. Slide the bracket in towards the centre of the cushion. Adjust the cushion to a position which suits the user. Tighten the bracket screws. Replace the plastic plugs on those cushions which have them.

*Fig. 7. Assembly of cushion and bracket. Left. If the cushion has a plastic plug, remove it. Then slide the nuts into the grooves on the cushion. Right. Fix the cushion in the desired position by tightening the screws. Remount any plastic plugs.*

**Assembly Instruction**


Document No. <b>228685</b>	Revision <b>C</b>				
Production state <b>SERIES</b>					
Preform Date <b>2007-04-25</b>	Preform By <b>BTA</b>	Approved Date <b>2011-01-20</b>	Approved By <b>GHT</b>	Page <b>6</b>	

**Attachment of accessory**

Remove the plastic plug from the front edge of the accessory rail, see Fig. 9. Undo the screws on the bracket holder, Fig. 8, so that the nuts can slide easily in the grooves accessory rail grooves. Replace the plastic plug. Slide the holder towards the middle of the rail. Tighten the holder screws. This is the starting position and may need to be re-adjusted when adapting the thigh support to the user. Undo the screw lever on the bracket holder. Set the thigh support in place in the holder. Tighten the lever. Figure 10.

*Fig. 8. Thigh support holder.*

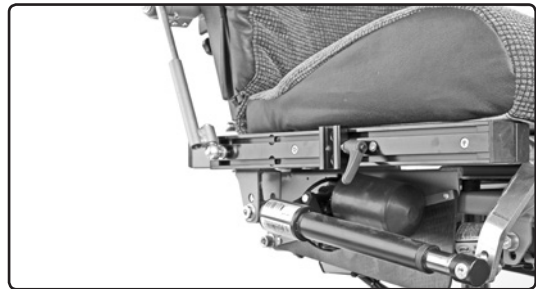*Fig. 9. The bracket holder mounted on the accessory rail.*

**Adjustment of accessory**


The thigh support is adjusted with the two articulated joints and by moving the cushion horizontally, Figs. 7 and 11. The height position is fixed with the stop screw on the bracket. The stop screw must be in contact with the edge of the holder, Fig. 10.

The distance between the thigh supports is set by the two articulated joints, Fig. 11.

The distance to the back rest is adjusted with the screws holding the cushion, Fig. 7, and by adjusting the bracket in the accessory rail, Fig. 9. If necessary, adjust the height position by undoing the stop screw and moving it to a suitable hole. Tighten the stop screw. Fig. 10.

Loosen the screws in both the articulated joints and set to the desired position. Fix the position by tightening the screws. Loosen the screws holding the cushion in place, and adjust the distance to the back rest. Fix the position by tightening the screws. Figure 11. If the cushion's adjustment range is insufficient, alter the position of the bracket holder on the accessory rail.

**NOTE! Make sure the thigh support doesn't collide with the armrest and the control panel. Also make sure its bracket doesn't collide with the wheelchair chassis when the seat lift is in its lowest position. If needed, adjust the position of the bracket holder on the accessory rail and the position of the cushion on the bracket.**

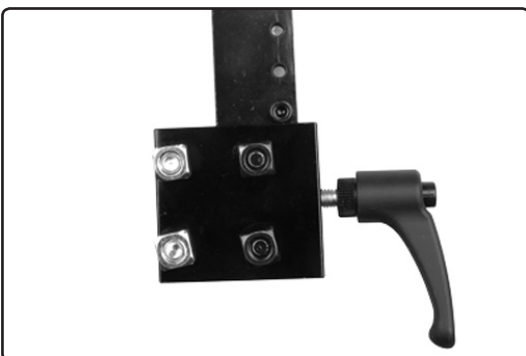*Figure 10. Screw holes and stop screw for fixing the support's height position.*

*Fig. 11. Adjusting the thigh support to the user's requirements.*

**PS seat, Miniflex and KB seat**

The PS seat, Miniflex and KB seat have no accessory rail, Fig. 9. Special accessory brackets are available for these seats.