

permobil

RAM® X-GRIP® 10" TABLET HOLDER INSTALLATION



XG10TAB043014

RAM® X-GRIP® 10" TABLET HOLDER INSTALLATION GUIDE

OVERVIEW

The RAM® X-Grip® 10" Tablet Holder is designed to hold most 10" screen tablets, with or without a protective case, including all models of the Apple iPad (excluding the iPad Mini). The following guide illustrates how to attach the holder to the UniTrack rail under the armrest.



Warning: Be careful of pinch risk associated with swing-away mechanism.

Warning: Mount tablet holder within the widest point of the chair to avoid hitting objects.

Caution: Refrain from using tablet when driving wheelchair to prevent accidents.

Caution: Permobil accepts no liability for any damage or destruction to consumer electronic devices or other personal property contained within or used in conjunction with the RAM® X-Grip® 10" Tablet Holder.

RAM® X-GRIP® 10" TABLET HOLDER INSTALLATION GUIDE



Slide Panel Mount onto UniTrack rail and tighten upper screws with a 5 mm Allen wrench.



Slide Swing-Away Mount post into Panel Mount and tighten clamp screw with a 5 mm Allen wrench.


NOTE: Make sure swing-away joint is accessible and not blocked by the front of armrest.



Orient the upper arm vertically and tighten with 13 mm wrench. Position the clamp far enough forward to avoid interfering with the swing-away mechanism. The clamp lock nuts should face outward for better access and positioning.




Attach the Mounting Bracket to the back of the Tablet Holder using the four self-tapping screws and a Phillips screwdriver.

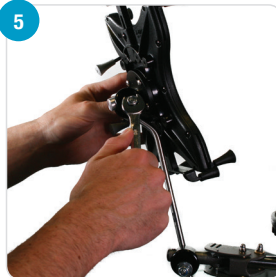

Slide the Mounting Bracket post into the upper post clamp and tighten with a 13 mm wrench.



Position the upper arm and orient the tablet as desired. This may require loosening one or both post clamps.

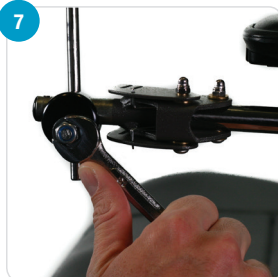

Once the tablet is positioned properly, securely tighten all panel mount screws and post clamps nuts. This is critical to ensure the security of the tablet and its desired position.



To mount tablet, squeeze the grey knobs on the side of the Tablet Holder to expand the arms and insert tablet. Release knobs to allow the arms to spring back into place.



Reposition the side supports, using a Phillips screwdriver, to provide additional support. These supports may be removed, when necessary, to hold wider tablets.

permobil

THE POWER OF MOBILITY™

