**Assembly Instruction**

Document No. <b>232778</b>	Revision <b>C</b>				
Production state <b>SERIES</b>					
Preform Date 2008-01-30	Preform By BTA	Approved Date 2011-03-01	Approved By GHT	Page 1	


**Table fixed***Order no.: 103426-99-0***List of parts in the set***Fig. 1. Parts included in the set.*

- 1 pc Table. Table top and frame.
- 1 pc. Inner square tube, UniTrack.
- 2 pcs. Square nuts.
- 2 pcs. Screws M6x12.
- 1 pc. Accessory bracket.
- 1 pc. Inner square tube, for older accessory brackets.

**Read the instructions carefully *before* starting installation****Attachment of accessory**

The table can be fixed on the left or right side. Undo the four screws securing the table top, fig. 3. Turn the table frame to suit the chosen alternative.

Remove the forward plastic plug from the accessory rail. Screw the two M6x12 screws and square nuts in the accessory bracket. Figure 2. Only screw the nuts onto the screws by a few turns. Slide the bracket with the nuts mounted into the grooves on the accessory rail. Fig. 4.

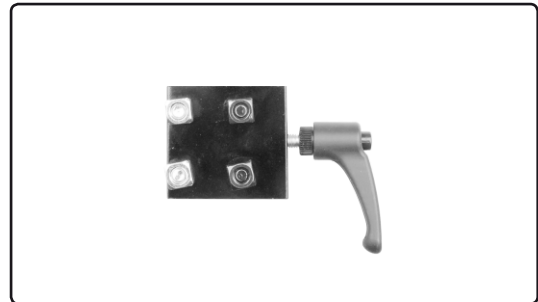*Fig. 3. Screw the two M6x12 screws and square nuts in the accessory bracket. Only screw the nuts onto the screws by a few turns.*

*Fig. 3. Undo the four screws that hold the table top in place. Turn the table top for right or left assembly.*

*Figure 4. Remove the plastic plug from the rail. Slide the accessory bracket into the grooves on the accessory rail.*

**Assembly Instruction**

Document No. <b>232778</b>	Revision <b>C</b>				
Production state <b>SERIES</b>					
Preform Date <b>2008-01-30</b>	Preform By <b>BTA</b>	Approved Date <b>2011-03-01</b>	Approved By <b>GHT</b>	Page <b>2</b>	


Refit the plastic plug. Slide the accessory rail forward towards the plastic plug. Tighten the bracket. Fig. 5. Unscrew the knob on the accessory bracket and mount the inner square tube for UniTrack in the bracket. The head of the stop screw must rest against the top edge of the accessory bracket. Tighten the accessory with the knob.

Screw out the two stop screws on the outer square tube of the frame. Mount the table on the inner square tube. Figure 6. Check that the table has been lowered so far that both stop screws engage with the inner square tube. Tighten the stop screws.

Remove the protective plastic from the table top.

*Fig. 5. The bracket mounted as far forward as the fully inserted plastic plug will allow.*

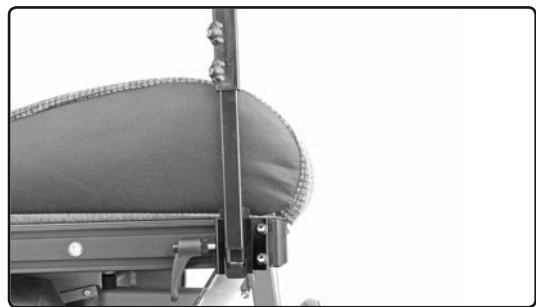
*Fig. 6. The table mounted on the accessory holder. Check that the top part of the table has been lowered so far that both fixing screws engage with the inner square tube.*

**Adjustment of accessory**

Adjust the distance between the user and the table by undoing the two stop screws fixing the table top in position along the central square profile. Figure 7. Tighten the stop screws once the desired position has been reached.

Adjust the table top sideways by releasing the table top from the frame, fig. 3. Move the central square profile to a suitable screw hole in the table top fixing. Figure 7. Tighten the screws and refit the table top. Tighten the four screws which hold the table top in position.

Adjust the table top angle by undoing the knob, fig. 8. Adjust to the desired position. Tighten the knob.

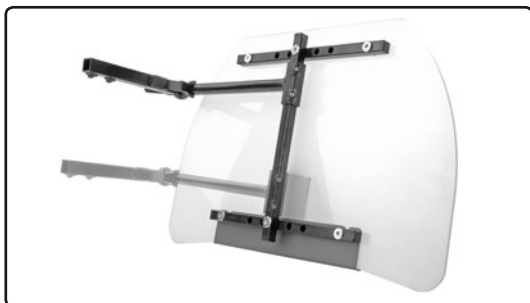*Fig. 7. Underside of table. The distance of the table top to the user is adjusted with the two screws which hold the table top in position along the central square profile. The table top is adjusted sideways by moving the central square profile to one of the other screw holes in the table top fixing.*

*Fig. 8. The table top angle is adjusted with the frame knob.*

**Assembly Instruction**

Document No. <b>232778</b>	Revision <b>C</b>				
Production state <b>SERIES</b>					
Preform Date <b>2008-01-30</b>	Preform By <b>BTA</b>	Approved Date <b>2011-03-01</b>	Approved By <b>GHT</b>	Page <b>3</b>	

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**Assembly Instruction**

Document No. <b>232778</b>	Revision <b>C</b>				
Production state <b>SERIES</b>					
Preform Date <b>2008-01-30</b>	Preform By <b>BTA</b>	Approved Date <b>2011-03-01</b>	Approved By <b>GHT</b>	Page <b>4</b>	

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