Assembly Instruction

Document No. 248085	Revision B				
Production state SERIES					
Preform Date 2011-07-07	Preform By TSN	Approved Date 6/12/2013	Approved By GHT	Page 1	

Mounting trunk support

List of parts in the set.

1 pc. Trunk support bracket
2 pcs. Washer ISO 7093-1 (5,3x15x1)

2 pcs. Screw ISO 7380 M5x12

Mounting

Read the instructions carefully *before* starting installation.

1. Remove the seat cushion.
2. Remove the existing trunk support, save the screw and nut for fitting of the new trunk support. Fig.1. You can not have the two different trunk supports mounted simultaneously.
3. Mount the alignment rail on the back rest using the two attached screws and nuts. Fig. 2.
4. Mount the trunk support using the screw and nut previously removed. Fig. 3.

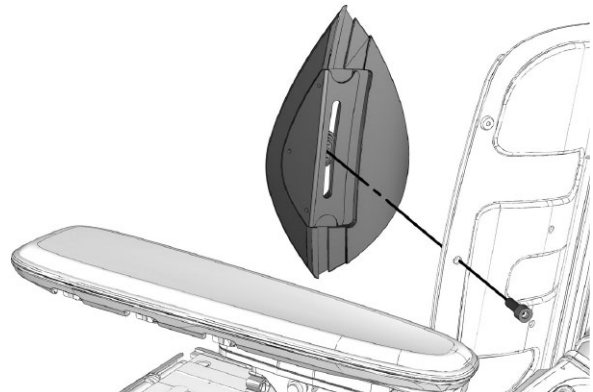
Fig. 1. Removing the standard trunk support.

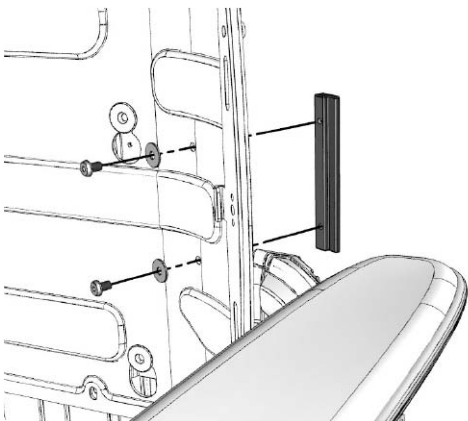
Fig. 2. Mounting the alignment rail on the back rest.

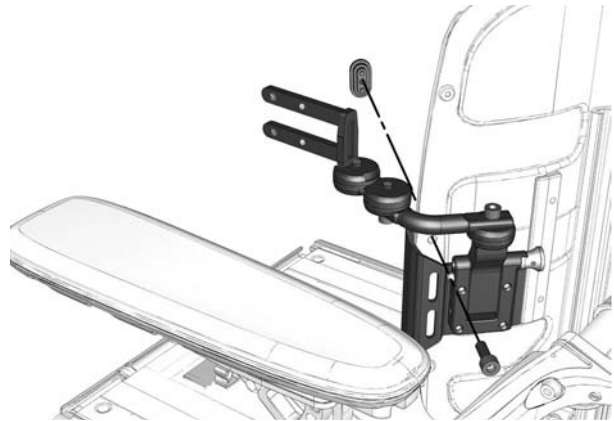
Fig. 3. Mount the trunk support using the screw and nut previously removed.

Assembly Instruction

Document No. 248085	Revision B				
Production state SERIES					
Preform Date 2011-07-07	Preform By TSN	Approved Date 6/12/2013	Approved By GHT	Page 2	

5. Mount a cushion (Separately ordered) by inserting the two tabs of the trunk support into the UniTrack rails on the cushion. Fix into position by tightening the screws on the tabs. Fig. 4.
6. Reassemble the backrest cushion.

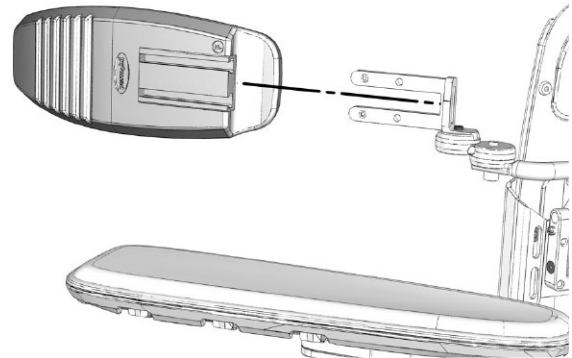
Fig.4. Mount a cushion by inserting the two tabs of the trunk support into the UniTrack rails on the cushion.

Adjusting

1. Loosen the screw (B). Adjust the support to the desired height and secure by retightening the screw. Fig. 5.
2. Loosen the screws (A). Adjust the support to a comfortable position for the user and secure by retightening the screws. Fig. 5.

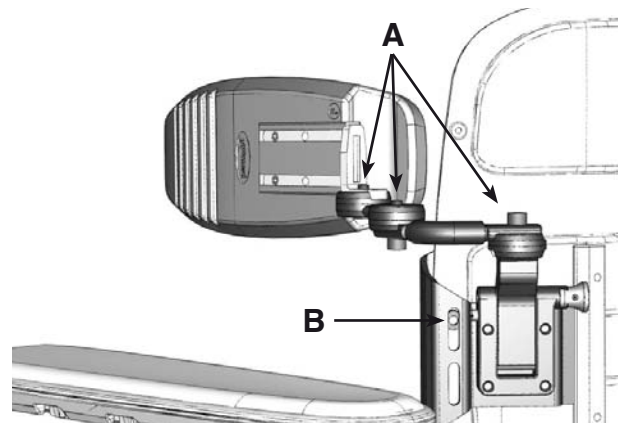
Fig.5. Adjusting the Trunk support.

Quick removal/fitting

The Trunk support can when needed quickly be removed/fitted. Removal and fitting is possible using only one hand.

Removal: Pull the locking pin straight out and rotate it approx. 90 ° to fix it in the unlocked position. See Fig. Fold the trunk support upwards/backwards and lift it out of its bracket, see Fig.

Fitting: Line up the trunk support on the axle at the lower part of the bracket. See Fig. Fold the trunk support downwards/forwards until the trunk rest is fixed with a "click". Rotate the locking pin to the locked position. See Fig.

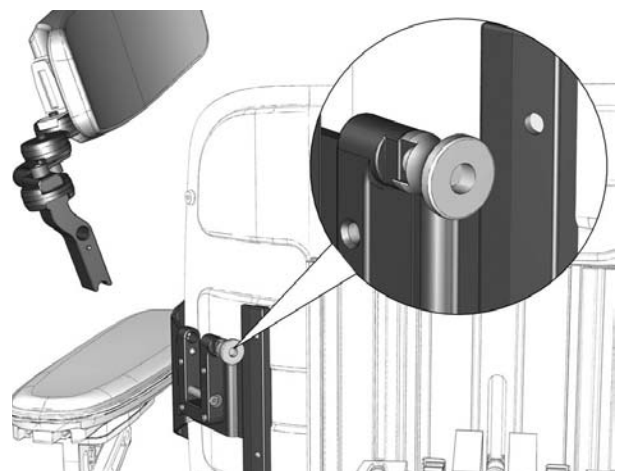
Bild 5. Quick removal/fitting of the Trunk Support.

Assembly Instruction

Document No. 248085	Revision B				
Production state SERIES					
Preform Date 2011-07-07	Preform By TSN	Approved Date 6/12/2013	Approved By GHT	Page 3	

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