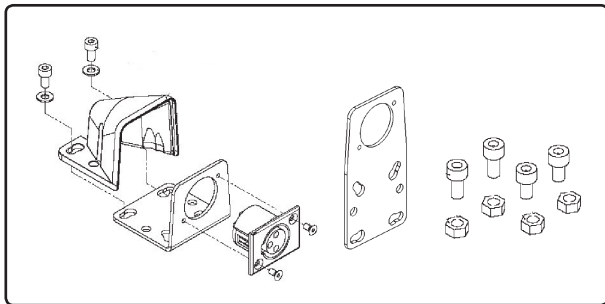
**Assembly Instructions**

Document No. <b>239576</b>	Revision <b>B</b>				
Production state <b>SERIES</b>					
Date performed <b>2009-10-23</b>	Performed by <b>ETSN</b>	Date Approved <b>2011-01-04</b>	Approved By <b>GHT</b>	Page <b>1</b>	

**Charger connector R-net***Order no.: 1823043***List of the parts included in the set***Parts included in the set*

- Qty. 1. Charger connector with cable.
- Qty. 1. Plastic cover.
- Qty. 1. Angled bracket.
- Qty. 1. Flat bracket.
- Qty. 2. Screws, M3x6.
- Qty. 2. Screws, M4x8.
- Qty. 2. Screws, M6x12.
- Qty. 2. Screws, M6x10.
- Qty. 4. Nuts, M6.
- Qty. 2. Washers, 4,3x9x0,8.

**Read the instructions carefully *before* starting work**

Charger connector can be fitted on either the armrest or on the seat frame.




**Assemble on the arm rest**

- Use the two enclosed M3x6 screws to fit the connector on the angled bracket. See fig. above.

The charger connector can be fitted in three different positions on the armrest, see fig.

**Position A** (Only for arm rest with length 400 mm.)

- Remove the M5 screw (E) from the arm rest, see fig.
- Fit the charger connector with two enclosed M4x8 screws and washers (D). Refit the M5 screw (E) on the armrest again. See fig.

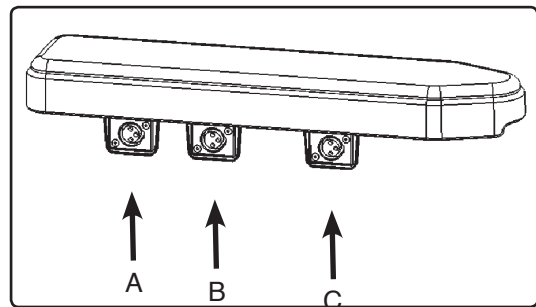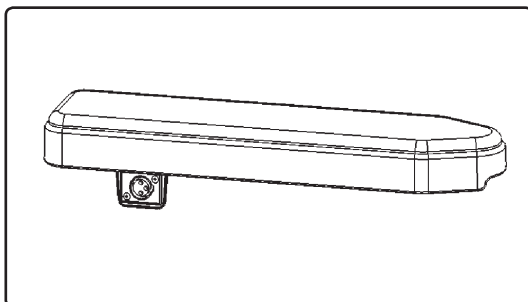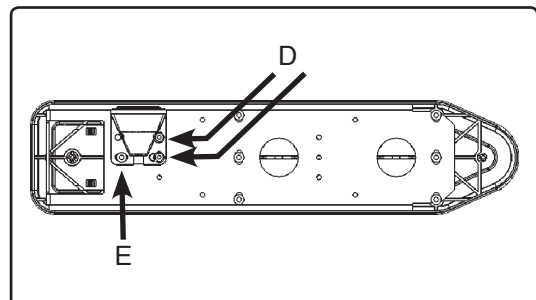
*The charger connector can be fitted in three different positions on the arm rest.**Connector in position A.**Fit the charger connector with the two enclosed M4x8 screws and washers (D) and the earlier removed M5 screw (E).*

**Assembly Instructions**

Document No. <b>239576</b>	Revision <b>B</b>				
Production state <b>SERIES</b>					
Date performed <b>2009-10-23</b>	Performed by <b>ETSN</b>	Date Approved <b>2011-01-04</b>	Approved By <b>GHT</b>	Page <b>2</b>	


**Position B**

(Only for arm rest with length 350 or 400 mm.)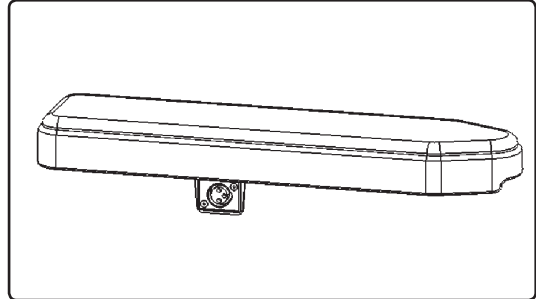


*Connector in position B.*

2. Remove the M5 screw (G) from the arm rest, see fig. below.
3. Fit the connector with the enclosed M4x8 screw and washer (F). See fig.

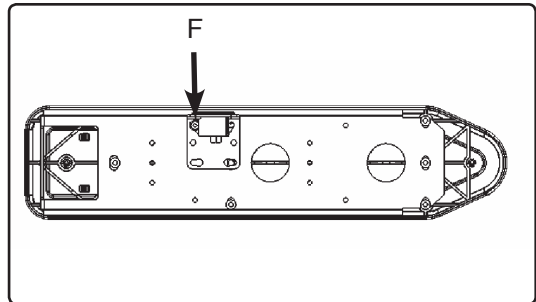*Fit the connector with the enclosed M4x8 screw and washer (F).*

4. Fit the cover on the contact and secure it with the earlier removed M5 screw (G). See fig.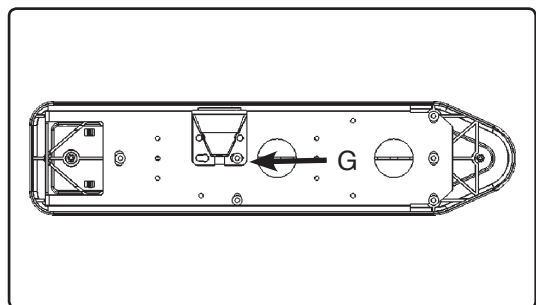


*Connector in position A.*

**Assembly Instructions**

Document No. <b>239576</b>	Revision <b>B</b>				
Production state <b>SERIES</b>					
Date performed <b>2009-10-23</b>	Performed by <b>ETSN</b>	Date Approved <b>2011-01-04</b>	Approved By <b>GHT</b>	Page <b>3</b>	


**Position C**

(For arm rest with length 250, 300, 350 or 400 mm.)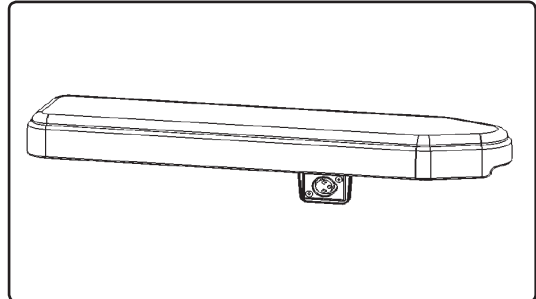

*Connector in position C.*

2. Fit the charger connector and its cover with two enclosed M4x8 screws and washers (H). See fig.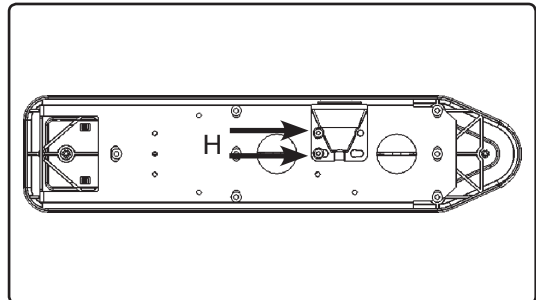


*Fit the connector with the enclosed M4x8 screws and washers (H).*

**Securing the cable**

Fit the cable around the armrest bracket with a cable tie. See fig.

Fit the rest of the cable together with the other cables on its way down to the chassis, using cable ties. Make sure the cables doesn't stretch or get cut while the seat lift or other seat function is used. Connection to the R-net output stage is described on page 4.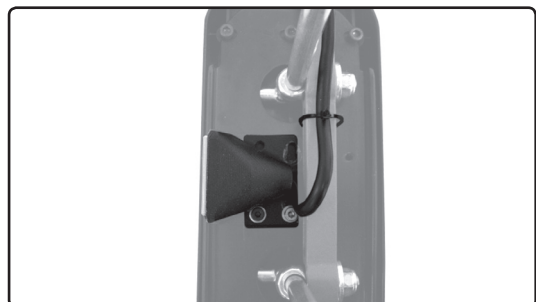


*Fit the cable with cable ties.*

**Assembly Instructions**

Document No. <b>239576</b>	Revision <b>B</b>				
Production state <b>SERIES</b>					
Date performed <b>2009-10-23</b>	Performed by <b>ETSN</b>	Date Approved <b>2011-01-04</b>	Approved By <b>GHT</b>	Page <b>4</b>	

**Assemble on UniTrack rail**


1. Use the two enclosed M3x6 screws to fit the connector on the desired bracket, angled or flat. See fig. on page one.
2. Insert the enclosed M6 nuts into the UniTrack rail.
3. Use the two enclosed M6x12 screws to fit the connector on the UniTrack rail, see figs.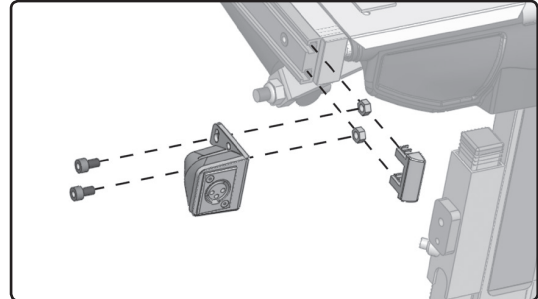

*Fitting of the charger connector on the UniTrack rail using the angled bracket.*

**Securing the cable**


Fit the cable together with the other cables on its way down to the chassis, using cable ties. Make sure the cables doesn't stretch or get cut while the seat lift or other seat function is used.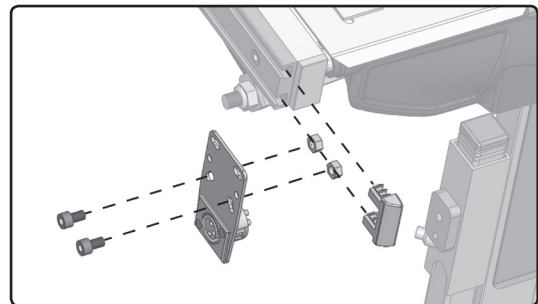

*Fitting of the charger connector on the UniTrack rail using the flat bracket.*

**Connection**

Connect the cable connection to the R-net Output Stage. See fig. below. To get more information about the R-net output stage, see the chassis service manual.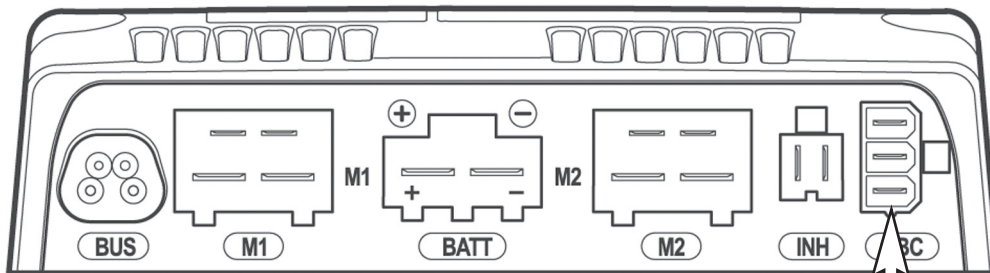


Connection of the charging connectors cabling.

*Connection of the charger connector to the R-net output stage.*

**Assembly Instruction**

Document No. <b>239576</b>	Revision <b>B</b>				
Production state <b>SERIES</b>					
Preform Date <b>2009-10-23</b>	Preform By <b>TSN</b>	Approved Date <b>2011-01-04</b>	Approved By <b>GHT</b>	Page <b>5</b>	

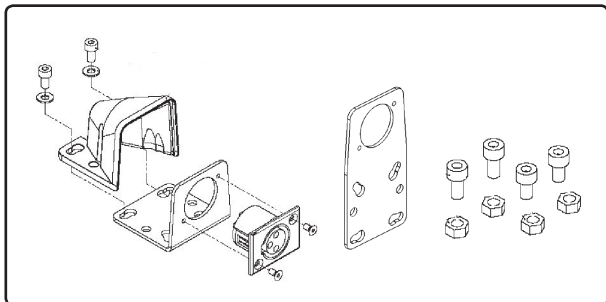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