


# Arm Troughs & Hand Pads

Assembly instruction

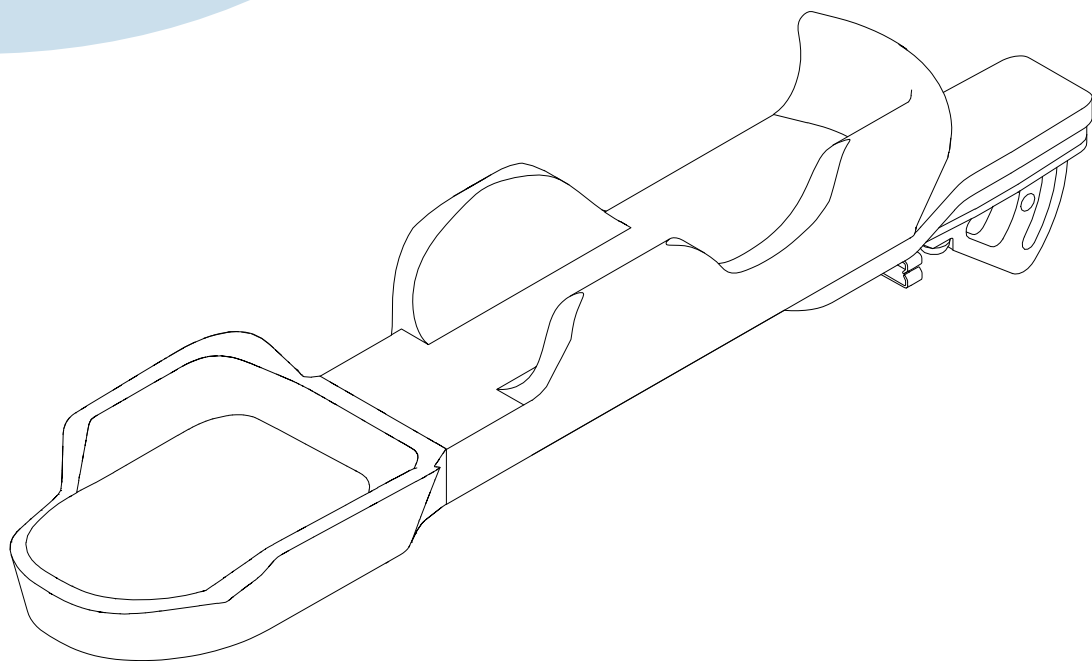

**permobil**

## Thank you for choosing to be part of the Permobil family!

This is the Assembly Instruction of your new accessory. An assembly instruction is not a stand-alone document, but rather a complement to the wheelchair's User's Manual. It can sometimes refer to the User's Manual or the Service Manual.

This Assembly Instruction contains important information, installation instructions, and adjustment instructions. The instructions on how to install the accessory is intended for the service technician only. All other parts of the Assembly Instruction are intended for both the service technician and the user of the wheelchair. In the section Safety information, you find definitions of all admonitions, and in the section Safety instructions you find instructions for admonitions used in this Assembly Instruction.

We hope that you are going to enjoy your new accessory. This accessory is intended to be used with a Permobil wheelchair. For more information regarding indoor and outdoor use, and for further information about your wheelchair, please see the wheelchair's User's Manual.

# Contents

Important information .....	4
Safety information .....	5
Preparation.....	6
General warnings for this product .....	7
Installation.....	8
Safety instructions.....	13
Contact us .....	15

## Important information

### Read this manual

Read this manual, including all admonitions and instructions, before you install, maintain, or use your accessory. Failure to follow these instructions and admonitions can lead to bodily injury or damage to the accessory or wheelchair.

### Visually impaired

This document can be downloaded at [www.permobil.com](http://www.permobil.com). If you are visually impaired, use the magnifying glass in your PDF reader to achieve desired text and picture size.

### Order or download a copy of the instructions

If you need another copy of this Assembly Instruction, you can order one from Permobil. Please state the order number specified on the back cover. You can also download a copy from our website: [www.permobil.com](http://www.permobil.com)

### Service and repairs

Please refer to the wheelchair's User's Manual for information about service and repairs.

### Reporting an incident

Please contact Permobil or your dealer if you want to report an incident. The accessory does not have a serial number. However, if the accessory is attached to the wheelchair, we will ask for the wheelchair's serial number. This serial number is located on the chassis.

### Warranty and expected service life

Contact your dealer or Permobil for information about the warranty period of this product. You can find the conditions of the warranty and further information about applicable warranties on our website.

This accessory has an expected service life of five years.

### Scrapping and recycling

Contact Permobil for information about scrapping and recycling agreements in force.

# Safety information

The following safety labels are used throughout this manual to draw attention to items of significant importance to safety:

## Labels



Warning

Signal word used to indicate a hazardous situation which, if not avoided, could result in personal injury or death.




Notice

Signal word used to indicate informative remarks.

# Preparation




Necessary tools

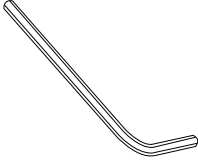

5 mm



Assembly kit

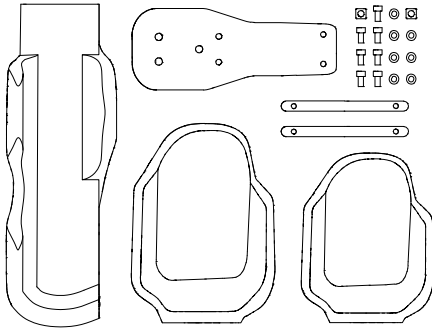

# General warnings for this product



## ① Warning

Do not use the hand pad to support the user or caregiver during transfers or repositioning.

MUL page 13



## ① Notice

Periodically check the arm trough for any damage or need for adjustment.

MUL page 13



## ② Notice

An installed hand pad/arm trough may be the widest point of the chair.

MUL page 13




## ③ Notice


The added weight of the arm trough can cause the armrest to drop more easily when lifted up.

MUL page 14

# Installation

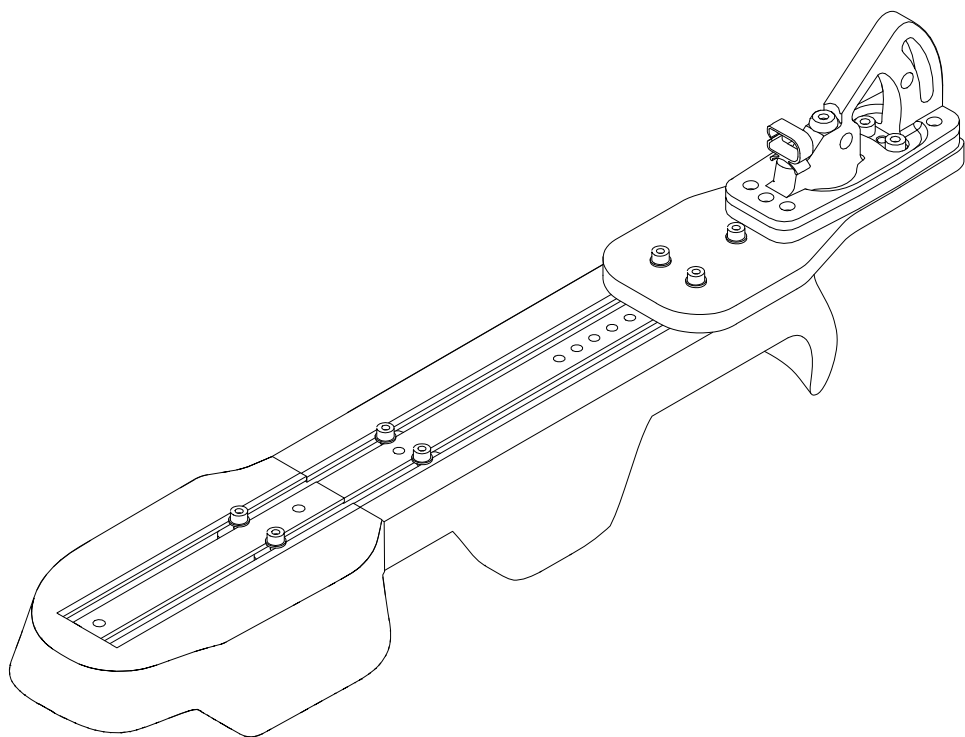


1.

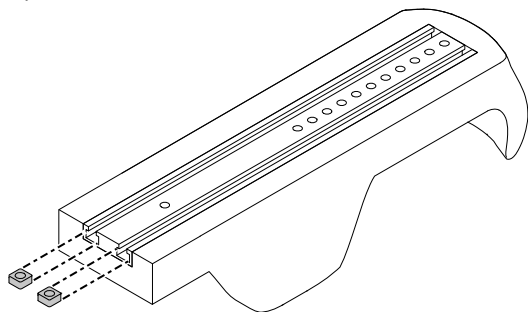




2.

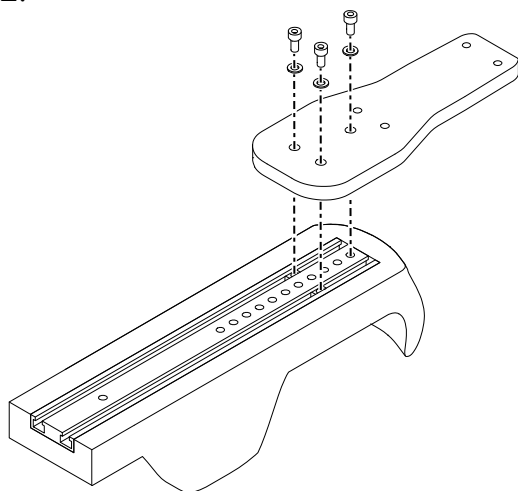


3.

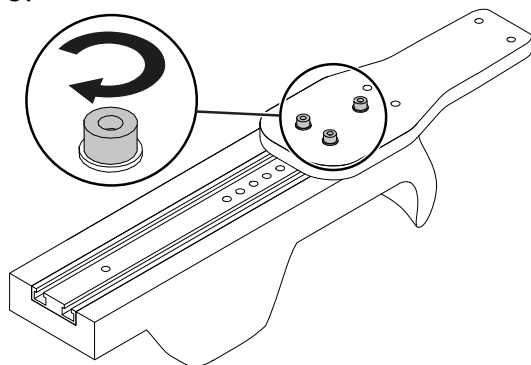


$T = 9.8 \text{ N m} / 7.2 \text{ lb. ft.}$


4.

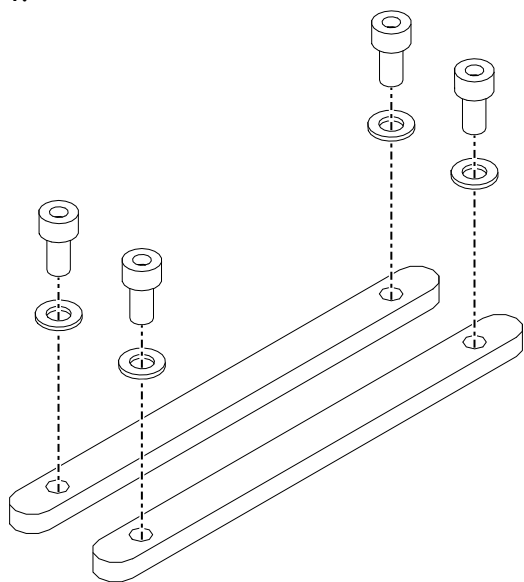

5.

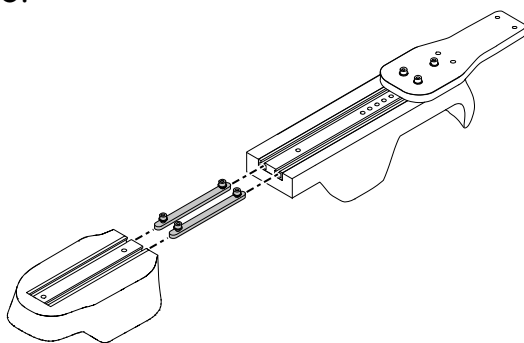


6.

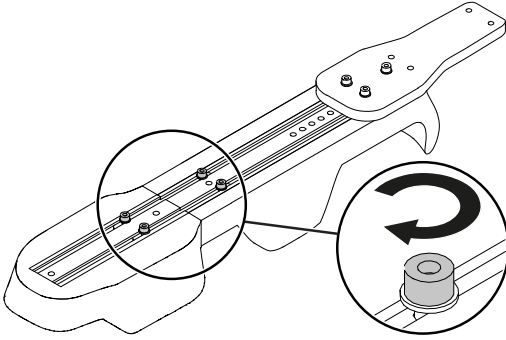


$T = 9.8 \text{ N m} / 7.2 \text{ lb. ft.}$


7.

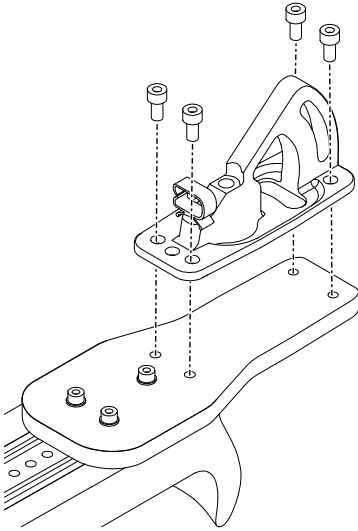


or

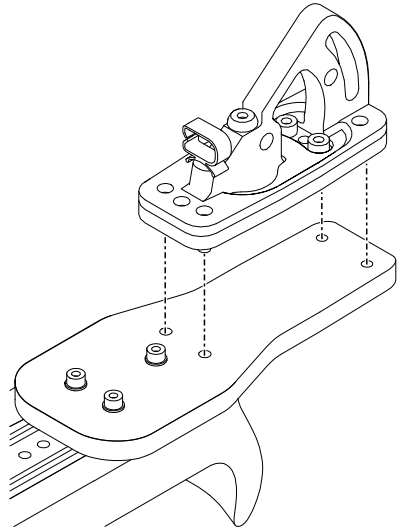

8.

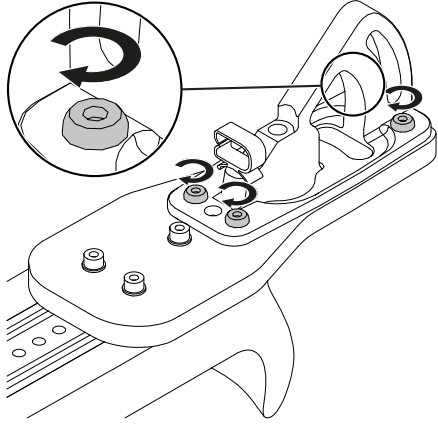

or

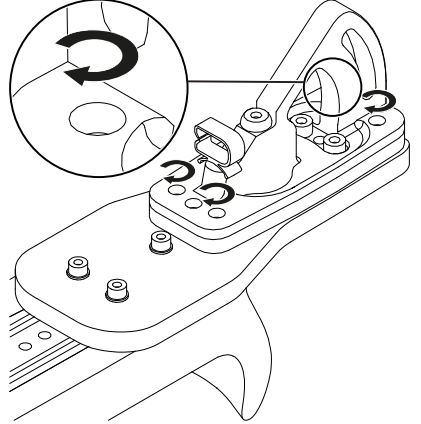

$T = 9.8 \text{ N m} / 7.2 \text{ lb. ft.}$

# Safety instructions

## Warning



### ① Warning

- US:** Do not use the hand pad to support the user or caregiver during transfers or repositioning.
- MX:** No utilice el apoyamanos para apoyar al usuario o cuidador durante las transferencias o el reposicionamiento.
- CA:** N'utilisez pas le coussin d'appui pour la main pour soutenir l'utilisateur ou le soignant pendant les transferts ou le repositionnement.
- US:** Não utilize o apoio de mão como apoio para o usuário ou o profissional de saúde durante as transferências ou reposicionamento.

## Notice



### ① Notice

- US:** Periodically check the arm trough for any damage or need for adjustment.
- MX:** Revise periódicamente el reposabrazos para ver si está dañado o si necesita algún tipo de ajuste.
- CA:** Vérifiez régulièrement que le coussin gouttière n'est pas endommagé ou qu'il n'a pas besoin d'être réglé.
- US:** Verifique periodicamente se há danos ou necessidade de ajuste na canaleta para o braço.



### ② Notice

- US:** An installed hand pad/arm trough may be the widest point of the chair.
- MX:** El punto más ancho de la silla puede ser un apoyamanos/reposabrazos.
- CA:** Un coussin d'appui pour la main ou un coussin gouttière installé sur le fauteuil peut constituer son point le plus large.
- US:** O ponto mais largo da cadeira pode ser um apoio de mão/canaleta para o braço instalados.

**③ Notice**

- US:** The added weight of the arm trough can cause the armrest to drop more easily when lifted up.
- MX:** El peso adicional del reposabrazos puede hacer que el descansabrazos se caiga más fácilmente cuando se levanta.

- CA:** Le poids supplémentaire du coussin gouttière peut faire tomber l'accoudoir plus facilement quand il est en position relevée.
- US:** O peso adicional da canaleta para o braço pode fazer com que o apoio do braço caia mais facilmente quando levantado.

## Contact us

Permobil AB  
Box 120  
861 23 Timrå  
Sweden

Tel.: +46 60 59 59 00  
Fax: +46 60 57 52 50  
E-mail: [info@permobil.com](mailto:info@permobil.com)  
Web: [www.permobil.com](http://www.permobil.com)

Permobil Inc.  
300 Duke Drive  
Lebanon, TN 37090  
USA

Tel.: 800.736.0925  
Fax: 800.231.3256  
E-mail: [TechSupport@permobil.com](mailto:TechSupport@permobil.com)  
Web: [www.permobilus.com](http://www.permobilus.com)

# permobil

Produced and published by Permobil  
Edition: 2  
Date: 2019-08-29  
Order no: 336235 mul

