

BROMS S3



Monteringsanvisning för:
Broms S3
gäller även med bromsspaksförlängare
art.nr. 4630000, 4630001, 4630002, 4630010
4630011, 4630012



Länk till video

Verktyg som behövs:

Insexnyckel 5 mm



1) Fästet på bromsen (inringat) monteras runt ramröret.



2) Bromsen på plats i en ungefärlig position. Justeras in sen.




3) Drag åt skruven lite lätt med insexnyckel 5 mm.



4) Lås bromsen och kontrollera att bromsbladet går in det välpumpade däcket ca 5 mm.



5) Vid behov så justera in bromsen i djupled så att bromsverkan blir tillräcklig

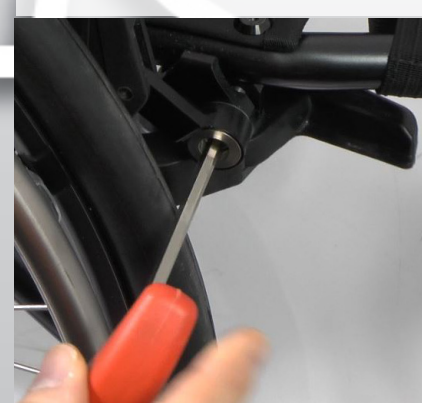

6) När bromsen är rätt injusterad så drag åt ordentligt med insexnyckel 5 mm. Upprepa proceduren 1- 6 för bromsen på andra sidan.

BRAKE S3



Assembly instruction for:
Brake S3
also valid for brake lever extender
art.no. 4630000, 4630001, 4630002, 4630010
4630011, 4630012



[Link to video](#)

Tools needed:

Allen key 5 mm



1) The attachment for the brake should be mounted around the chassis tube.



2) The brake placed in an approx. positions. Adjusted later.




3) Tighten the screw slightly with allen key 5 mm.



4) Engage the brake and check that the brake blade cuts into the pumped tire approx. 5 mm.



5) If needed adjust the brake in depth to get the correct brake force.

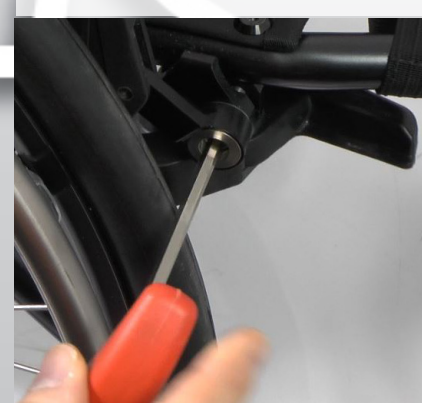

6) When the brake is properly adjusted tighten the screw firmly with allen key 5 mm. Repeat the procedure 1-6 on the other brake.