


Assembly Instruction

Document No. 2306622	Revision A.00				
Production state SERIES					
Preform Date 2007-09-05	Preform By UDM	Approved Date 2008-01-10	Approved By GHT	Page 1	

Mounting Rear-view mirror

Part No: 1821432 Rear-view mirror

List of parts that should be found in this set.

- Qty. 1 Rear-view mirror

Mounting rear-view mirror

Read through the instructions before starting installation

- Check that the stop screw is unscrewed and fit the rearview mirror to the accessory bracket. See Figures 1 and 2.
- Turn the rearview mirror to the desired position and tighten the stop screw.

NB! *The stop screw must face forwards with right-hand installation and backwards with left-hand installation.*



Fig. 1.
Holder for accessories.




Fig. 2.
Placing of stopscrew.



Fig. 3.

Assembly Instruction

Document No. 2306622	Revision A.00				
Production state SERIES					
Preform Date 2007-09-05	Preform By UDM	Approved Date 2008-01-10	Approved By GHT	Page 2	

Montering av backspegel

Best.nr: 1821432 Backspegel.

Förteckning över detaljer som skall finnas i satsen.

1. 1st Backspegel

Montering

Läs igenom anvisningen *innan* monteringen påbörjas.

1. Kontrollera att stoppskruven är utskruvad och montera backspegeln på tillbehörsfästet
Se bild 1 och 2.
2. Vrid backspegeln till önskat läge och skruva fast stoppskruven.

OBS! Stoppskruven ska vara framåt vid högermontage och bakåt vid vänstermontage.



Bild 1.
Tillbehörsfäste.



Bild 2.
Stoppskruvens placering.



Bild 3.