LÄNKHJUL S3

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CASTOR WHEEL S3

Assembly instruction for: Castor wheel S3

art.no. 2002010 Frontcaster S3 90 mm
art.no. 2002020 Frontcaster S3 120 mm
art.no. 2002030 Frontcaster S3 150 mm

QR link to video

Tools needed:

4 mm allen-key, 19 mm spanner, knife

1) To make it easier to change the castor wheel you can remove the castor fork. Remove the cap with e.g. a knife.


2) Loosen the nut with a 19 mm spanner and pull out the fork from the bearing house. If the bearing drops out, put it back into the house.

3) Remove the castor wheel, use a 4 mm allen key.


4) Remove hair, fuzz and other trash from the castor and washers if the same wheel will be reused.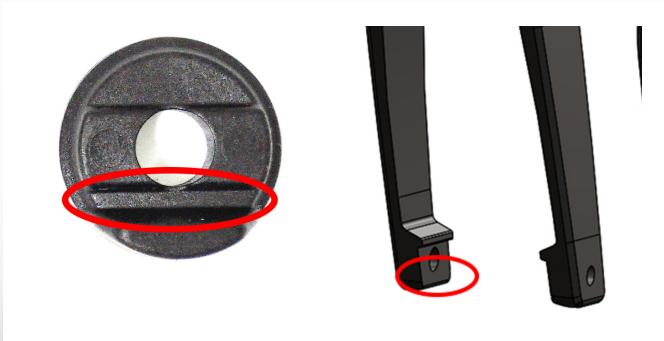

5) The new version of bushing and fork. The new bushing has only one edge and the fork is missing the lower edge. This change provides easier mounting, and the same bushing is used for both left and right side.

The new version of bushing fits the previous forks and the previous bushings fits the new fork.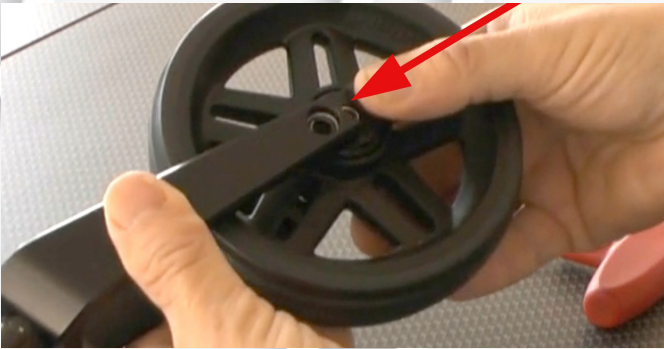

6) Place the bushings on the axle of the castor wheel on both sides and hold with your fingers on the edges of the bushing and enter the castor wheel into the fork from the underside until stop and when the holes line up with each other.

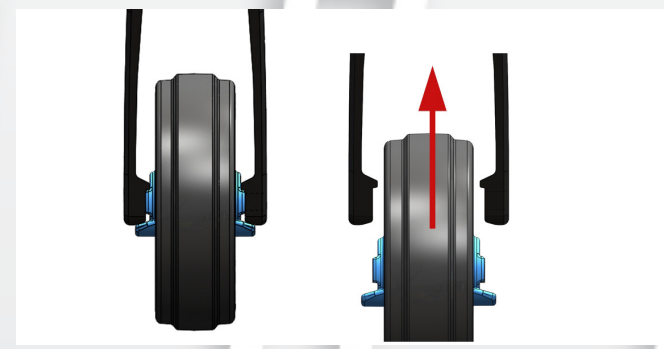
Illustration of how to enter the castorwheel onto the fork.


7) When the castor wheel is in place you can see that holes are lining up with each other.

8) Get the bolt and check that the threads has some paint left on it, which indicates sufficient thread locking.

NOTE! If the tread locking is missing, always order a new bolt from Panthera or apply a small amount of weak thread locking.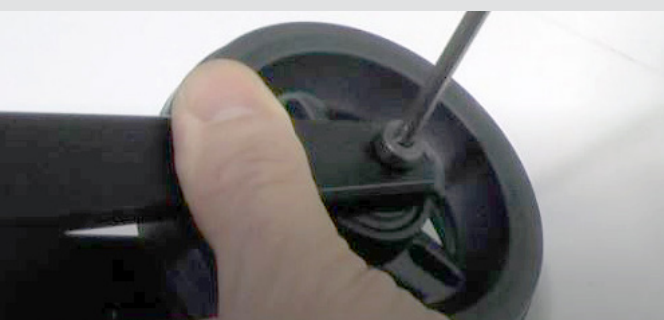


9) Place the bolt through the holes and tighten it into the threads located at one of the fork legs. Use allen key 4 mm.

5) Note that the bushings comes in a left and a right version. The surface is marked with a "L" for left and "R" for right side. Also note the little boss on the washers, that is a stopper against the slots on the castor fork.

When mounting the castor on the fork you can hold your fingers on those bosses to facilitate guiding them onto the slots on the castor fork.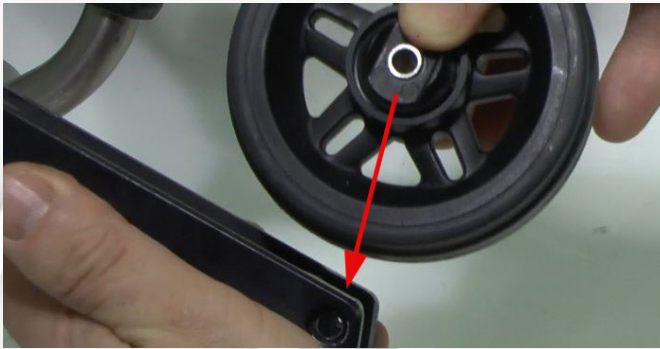

6) The bushings mounted on the castor. Make sure you know which side is the left and the right. Here the left side, in the direction of travel for the wheelchair, is facing the floor. With a good grip on the bosses with your thumb and the index finger slide the washers onto the slots on the castor fork.

Could be a little tricky.
See illustration below

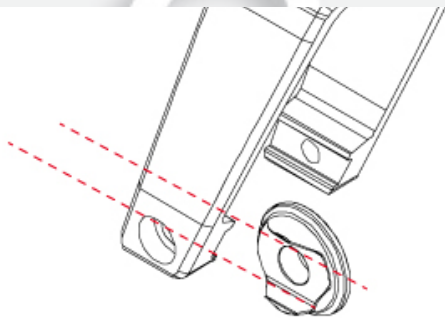
Illustration of how the washers is guided onto the slots on the castor fork.


7) When the castor wheel is in place you can see that holes are lining up with each other.

8) Get the bolt and check that the threads has some paint left on it, which indicates sufficient thread locking.

NOTE! If the tread locking is missing, always order a new bolt from Panthera or apply a small amount of weak thread locking.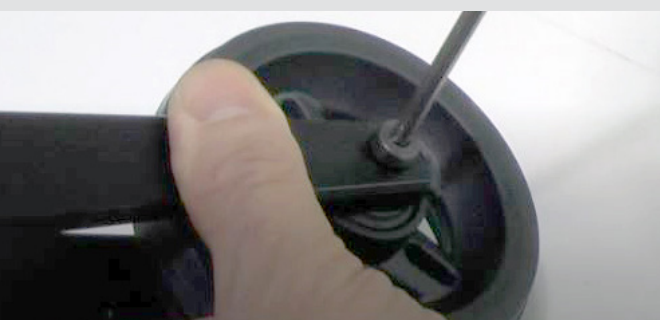

9) Place the bolt through the holes and tighten it into the threads located at one of the fork legs. Use allen key 4 mm.

10) Reassemble the fork on the wheelchair.

11) Place the nut in the plastic washer.

12) Put the nut on the threaded top of the castor fork. Tighten with a 19 mm spanner until there is no play between the fork and the chassis but it should also swivel easy.

13) Finally snap on the cap over the nut. Repeat the procedure on the castor wheel on the other side. Done!