Assembly Instruction

Document No. 232069	Revision A.01				
Production state Not Released					
Preform Date	Preform By	Approved Date	Approved By		Page 1

Adjustment of max speed X850

Part no: 314398-99-0 Adjustment of max speed X850

Adjustment of max speed 6km/h on X850

Read through the instructions *before* starting adjustment.

Unlocking Mini-Config

Mini-Config can be locked/unlocked by holding down the SEAT-DOWN button when starting up the chair using the ON button and holding it for one second. Two beeps mean 'Mini-Config locked' and three beeps mean 'Mini-Config unlocked'. **NB!** *Mini-Config must be unlocked.*

Starting Mini-Config

Hold the LIGHT and MAX SPEED buttons down, and start the chair using the ON button.

LED 4 (for side 1) and LED 1 (for Parameter 1) light up.

Changes to parameter values are made using the joystick. Moving the lever to the right or forwards increases the parameter value, and moving it to the left or backwards reduces it.

The current parameter values can be seen on the BATTERY scale. The number of LEDS lit up depends on the maximum value of the parameter.

To change the maximum speed to 6km/h, LED 1 must light up on the speed scale. Then press the left arrow (left indicators). See illustration.

Adjust the value using the joystick so that six segments of the battery scale light up.

Save the value in all three Mini-Config parameter sets by pressing the ON button.

LED 1, LED 2 and LED 3 flash.

Press the LIGHT button.

Press the ON button.

Switch the system off using the OFF button.

Start Mini-Config again.

Press the ON button. LED 1, LED 2 and LED 3 flash.

Press the LEFT INDICATORS button.

Press the ON button.

Switch the system off using the OFF button.

Start Mini-Config again.

Press the ON button.

LED 1, LED 2 and LED 3 flash.

Press the SEAT-UP button.


Press the ON button.

The setting has now been saved and the chair can be operated using the parameters just set.

Test-run the chair!

Warning!

All changes should be made by staff familiar with Mini-Config, as otherwise the chair may be set incorrectly, and could even cause personal injury.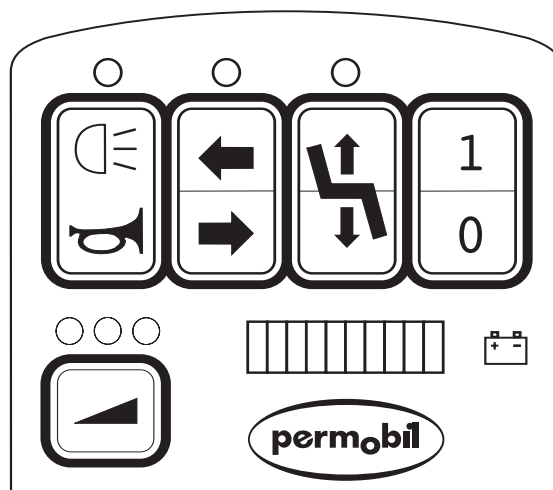

Assembly Instruction

Document No. 232069	Revision A.01				
Production state NOT RELEASED					
Preform Date	Preform By	Approved Date	Approved By		Page 2

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