Assembly Instruction

| | | | | | |
|-------------------------------------|--------------------------|------------------------------------|---------------------------|------------------|---|
| Document No. 230736 | Revision A.00 | | | |  |
| Production state Released | | | | | |
| Preform Date 2007-09-10 | Preform By UDM | Approved Date 2008-01-10 | Approved By GHT | Page 1 | |

Mounting of manual/electrical seat swivel

Part no: 1821476 Electrical seat swivel

Part no: 1821470 Manual seat swivel

List of parts that should be found in this set.

- | | | | | | |
|----|--------|----------------------------------|----|--------|----------------------------|
| 1. | Qty. 1 | Seat swivel Electrical or Manual | 3. | Qty. 4 | Washer ISO 7089 8,4x16x1,6 |
| 2. | Qty. 2 | Screw ISO 4762 M8x80 | 4. | Qty. 2 | Nut ISO 7040 M8 |

Electrical seat swivel

Read through the instructions before starting installation

Mounting of electrical seat swivel

- Place the wheelchair on a level surface. If the chassis has an electric seat lift, raise the seat to the highest position.
- Switch off the power supply using the ON/OFF key on the control panel and switch the automatic main fuse to OFF. It is accessible via a recess on the front right of the chassis.
- Remove the chassis covers.
- Divide the control panel's cabling at the contact on the cabling on the rear of the seat. Loosen any cable ties that hold the cabling in place.
- Remove the cabling from the quick release mountings on the seat lift.
- Remove the seat, which is held in place with two screws (see fig. 1).

**Note**

As the seat is heavy, this should be done by two people. Be careful with the cabling.

- Detach the seat bracket from the seat frame. It is held in place with four screws.
- Mount the seat swivel on the seat left using the accompanying screw kit. The seat swivel can be mounted at three different heights as preferred. See figs. 1 and 2.

**Note**

The seat swivel must not be mounted at an angle. For example, the front edge screwed in hole 1 and the rear edge in hole 2.

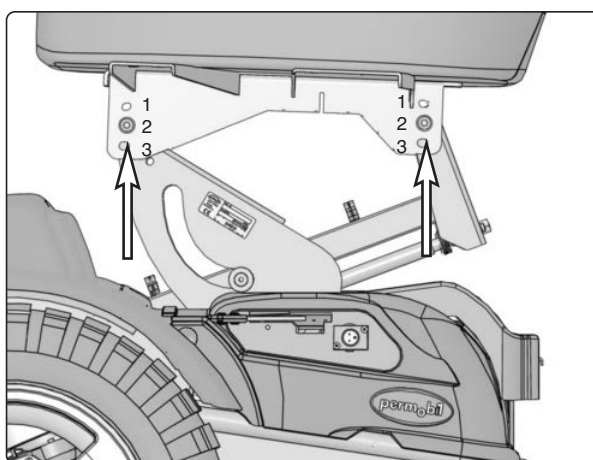
Fig. 1.

The seatbracket is mouted with two screws.

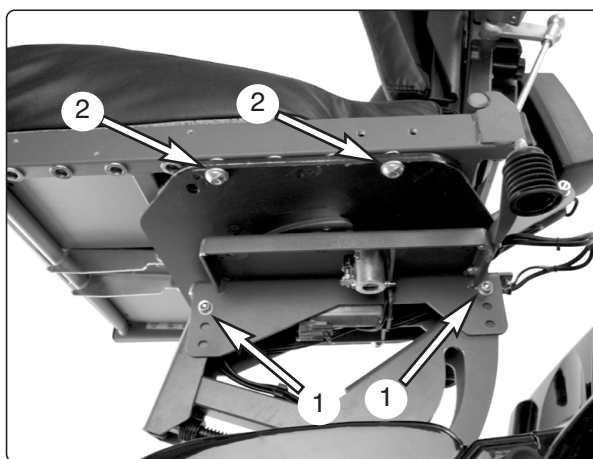
Fig. 2.

- 1. The seatswivel is mouted with two screws.*
- 2. The seat is mouted with four screws.*

Assembly Instruction

| | | | | | |
|-----------------------------------|--------------------------|------------------------------------|---------------------------|------------------|---|
| Document No. 230736 | Revision A.00 | | | |  |
| Production state Series | | | | | |
| Preform Date 2007-09-10 | Preform By UDM | Approved Date 2008-01-10 | Approved By GHT | Page 2 | |

9. Mount the seat in accordance with the relevant seat instructions.
10. Mount the seat cabling and seat swivel cabling in the quick-release mountings on the seat lift. Check that the cabling is long enough.
11. Mount the contact on the connection box. See fig. 3.
The remaining mounting takes place following points 1-4 in reverse order.

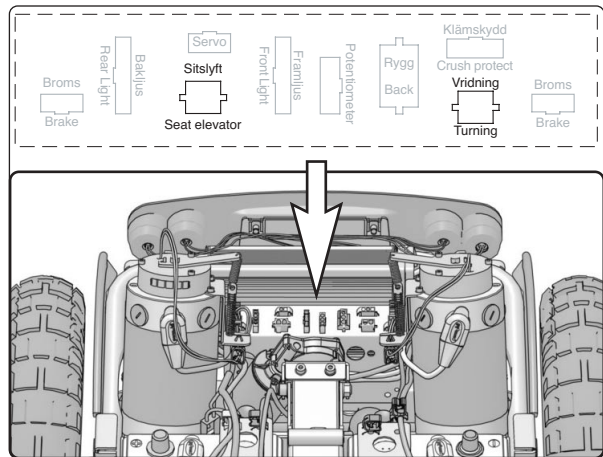
Fig. 3.

Seatelevator and seatswivel electrical connection.

Mounting manual seat swivel.

1. Place the wheelchair on a level surface. If the chassis has an electric seat lift, raise the seat to the highest position.
2. Switch off the power supply using the ON/OFF key on the control panel and switch the automatic main fuse to OFF. It is accessible via a recess on the front right of the chassis.
3. Divide the control panel's cabling at the contact on the cabling on the rear of the seat. Loosen any cable ties that hold the cabling in place.
4. Remove the cabling from the quick release mountings on the seat lift.
5. Remove the seat, which is held in place with two screws. See fig. 1.
6. Detach the seat bracket from the seat frame. It is held in place with four screws.
7. Mount the seat swivel on the seat left using the accompanying screw kit. The seat swivel can be mounted at three different heights as preferred. See figs. 1 and 2.
8. Mount the seat in accordance with the relevant seat instructions.

Fig. 4.

Channels for control wire.

**Note**

The seat swivel must not be mounted at an angle. For example, the front edge screwed in hole 1 and the rear edge in hole 2.

9. Mount the controls in the desired position on the panel holder. For example, as shown in fig. 5. Lock in position using the two stop screws.
10. The control lever can be mounted in two different positions on the controls:
 - A. Release by pushing the lever downwards.
 - B. Release by pulling the lever upwards.

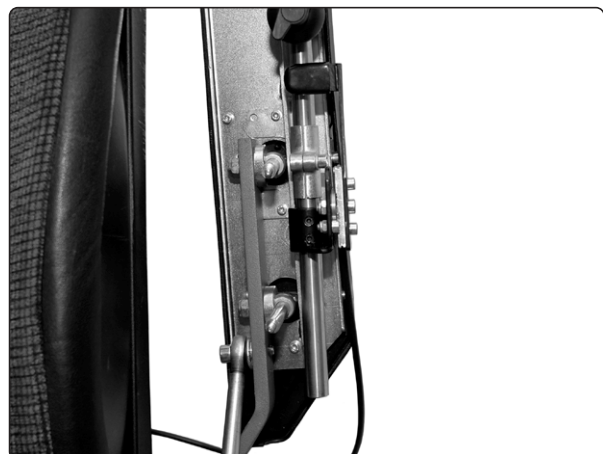
Fig.5.

Control lever mounting.

Assembly Instruction

| | | | | | |
|-------------------------------------|--------------------------|------------------------------------|---------------------------|------------------|---|
| Document No. 230736 | Revision A.00 | | | |  |
| Production state Released | | | | | |
| Preform Date 2007-09-10 | Preform By UDM | Approved Date 2008-01-10 | Approved By GHT | Page 3 | |

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