Assembly Instructions

Document No. 233065	Revision A				
Production state SERIES					
Preform Date 2008-02-18	Preform By BTA	Approved Date 2008-02-29	Approved By GHT	Page 1	

Panel holder

Order no.: 103473-99-0 335 mm long
Order no.: 103474-99-0 435 mm long

List of the parts in the set

- 1 Panel bracket
- 1 Panel holder
- 2 M5x16 screws
- 2 M4x8 screws

Fig. 1. The parts in the set.

Read the instructions carefully *before* starting installation

Attachment of the accessory

Unscrew the arm rest bracket screw. Figure 2. Insert the panel holder and set the required distance from the arm rest. Tighten the screw. The basic position for the panel bracket's height depends on the seat on which the panel holder is installed. Figures 3 - 5.

Fig. 2. The accessory attached to the arm rest bracket.

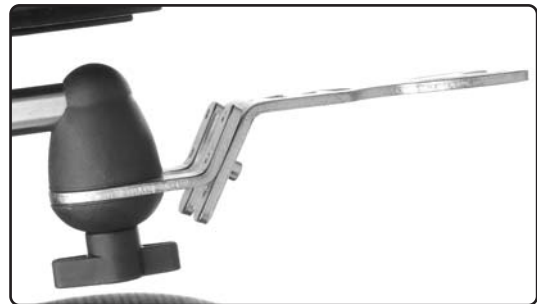
Fig. 3. Initial height for CII, KB and Vertical seats.

Fig. 4. Initial height for CS seat.

Fig. 5. Initial height for PS seat.

Assembly Instructions

Document No. 233065	Revision A				
Production state SERIES					
Preform Date 2008-02-18	Preform By BTA	Approved Date 2008-02-29	Approved By GHT	Page 2	

Adjustment of the accessory

Adjust the distance of the panel from the arm rest by unscrewing the arm rest bracket screw and pushing the panel holder until the panel is in the required position. Figure 2. Tighten the screw after adjustment.

Height adjustments from the initial heights (Figures three to five) can be achieved by attaching the panel bracket in one of the other screw holes. Figure 6.

The joystick can be fitted in a fixed screw hole that fixes it. In Figures seven and eight, the hole is visible at the top of the slide track. If the opposite screw is fitted in the slide track, the panel can be turned sideways. Figure 7.

Unscrew the panel holder's knob to adjust the panel in relation to the centre of the arm rest. Figure 8.

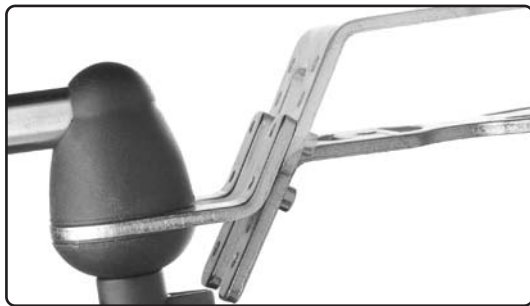
Fig. 6. Panel bracket height adjustment.

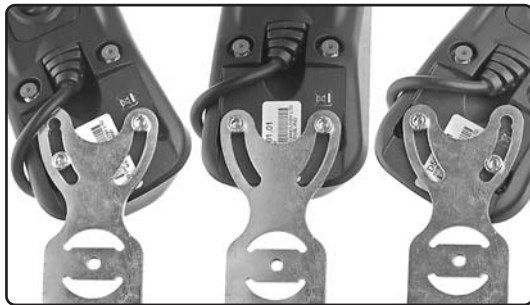
Fig. 7. The screw can be fitted fixed or in one of the slide tracks.

Fig. 8. Adjustment in relation to the centre of the arm rest. Unscrew the knob and turn to the desired position.

If you turn the bracket located on the panel holder, the panel can be fitted in other lower positions. Figure 10.

Disassemble the holder's swivel as shown in Figure nine.

When it is reassembled, tighten the nut so that the bracket turns slightly stiffly. Check that the knob locks the swivel. If it is not locked, the nut has to be tightened further.

Fig. 9. Disassembly to reverse the bracket.

Fig. 10. Panel holder's bracket fitted low.

Assembly Instructions

Document No. 233065	Revision A				
Production state SERIES					
Preform Date 2008-02-18	Preform By BTA	Approved Date 2008-02-29	Approved By GHT	Page 3	

Attaching the button box

Attach the button box between the bracket and the joystick. Figure 11. Unscrew the two M5x16 screws that hold the joystick. Insert the button box bracket between the joystick and the panel holder bracket. Refit the screws and tighten them.

Fig. 11. The button box is attached under the joystick.

Fig. 12. Button box and joystick in place.

Fixing the cables

Conceal the cables by attaching them on the inside of the arm rest where possible. Figures 13 and 14.

Gather the cables together and fix them first in the panel bracket (top of Figure 13). Unscrew the panel holder's knob and turn the panel out to the required outer position. Attach the cables to the panel holder's tube. Turn the panel back to the initial position. A loop (top of Figure 13) has now been created. Fix the cables along the entire tube. Fix the contact to the arm rest holder. Figure 13.

Lay the cables in a loop so that they do not get caught when the arm rest is folded up. Figure 14. The back reinforcement's protective cover has two cable fixing devices. Fix the cables in them.

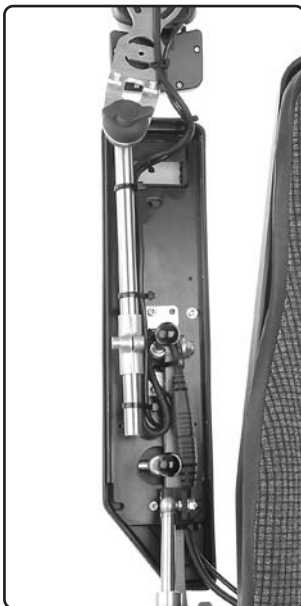
Fig. 13. Under the arm rest. The cables are concealed inside the arm rest. The contact is fixed to the arm rest holder. Check that the cables do not get caught when the arm rest is folded up.

Fig. 14. Fix the cable in the back joint's protective cover.

Assembly Instruction

Document No. 233065	Revision A				
Production state SERIES					
Preform Date 2008-02-18	Preform By BTA	Approved Date 2008-02-29	Approved By GHT	Page 4	

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