

**Assembly Instruction**

Document No. 270656	Revision A				<b>permobil</b>
Production state SERIES					
Preform Date 2015-09-07	Preform By ROHO	Approved Date 9/7/2015	Approved By GHT	Page US1	

## Assembly

## INFLATABLE LUMBAR SUPPORT


**1** The inflatable lumbar support is supposed to be placed in the backrests cushion.



**2** Take loose the cushion from the backrest.



**3** Unzip the zippers that's placed on the lower edge of the cushion.

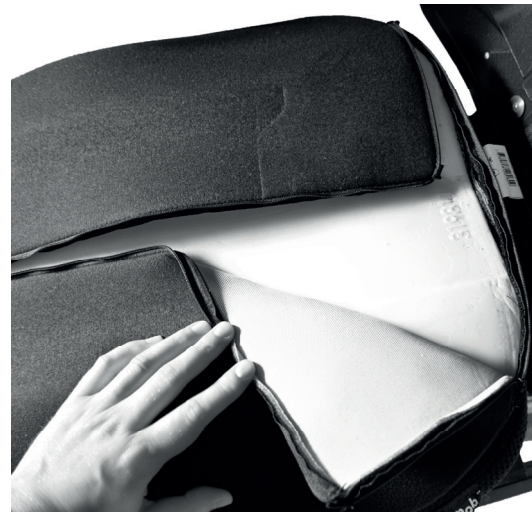

**Assembly Instruction**

Document No. 270656	Revision A				<b>permobil</b>
Production state SERIES					
Preform Date 2015-09-07	Preform By ROHO	Approved Date 9/7/2015	Approved By GHT	Page US2	

- 
- 4** Place the lumbar support under the upholstery.



- 
- 5** Pull back the zippers and put the cushion back on the backrests velcro.



- 
- 6** Attaching the pumps tubing can be necessary, but be observant so that the tubing doesn't get stuck and is pulled apart when maneuvering the chair and its functions. The pump has a discharge button to empty the support of air.



**Assembly Instruction**

Document No. 270656	Revision A				<b>permobil</b>
Production state SERIES					
Preform Date 2015-09-07	Preform By ROHO	Approved Date 9/7/2015	Approved By GHT	Page SE1	

## Montering

## UPPBLÅSBART SVANKSTÖD

- 1 Svankstödet ska sitta i ryggdynan.



- 2 Ta lös ryggdynan från ryggstödet kardborrband.



- 3 Dra upp de tre dragkedjorna i dynans underkant.



**Assembly Instruction**

Document No. 270656	Revision A				<b>permobil</b>
Production state SERIES					
Preform Date 2015-09-07	Preform By ROHO	Approved Date 9/7/2015	Approved By GHT	Page SE2	

---

**4** Placera in svankstödet under klädseln. Tänk på vilken sida av sitsen som du vill ha pumpen på.



---

**5** Dra igen dragkedjorna och sätt tillbaka ryggdynan.



---

**6** Slangen till pumpen kan behövas fästas för önskad position, men tänk på att fästa den så att den inte fastnar och dras sönder vid höjning och sänkning av stolen. Pumpen har även en utsläppsventil för att tömma svankstödet på luft.

