Assembly Instruction

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Assembly Instruction

Document No. 228683	Revision B				
Production state FOR SERIES					
Preform Date 2007-04-25	Preform By BTA	Approved Date 2010-05-05	Approved By GHT	Page 4	

Hip support*Order no.: 103069-99-0**Order no.: 103063-99-0**Order no.: 103072-99-0**Order no.: 103076-99-0**Order no.: 103079-99-0**Order no.: 103077-99-0**Order no.: 103081-99-0**Order no.: 103083-99-0***List of parts that should be in the set****Always included:**

- 2 pcs Brackets.
- 4 pcs Nuts, M6.
- 4 pcs Screws, M6x10.

Depending on model, the following will be included:

- 2 pcs Cushions, A.
- 2 pcs Covers for cushion A.
- 2 pcs Cushions, B.
- 2 pcs Covers for cushion B.
- 2 pcs Cushions, C.
- 2 pcs Covers for cushion C.
- 2 pcs Cushions, D.
- 2 pcs Covers for cushion D.

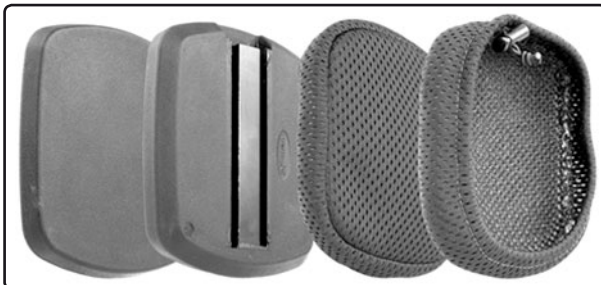
Read the instructions carefully *before* starting installation

Fig. 1. Cushion A. The set with order number 103069-99-0 contains cushions and brackets, Fig. 6. Order number 103079-99-0 also contains cushion covers.

Fig. 2. Cushion B. The set with order number 103063-99-0 contains cushions and brackets, Fig. 6. Order number 103077-99-0 also contains cushion covers.

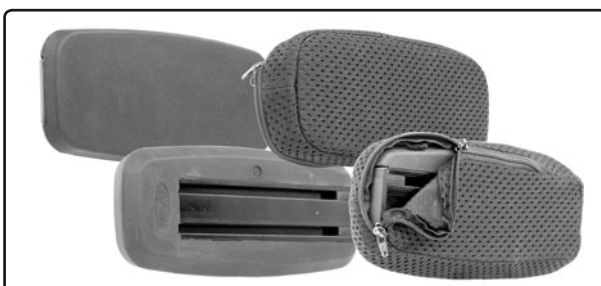
Fig. 3. Cushion C. The set with order number 103072-99-0 contains cushions and brackets, Fig. 6. Order number 103081-99-0 also contains cushion covers.

Assembly Instruction

Document No. 228683	Revision B				
Production state FOR SERIES					
Preform Date 2007-04-25	Preform By BTA	Approved Date 2010-05-05	Approved By GHT	Page 5	

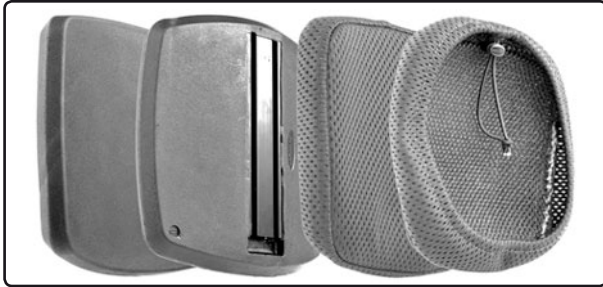
Fig. 4. Cushion D. The set with order number 103076-99-0 contains cushions and brackets, Fig. 6. Order number 103083-99-0 also contains cushion covers.

Assembly of cushion and hip support bracket

The assembly method is the same regardless of the cushion alternative selected.

The cushion can be identified by the letter of the alphabet moulded into it. Figure 5.

The hip support bracket is universal and used on all combinations. Figure 6.

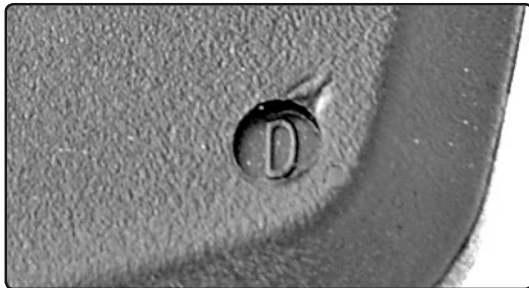
Fig. 5. The moulded identification letter.

Fig. 6. Brackets for hip supports.

Mount the two screws on the bracket for accessory attachment. Screw in the nuts only a few turns. See figure 6.

Insert the nuts in the two grooves on the rear of the cushion. See figure 7. Slide the bracket in towards the centre of the cushion. Adjust the cushion to a position which suits the user. Tighten the bracket screws.

Fig. 7. Assembly of cushion and bracket. Left, the bracket nuts are slid into the cushion grooves. Right, the cushion is fixed in the desired position.

Assembly Instruction

Document No. 228683	Revision B				
Production state FOR SERIES					
Preform Date 2007-04-25	Preform By BTA	Approved Date 2010-05-05	Approved By GHT	Page 6	

Attachment of accessory

Remove the plastic plug from the front edge of the accessory rail, see Fig. 9. Undo the screws on the bracket holder, Fig. 8, so that the nuts can slide easily in the accessory rail grooves. Replace the plastic plug. Slide the holder in the direction of the back rest. Tighten the holder screws. This is a starting position and may need to be re-adjusted when adapting the hip support to the user. Undo the screw lever on the bracket holder. Set the hip support in place in the holder. Tighten the lever. Figure 10.

Fig. 8. Hip support holder.

Fig. 9. The bracket holder mounted on the accessory rail.

Adjustment of accessory

The hip support is adjusted with the two articulated joints and by moving the cushion horizontally, Figs. 7 and 11. The height position is fixed with the stop screw on the bracket. The stop screw must be in contact with the edge of the holder, Fig. 10.

The distance between the hip supports is set by the two articulated joints, Fig. 11.

The distance to the back rest is adjusted with the screws holding the cushion, Fig. 7, and by moving the bracket in the accessory rail, Fig. 9. If necessary, adjust the height position by undoing the stop screw and moving it to a suitable hole. Tighten the stop screw. Fig. 10.

Loosen the screws in both the articulated joints and set to the desired position. Fix the position by tightening the screws. Loosen the screws holding the cushion in place, and adjust the distance to the back rest. Fix the position by tightening the screws. Figure 11. If the cushion's adjustment range is insufficient, alter the position of the bracket holder on the accessory rail.

Fig. 10. Screw holes and stop screw for fixing the support's height position.

Fig. 11. Adjusting the hip support to the user's requirements.