**Assembly Instruction**

Document No. 246355	Revision A.00				
Production state					
Preform Date 2010-12-21	Preform By UDN	Approved Date	Approved By		Page 1

## Mounting the Co-Pilot bracket on Corpus 3G

Order no.: 1825810

Co-pilot bracket C3G

### List of parts in the set.

1 pcs. Co-pilot bracket  
4 pcs. screws ISO 4762 M6x12 Fe/Zn

2 pcs. groove nuts 2xM5 CC25

### Mounting

Read the instructions carefully *before* starting installation.

1. Undo the stop screws holding the cover on the back rest profile and remove the cover by sliding it upwards along the groove. (Fig. 1)
2. Undo the screws on the bracket so that it is possible to slide the groove nuts down into the two central grooves on the back rest profile. (Fig. 2)
3. Tighten the screws so that the groove nuts are just under the top edge of the back rest profile. **NB!** The bracket can be mounted lower, but then the arm rest height needs to be adjusted.
4. Replace the cover on the back rest profile, and check that the cover does not catch on any part of the mounted bracket. Fix the cover in place by tightening the stop screws.

Fig. 1. Remove the cover.

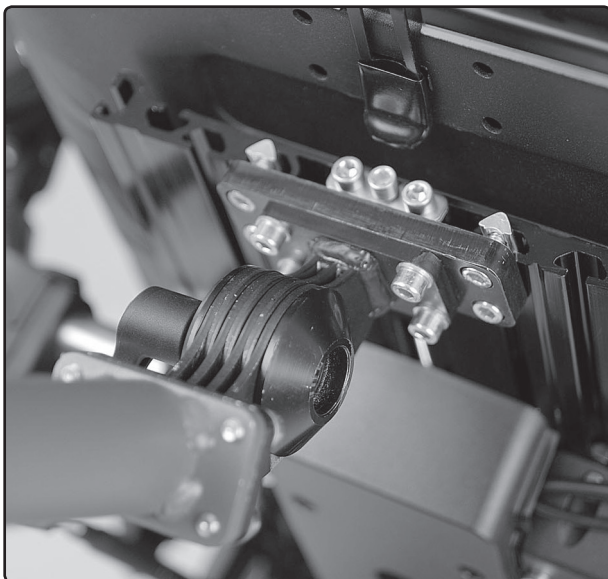
Fig. 2. Slide the bracket's groove track nuts in.

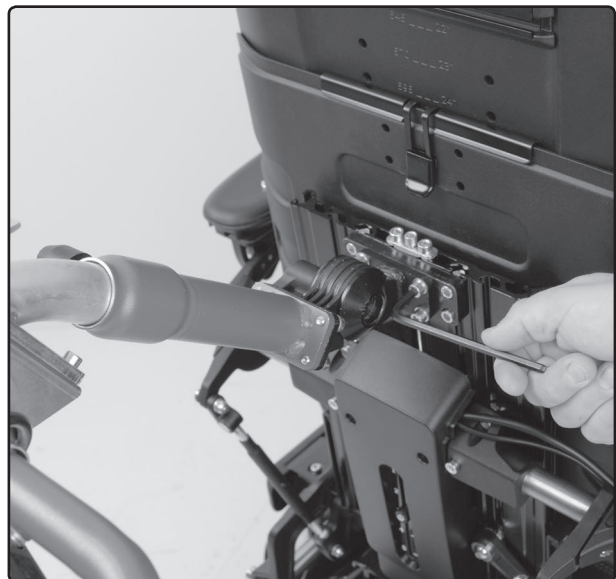
Fig. 3. The bracket mounted.

**Assembly Instruction**

Document No. 246355	Revision A.00				
Production state					
Preform Date 2010-12-21	Preform By UDN	Approved Date	Approved By	Page 2	

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