**Assembly Instruction**

Document No. 248595	Revision A				
Production state SERIES					
Preform Date 2011-10-05	Preform By UDN	Approved Date 12/8/2011	Approved By GHT	Page 1	

## Mounting Bag hooks set to a UniTrack rail

### List of parts in the set.

2 pcs. Holders for bag hook	2 pcs. Bag hooks
4 pcs. Washers 5.3x15x1,2	4 pcs. Washers 5.3x10x1
4 pcs. Lock nuts M5	4 pcs. Washers 6.4x12x1.6
4 pcs. Screws ISO 7380 M6x10 Fe/Zn	4 pcs. Screws ISO 7380 M5x16 Fe/Zn
4 pcs. Groove nuts M6	

### Mounting

Read the instructions carefully *before* starting installation.

1. Fit screw ISO 7380 M6x12 with washer 6.4x12x1.6 and groove nut in the corner hole on the holder. Thread the screw 2-3 turns into the nut. (*Fig. 1*)
2. Position the hook in the hole with the opening of the hook turned upwards. (*Fig. 1*)
3. Slide the groove nuts into the groove of the rail to the required position and tighten the screws. (*Figs. 1 and 2*)

**NB:** The positioning of the hooks depends on the type of bag that is to be used. Always test the gap using the bag itself.

Not all the securing elements in the set will be used for this type of mounting.

*Fig. 1.*  
*Slide in the groove nuts.*

*Fig. 3.*  
*Bag hooks mounted.*

**Assembly Instruction**

Document No. 248595	Revision A				
Production state SERIES					
Preform Date 2011-10-05	Preform By UDN	Approved Date 12/8/2011	Approved By GHT	Page 2	

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*Bild 3.*  
*Väskkrokarna monterade.*

**Assembly Instruction**

Document No. 248595	Revision A				
Production state SERIES					
Preform Date 2011-10-05	Preform By UDN	Approved Date 12/8/2011	Approved By GHT	Page 3	

## Mounting of Bag Hook Set on a plastic/metal backrest

### Mounting

Read the instructions carefully before starting installation.

1. Remove the back cushion.
2. Mark out and drill four 5.5mm holes (approx. 50mm from the top edge of the backrest) where the hooks are to be positioned. (Fig. 3)

The hooks should be positioned symmetrically on the intended central line on the backrest to avoid an uneven load.

**NB:** The positioning of the hooks depends on the type of bag that is to be used. Always test the gap between the hooks using the bag itself.

3. Fit the larger washers on the screws (M5x16) and thread the screws in from the front of the backrest. (Fig. 5)

Position the bag hook so the opening of the hook is facing upwards. Mount the holders and position the smaller washers between the holders and the nuts. Make sure the hooks are in the groove of the holders and tighten. (Fig. 4)

4. Replace the back cushion.

Not all the securing elements in the set will be used for this type of mounting.

Fig. 3.  
Bag hooks fitted on Corpus II.

Fig. 4.  
Bag hook fitted on Corpus II.

Fig. 4.  
Screw fitted on the front of Corpus II backrest..

**Assembly Instruction**

Document No. 248595	Revision A				
Production state SERIES					
Preform Date 2011-10-05	Preform By UDN	Approved Date 12/8/2011	Approved By GHT	Page 4	

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