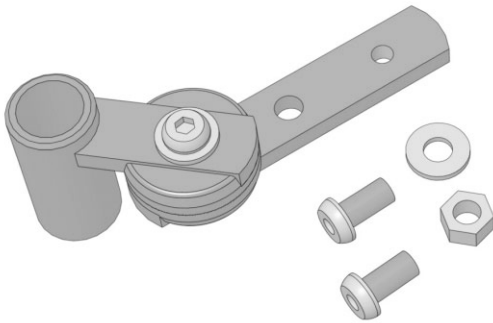
Assembly Instruction


Document No. 236314	Revision A				
Production state SERIES					
Preform Date 2008-12-10	Preform By TSN	Approved Date 2009-03-05	Approved By GHT	Page 1/6	

Assembly kit for table*Order no.: 104149-99-0***List of parts that should be in the set**


- 1 Table bracket
- 2 Socket head cap screws, M6x12 Fe/Zn
- 1 Hexagon nut, M6 8 Fe/Zn
- 1 Washer, 6.4x1.5 Fe/Zn

Read the instructions carefully *before* starting assembly**Fitting to PS Junior**

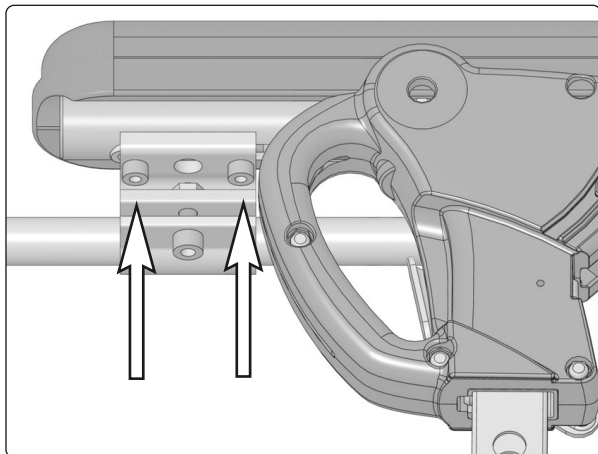1. When fitting to the same side as the control panel, loosen the two screws that hold the control panel bracket in place, move the bracket as far back as possible and tighten the screws again. See image below.
2. Remove the plastic plug fitted to the front of the armrest tubing. See image below.

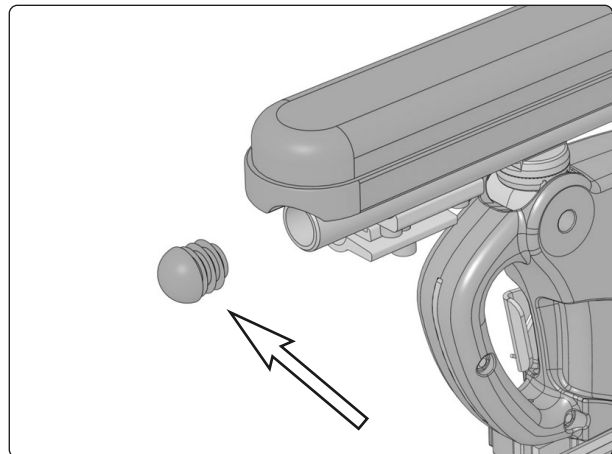The control panel bracket is attached with two screws.


The armrest tubing's plastic plug.

Assembly Instruction

Document No. 236314	Revision A				
Production state SERIES					
Preform Date 2008-12-10	Preform By TSN	Approved Date 2009-03-05	Approved By GHT	Page 2/6	


Fitting to PS Junior *cont.*

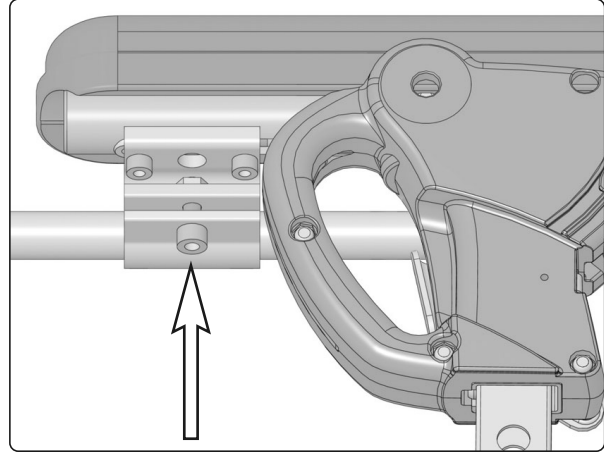- When fitting to the same side as the control panel, loosen the screw securing the panel bracket's length adjustment and move the control panel forward temporarily.


Screw for securing the panel bracket's length adjustment.

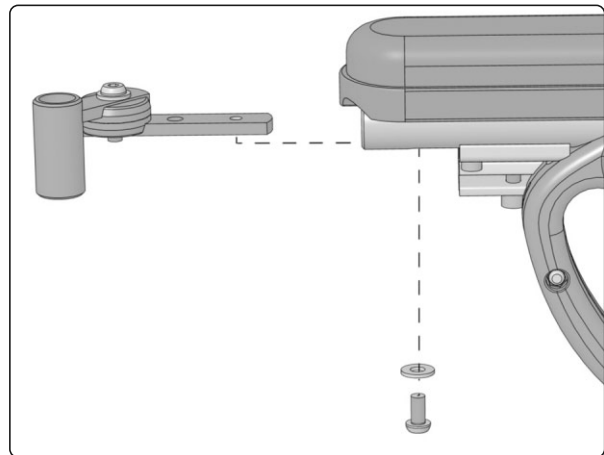- Fit the table bracket using the accompanying screw and washer. See image.

Fitting the table bracket.

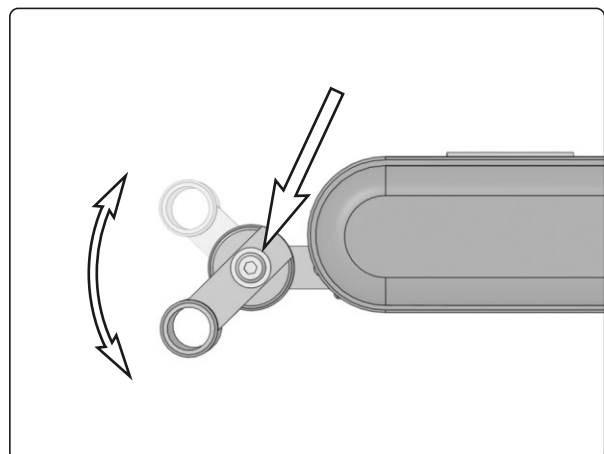- Adjust the table bracket angle to the table width. This is done by loosening the screw and turning the bracket to the required angle. Retighten the screw once the adjustment has been made. See image.
- When fitting to the same side as the control panel, push the control panel back to the required position and secure the panel bracket's length adjustment by tightening the screw. See image above.

Adjusting the table bracket angle.



Assembly Instruction

Document No. 236314	Revision A				
Production state SERIES					
Preform Date 2008-12-10	Preform By TSN	Approved Date 2009-03-05	Approved By GHT	Page 3/6	

Fitting to armrest with Accessory rail UniTrack

When fitting the bracket on arm rest with accessory rail UniTrack, fit the bracket with screws, washer and nut according to the figure to the right.

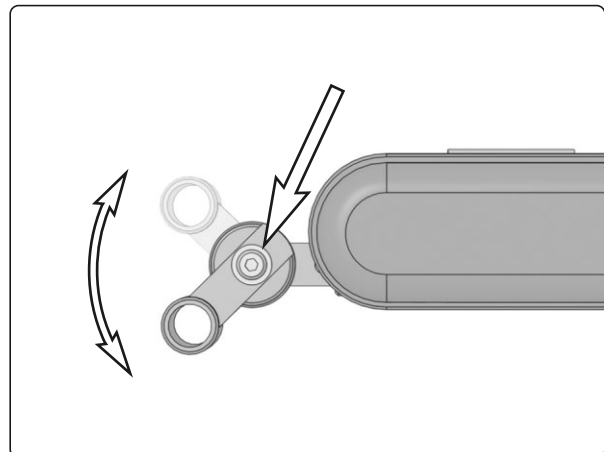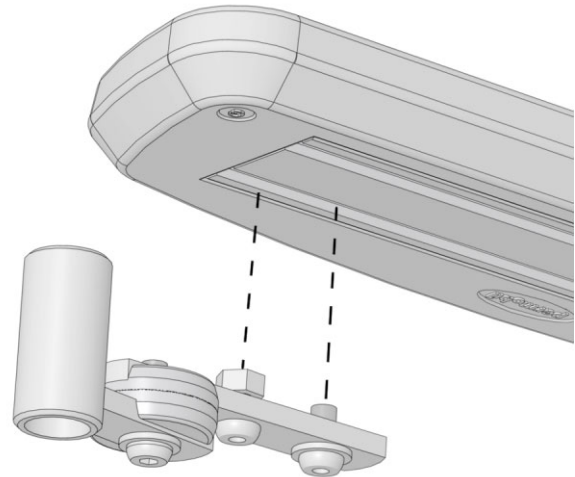1. Fit the screws, washer and nut on the bracket.
2. Insert the nut to the desired position in the rail. If needed, dismount the control panel bracket and arm rest bracket to get access to the rail.
3. Tighten the screws to lock the bracket in position. See fig.
4. If dismantled, remount the control panel bracket and arm rest bracket.
5. Adjust the table bracket angle to the table width. This is done by loosening the screw and turning the bracket to the required angle. Retighten the screw once the adjustment has been made. See fig.

Adjusting the table bracket angle.


Assembly Instruction

Document No. 236314	Revision A				
Production state SERIES					
Preform Date 2008-12-10	Preform By TSN	Approved Date 2009-03-05	Approved By GHT	Page 4/6	

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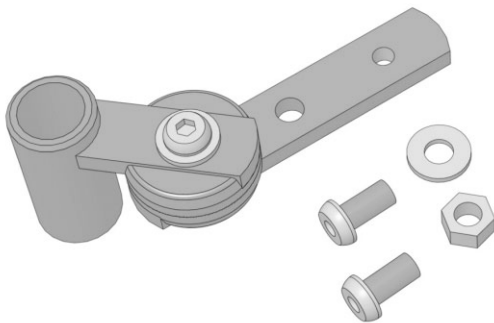
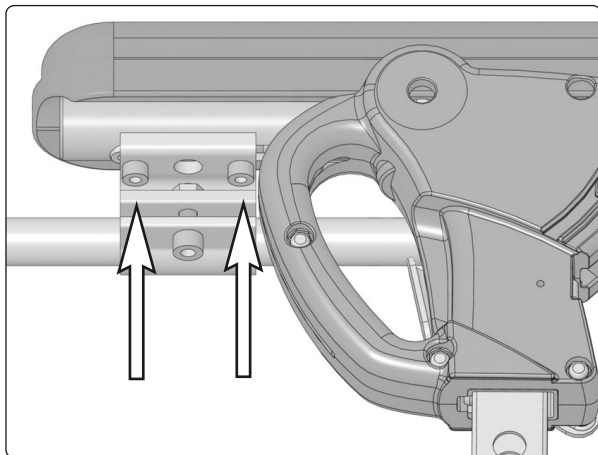
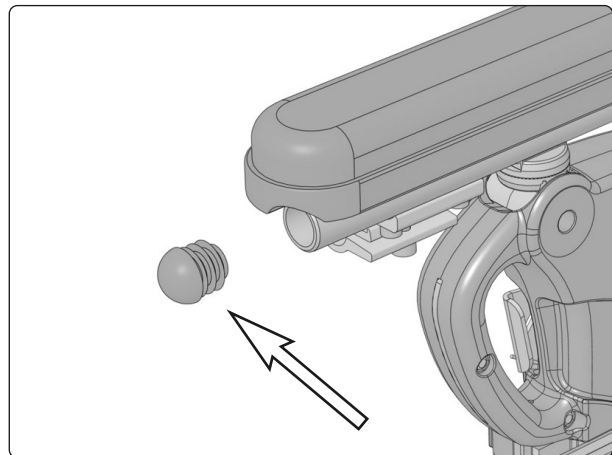
Armstödsrörets plastplugg.

Assembly Instruction

Document No. 236314	Revision A				
Production state SERIES					
Preform Date 2008-12-10	Preform By TSN	Approved Date 2009-03-05	Approved By GHT	Page 5/6	

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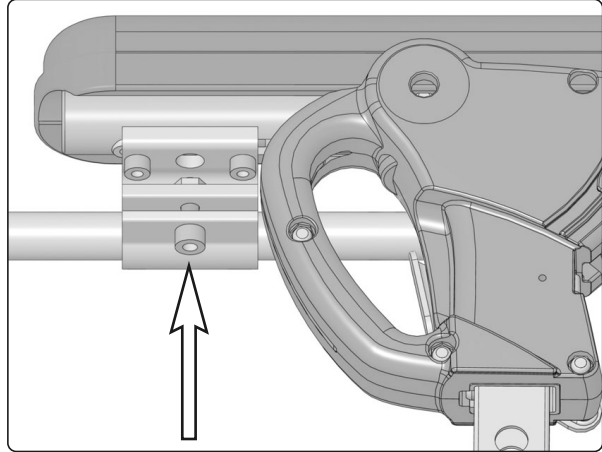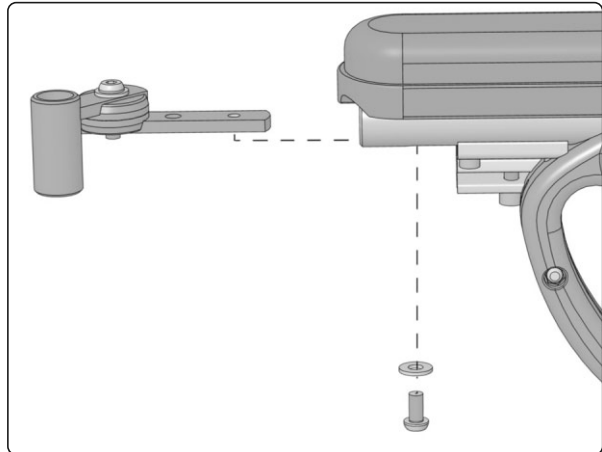
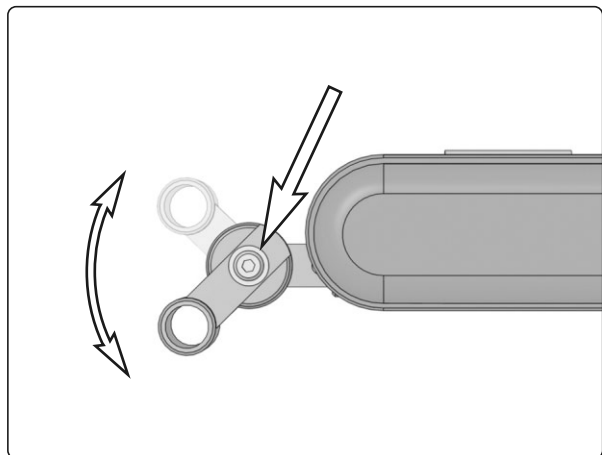
Justering av bordsfästets vinkel.

Assembly Instruction

Document No. 236314	Revision A				
Production state SERIES					
Preform Date 2008-12-10	Preform By TSN	Approved Date 2009-03-05	Approved By GHT	Page 6/6	

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