Document No. 268488	Revision A				permobil
Production state FOR SERIES					
Preform Date 2015-04-10	Preform By ROHO	Approved Date 4/13/2015	Approved By GHT	Page US1	

Replacing safety grip to the attendant control

Description for how to replace the safety grip on the attendant control.

Remove the safety lever

Figure 1. Press your fingers underneath the safety grip and pull to get the rod loose from the holder in the rubber handle.

Figure 2 and 3. Choose either side, grab the safety grip and forcefully pull it back and then out from the handle to get it loose. The opposite end of the safety grip will now also be loose.

Document No. 268488	Revision A				permobil
Production state FOR SERIES					
Preform Date 2015-04-10	Preform By ROHO	Approved Date 4/13/2015	Approved By GHT	Page US2	

Install the new safety grip

Figure 4. With the ends in place, push back the safety grip in the holder.

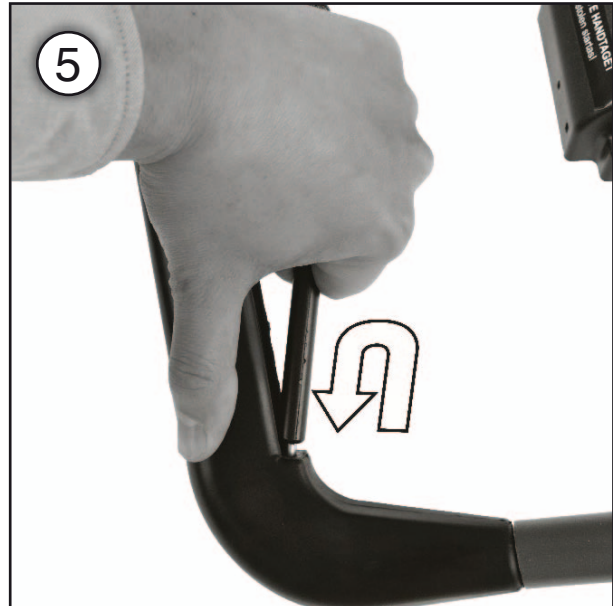
Figure 5. Put one of the ends into one of the cavities in the rubber handle.

Figure 6. The opposite end of the safety grip is aligned with force into the other cavity. Push it back and then forward to align it.

Document No. 268488	Revision A				permobil
Production state FOR SERIES					
Preform Date 2015-04-10	Preform By ROHO	Approved Date 4/13/2015	Approved By GHT	Page 3	

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