


Assembly Instruction

Document No. 248585	Revision				
Production state Not Released					
Preform Date 2011-10-04	Preform By UDN	Approved Date	Approved By	Page 1/2	

Mounting of alternative joystick grip

List of parts in the set

Qty.1 Ex. Alternative joystick grip.



Read the instructions carefully before starting installation

Removing the standard grip

1. Loosen the standard grip by pulling it straight upwards, see fig.



Standard grip mounted.



Standard grip removed.


Mounting the alternative grip

2. Push the alternative grip on to the joystick.
Make sure the grip is firmly fitted.



Alternative grip mounted.

Assembly Instruction

Document No. 248585	Revision				
Production state Not Released					
Preform Date 2011-10-04	Preform By UDN	Approved Date	Approved By		Page 2/2

Montering av alternativt styrspaksgrepp

Förteckning över de detaljer som ingår i satsen

1 st. Ex. alternativt styrspaksgrepp.



Läs noga igenom anvisningen *innan* montering påbörjas

Demontering av standard-grepp

1. Lossa standard-greppet genom att dra denna rakt uppåt, se bild.



Standard-greppet monterat .



Standard-greppet demonterat .

Montering av alternativt grepp.

2. Tryck fast det alternativa greppet på styrspaken. Försäkra dig om att greppet sitter stadigt monterat.



Montera alternativt grepp.