

BROMSSPAKSFÖRLÄNGARE FÖR BROMS S3



Monteringsanvisning för:
Bromsspaksförlängare S3
art.nr. 4630134, 4630133

Verktyg som behövs:
Skruvmejsel, Rundfil, Lim (epoxy DP490, behöver härda i 24h innan användning), Bommelstopp



1) Ta fram Broms S3 (i detta exempel höger broms) och avlägsna handtagsgummit med en skruvmejsel.



2) Bromshandtagsgummit borttaget.



3) Slipa två små spår, ca 0,5 mm djupa, på varje sida med hjälp av en rundfil. Detta för att limmet ska fästa ordentligt.



4) Applicera rikligt med lim (epoxylim) så att det fyller ut håligheter i bromsspaksförlängaren.




5) Limmet färdigapplicerat.



6) Tryck på bromsspaksförlängaren på bromsen.

panthera®

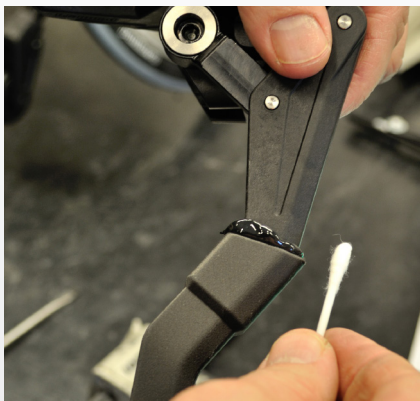

7) Avlägsna överflödigt lim med t.ex. bommulstops.



8) Färdigmonterad bromsspaksförlängare.

panthera



EXTENDED BRAKE LEVER FOR BRAKE S3



Assembly instruction for:
Extended brake lever S3
art.nr. 4630134, 4630133

Tools needed:

Screw driver, Round file, Glue (epoxy DP490, needs to harden for 4h before use), Cotton sticks



1) Bring out Brake S3 (in this case the right side) and remove the lever rubber with a screwdriver.



2) The rubber removed.




3) Use the round file to create two narrow slots approx. 0,5 mm deep, on each side. This is to make the glue to stick properly.



4) Apply plenty of glue (epoxy glue) until it fills out the holes in the brake lever extender.




5) The glue is applied.

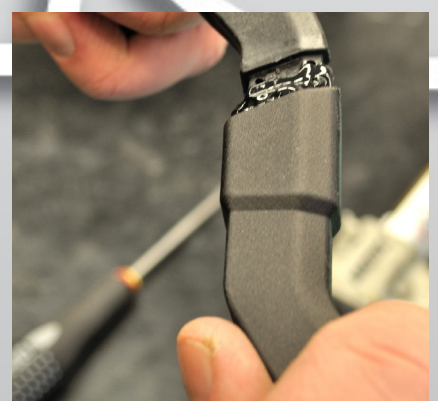

6) Push the brake lever extender onto the brake.

panthera®

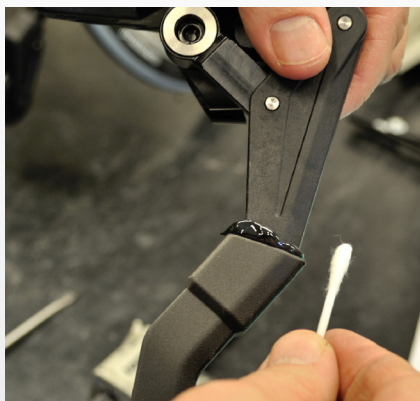

7) Remove excess glue with ie. cotton stick..



8) Mounted brake lever extender.

panthera