


Assembly Instruction

Document No. 246554	Revision A.00				
Production state					
Preform Date 2011-01-12	Preform By UDN	Approved Date	Approved By	Page 1	

Mounting of panel holder

Order no.: **1825815**

Panel holder

List of parts in the set.

3 pcs. M6 nuts

1 pcs. screw ISO 4762 M6x20

1 pcs. panel holder

2 pcs. screws ISO 4762 M6x12

3 pcs. washers ISO 7089 6.4x12x1.6

Mounting

Read the instructions carefully *before* starting installation.

1. Mount the panel holder on the arm rest by sliding the groove nuts into the required groove on the underside of the arm rest. (*Fig. 1*).

The choice of in which groove and direction the panel holder should be fitted depends on the use and required accessory location.

2. Adjust the panel holder to the required position and tighten the screws.

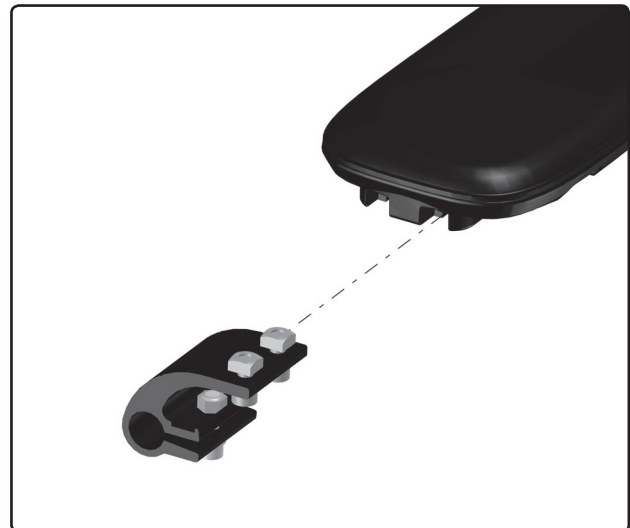


Fig. 1. Mounting the bracket.

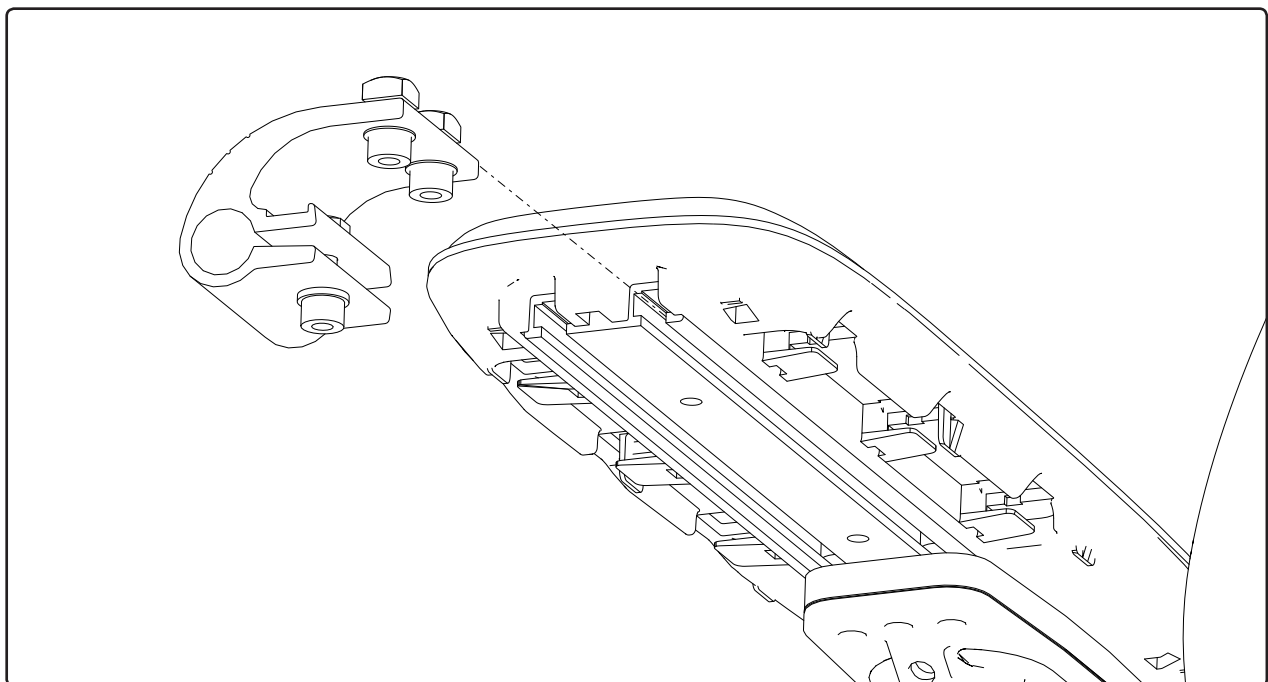



Fig. 2. Mounting the bracket.

Assembly Instruction

Document No. 246554	Revision A.00				
Production state					
Preform Date 2011-01-12	Preform By UDN	Approved Date	Approved By	Page 2	

Montering av panelhållare

Best.nr: 1825815 *Panelhållare*

Förteckning över detaljer som skall finnas i satsen.

3st	M6 Mutter	2st	Skruv ISO 4762 M6x12
1st	Skruv ISO 4762 M6x20	3st	Bricka ISO 7089 6,4x12x1,6
1st	Panelhållare		

Montering

Läs igenom anvisningen *innan* montering påbörjas.

1. Montera panelhållaren på armstödet genom att skjuta in spårmuttrarna i önskat spår på armstödetts undersida. (*Bild 1*)

Val av spår och vilket håll panelhållaren monteras, är beroende av användning och önskemål om tillbehörets placering.

2. Justera panelhållaren till önskat läge och dra åt skruvarna.

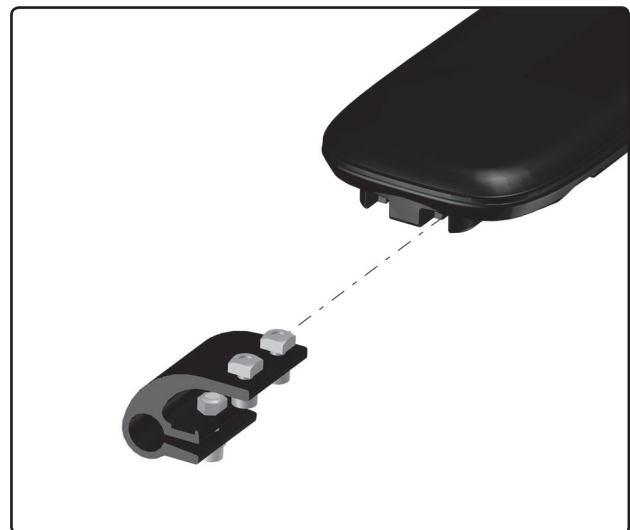


Bild 1. Fästets montering.

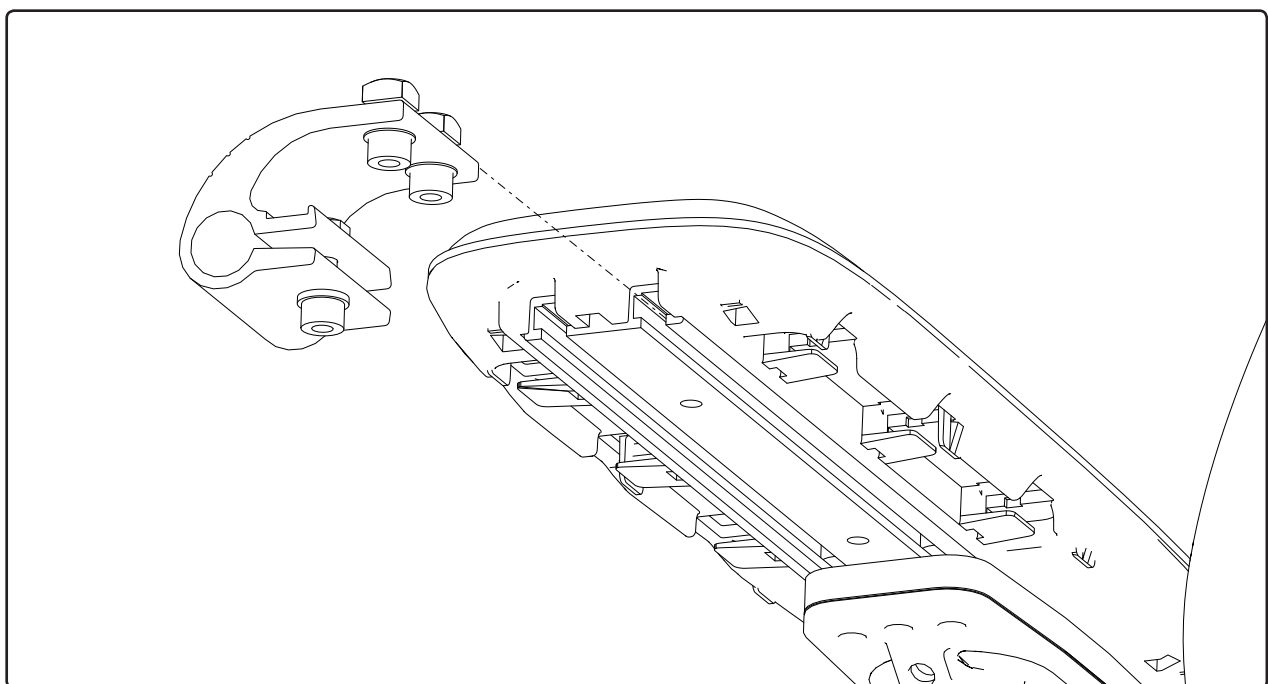


Bild 2. Fästets montering.