



Assembly instruction — Luggage Box X850


- 1 Turn the top cover knobs counter-clockwise with a 5 mm Allen wrench.

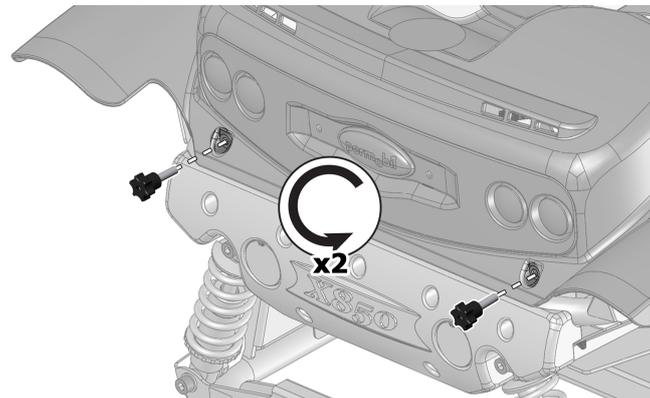


- 2 Pull out the top cover from the chassis.

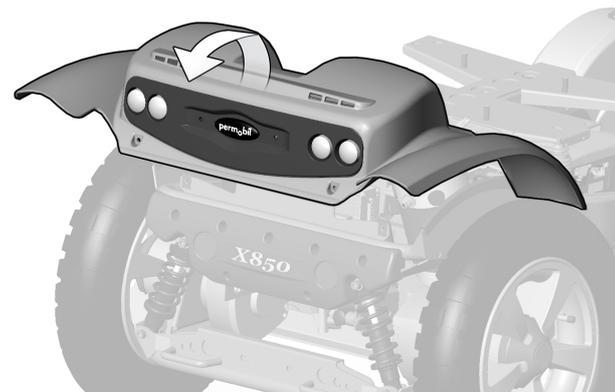


- 3 Disconnect the light cable from the electronics and release the cable from the cable holder.

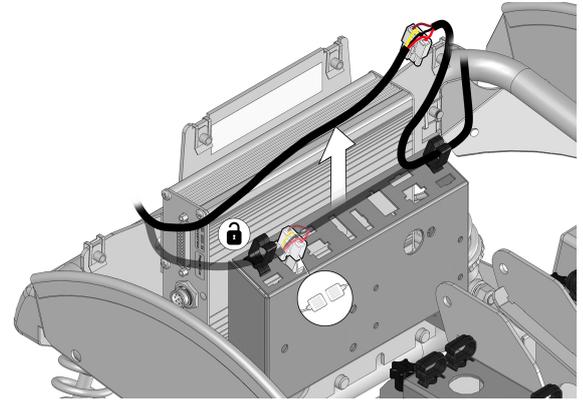


- 4 Connect the adapter cable to the light cable.

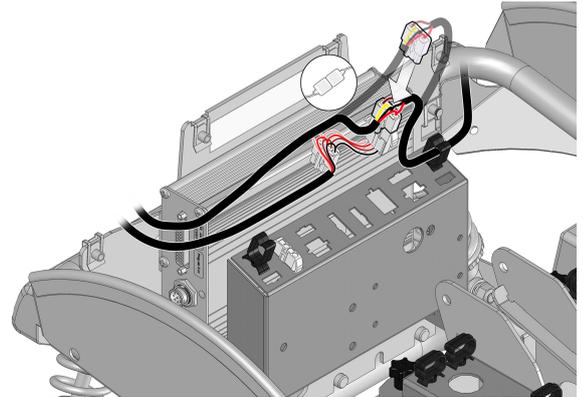


- 5 Connect the adapter cable contact into the electronics box.

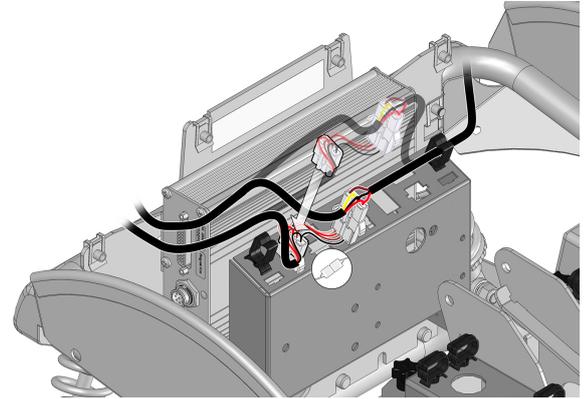


- 6 Lock the two cables into the cable holder.

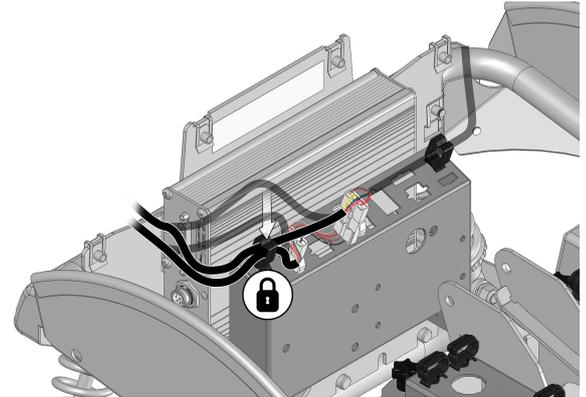


- 7 Run the cable through the third ventilation hole from the left.

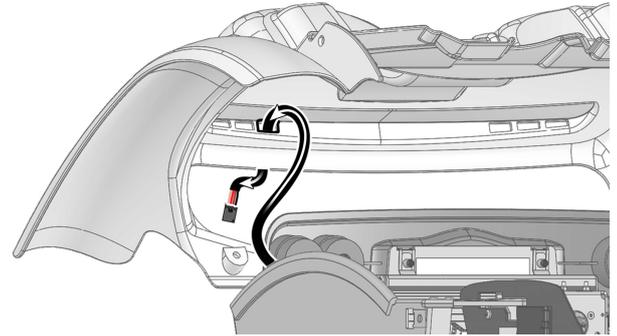

- 8 Push the top cover back on the chassis.

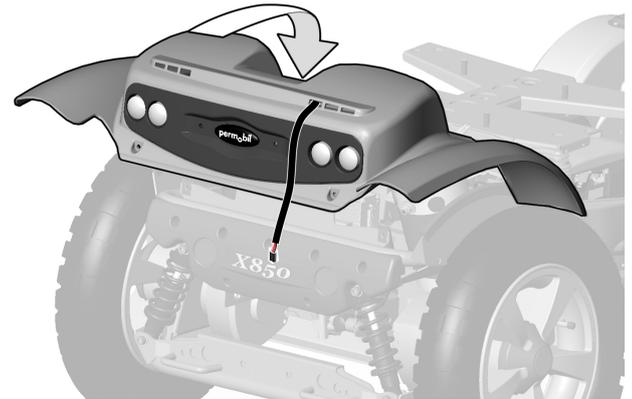

- 9 Turn the two cover knobs clockwise with a 5 mm Allen wrench to attach the top cover.

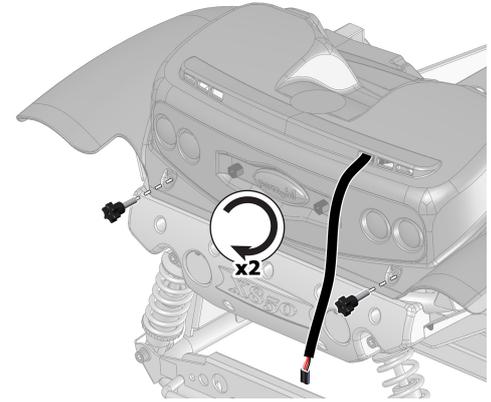


- 10 Remove the three screws and washers and put them into a recycle bin.




- 11 Turn the distance screws clockwise (torque 9.8 Nm) into the holes in the top cover.




- 12 Push the box holder bracket against the top cover and attach it with the three included screws. Turn the two small screws with washers with torque 9.8 Nm. Turn the long screw into the lower hole on the box cover with torque 9.8 Nm.

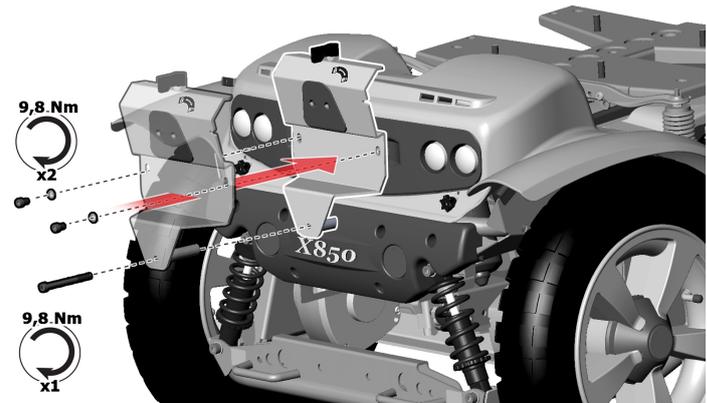

- 13 Push the box plate against the box. Turn the four screws through the upper and lower holes on the box plate and into holes 2 and 6 on the luggage box. Turn the four screws with torque 2,9 Nm.

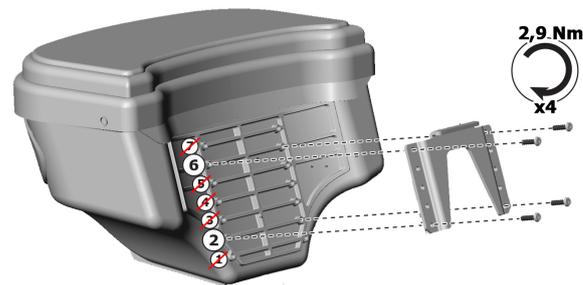


- 14 Push the luggage box down onto the box holder bracket.

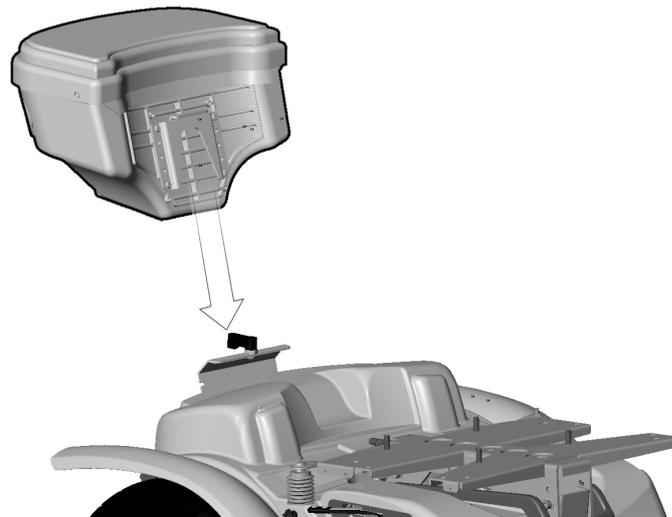


- 15 Lock the luggage box to the chassis by turning the lock lever to the locked position.




- 16 Fold the box fabric cover up to expose the luggage box cable.

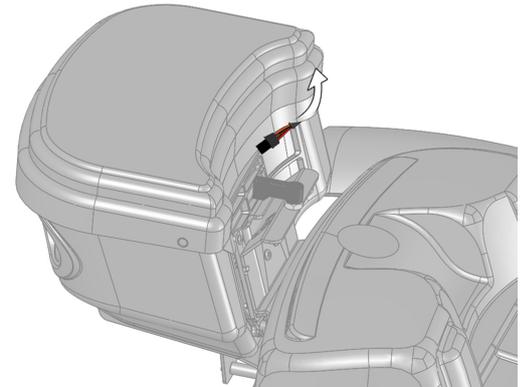

- 17 Run the adapter cable along the back of the box towards the box cable.

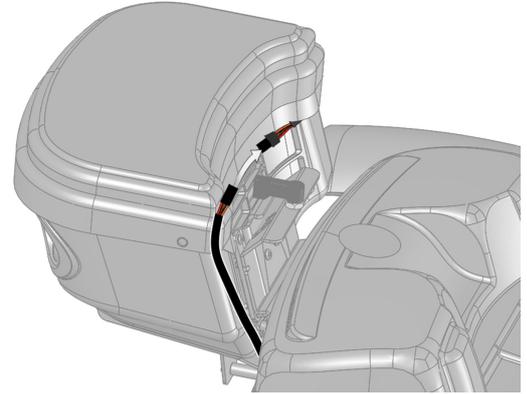

- 18 Connect the adapter cable to the box cable.

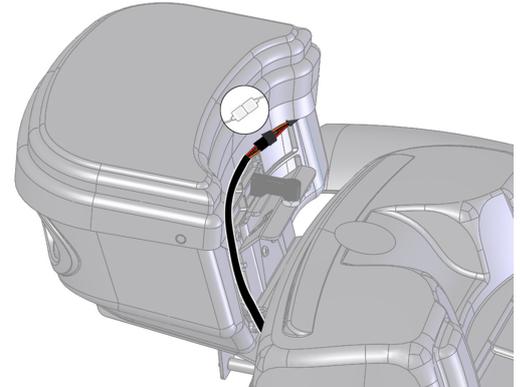

- 19** Attach the cable into the cable slot on the box holder bracket.
Fold the fabric down over the cable.

